



REBOTEC Rollators and Walkers Instructions

[Home](#) » [REBOTEC](#) » REBOTEC Rollators and Walkers Instructions



REBOTEC® Rehabilitationsmittel GmbH
PO Box Q541
Queen Victoria Building
NSW 1230
rebotec.com.au
info@rebotec.com.au



WA-No.: 02.11.149/2017-06

Contents [[hide](#)]

1 Maintenance Instructions

1.1 Rollators and Walkers

1.2 Maintenance of the rollator and walker

2 Maintenance protocol No.:

3 Maintenance overview

4 Documents / Resources

4.1 References

5 Related Posts

Maintenance Instructions

Rollators and Walkers



Images show a selection of rollators and walkers

Important advisory!

Store the maintenance inspection reports carefully. This will allow you to keep an overview of the product's maintenance status.

Maintenance of the rollator and walker

1.1 Maintenance / inspection requirements

In this maintenance instructions pamphlet, you'll find information and advisories on the safe and professional maintenance and/or repair work on the rollator and walker.

The products are **Class I** medical products as to the **93/42/EEC** and the German **MPG/Medical Products Act**.

As to the requirements of the German MPBetreibV Medical Products Ordinance, the products must be maintained and the hygiene requirements must be adhered to. Check your local requirements.

The product's lifetime is in large part affected by how it is used. The frequency of use and manner of use reduces the lifetime as does improper handling and inadequate maintenance.

In order to ensure safe use, a visual inspection and function check must be done at least once per year. Shorter time intervals between maintenance may be necessary when the frequency of use or the condition of the product exists due to safety reasons.

The operator has to carry out the maintenance on his own or must instruct a competent person (specialized supplier, medical supply store, or medical maintenance service) if he has not the needed expertise.

Inspection results must be documented and be confirmed by a signature. The product is recommended with an inspection sticker that states the next inspection. The product operator is responsible for assuring regularly conducted inspections.

1.2 Maintenance scope

The product's inspection extends from the general condition to the individual components, function and safety.



Necessary repair works must be done without delay. If immediate repair work is not possible, then it is in the patient's interest and health to put the product out of service.

1.3 Maintenance advisories

When making repairs, use only original spare parts from REBOTEC.

Do not undertake any modifications or conversions to the product on your own. These will adversely impact the product's safety and function. REBOTEC accepts no liability in such cases.

When performing maintenance and repairs, please use the product's User Instructions booklet as an aid and follow the instructions concerning the proper use, re-use, lifetime, guarantee, care, and maintenance.

After completing maintenance or repair work on the product, follow the requirements for hygiene for re-use.

1.4 Maintenance reports and overview

In the appendix, you will find a form for documenting maintenance. You can make copies of the form. This allows you to keep an overview of the product's maintenance status.



Maintenance protocol (single report) It is a form for proof of a single maintenance session.

Maintenance overview (total report) It is a form for proof of executed maintenance sessions.

1.5 Disposal

For disposing of the product, follow all the advisories in the product's User Instructions booklet. Disposal must be done at a recycling company. Contact your local recycling company for more information.

1.6 Transport and storage

For transporting or storing the product follow the instructions of the particular user manual.

Advisory on other documents

At our www.rebotec.de website, you can download the User Instructions booklets, forms, and other information.

Maintenance protocol No.:

Product information / Rollator, Walker (medical product class I)

Product						
Serial no.		Year of		Use		
Manufacturer						
User						
Location of						

Maintenance- General Condition

P	Checks	Scope of work	Remarks	Finding*
1	Visual inspection	General damage, wear and corrosion		
2	Connection elements	Completeness, seat, function (screws, nuts, etc.)		
3	Functions	Folding mechanism		
		Height adjustment		
		Brake (permanent lock/normal brake)		

Maintenance- Component checking

P	Article	Scope of work	Remarks	Finding
1	rollator frame	Damages		
2	grip bow	Function, fastening		
3	swivel castors brake wheels	Function, damage, wear		
4	brake wheels	Function, damage, wear		
5	brake levers	Function, damage, wear		
6	hand switch	Function, damage, wear		
7	wood seat	Stability, damage, wear		
8	cushion	Damage		
9	accessory	Function, wear		
10	YANO-support	Function, fastening		
11	YANO-handle	Function, damage, wear		
12	Castor brakes	Function, damage, wear		
13	YANO-accessory	Function, damage, wear		

* Findings: I=inspected; E=exchanged' R=repaired, i.o. = in order; n.i.o. = not in order

The following parts were replaced: (Use only original parts from the manufacturer)

Qty.	Article description

Maintenance results/summary Next maintenance:

Visual check	<input type="checkbox"/> i.O. <input type="checkbox"/> n.i. O	Function check	<input type="checkbox"/> i.O. <input type="checkbox"/> n.i.O	Passed	<input type="checkbox"/> yes	<input type="checkbox"/> no
Cleaning carried out		<input type="checkbox"/> yes <input type="checkbox"/> no	Disposal carried out		<input type="checkbox"/> yes	<input type="checkbox"/> no

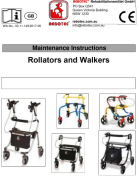
Maintenance overview

Product information / Rollator, Walker (medical product class I)

Product					
Serial no.		Year manuf.		Use start	
Manufacturer					
User					
Location of use		Inventory no.			

1st Maintenance Date	Maintenance company	Report	Signature
Remarks:			
2nd Maintenance Date	Maintenance company	Report	Signature
Remarks:			
3rd Maintenance Date	Maintenance company	Report	Signature
Remarks:			
4th Maintenance Date	Maintenance company	Report	Signature
Remarks:			
5th Maintenance Date	Maintenance company	Report	Signature
Remarks:			

Documents / Resources

	REBOTEC Rollators and Walkers [pdf] Instructions Rollators and Walkers
---	---

References

-  [Rebotec Australia](#)
-  [REBOTEC Rehabilitationsmittel GmbH](#)