

# **REFINE A7 Apex Locator Instruction Manual**

Home » Refine » REFINE A7 Apex Locator Instruction Manual

#### **Contents**

- 1 REFINE A7 Apex Locator
- 2 Intended Use
- 3 Precautions
- 4 Contraindication
- **5 Package Contents**
- **6 Component**
- 7 Installation
- 8 Operation
- 9 Audio Alarm Volume Control
- 10 Maintenance
- 11 Charging Battery
- 12 Regular Maintenance Checks
- 13 Trouble Shooting
- 14 Specifications
- 15 Classification of Equipment
- **16 Declaration of Conformity**
- 17 Symbols
- 18 Warranty
- 19 Disposing of Product
- **20 EMC Information (Electromagnetic Compatibility Information)**
- 21 Documents / Resources
  - 21.1 References
- **22 Related Posts**



## **REFINE A7 Apex Locator**



For your safety and the safety of your patients please read this user manual carefully before use and file for future reference. This manual is published by the Manufacturer. We do not guarantee its contents and reserves the right to amend at anytime without prior notice, amendments will be published in new editions of this manual.

## https://www.alandental.com

#### **Intended Use**

#### Intended use

determination of the apical foramen position and measurement of the root canal length. The product is only to be used in dental surgeries by qualified dental personnel.

#### **Precautions**

- All precautions should be read and understood before use.
- Equipment is only to be used for its specified intended use.
- Safety instructions are provided in order to prevent the risk of personal injury or damage to the device and are classified as below by the level of potential risk.

**WARNING:** Indicates a hazard that may result in serious injury/device damage if instructions are not correctly followed.

**CAUTION:** Indicates a hazard that may result in mild to moderate injury/device damage if instructions are not correctly followed.

#### **WARNING**

- Use this product by its intended use and proper method of use.
- This product is not waterproof. Avoid water or chemical solutions on the control unit as it may cause electric shock due to a short circuit.
- The scale indication on the screen does not represent a distinct length or distance in mm or other linear units. It simply indicates the file's progression towards the apex.

- Do not expose to or dispose of the battery in a fire.
- Be sure to prevent the lip hook, file clip, file probe, and their connector parts from having contact with household power supply sources (such as electrical outlets) as it may cause an electric shock.
- The components in the product package are delivered in a non-sterile condition, be sure to sterilize the file clip, file probe, and lip hook by autoclave sterilization before use and after each patient.
- Do not operate close to patients with cardiac pacemakers as there is a danger that it may affect the pacemaker.
- Keep away from explosive substances and flammable materials.

#### **CAUTION**

- Do not keep using the product when the battery indicator " is flashing. Normal operation or indication may not be performed. Please recharge the battery.
- Should the product function abnormally during operation, cease operation immediately.
- Do not use the product by connecting or integrating into other medical devices.
- Do not drop or allow an impact on the product. This may result in personal injury or damage to the unit.
- Avoid using chemical solutions on the lip hook, file probe or file clip during procedures. Use of solutions may cause inflammation.
- When gripping the metallic part of a file or reamer with the file clip, grip the upper part (near the handle). If the lower part (blade transition part and blade part) is gripped, the root canal length cannot be correctly measured and the tip of the file clip may be broken.
- Do not use or leave the product in a high-temperature environment such as under strong direct sunlight, or next to equipment that produces heat as it may cause overheating or fire due to a failure of the internal circuit.
- Do not attempt to disassemble the product nor tamper with the mechanism except as recommended by the manufacturer in this User manual.
- This device is for indoor use only.
- · Keep the control unit on a level surface.
- If the product is not used for a long period check it is functioning correctly before using it on a patient.
- Portable and mobile RF communications equipment can affect Medical Electrical equipment. Do not use RF equipment near the product.
- During operation the Apex Locator may interfere with computers, LAN cables or may cause noise in radio receivers nearby.
- Installation and use of this product requires special precautions regarding EMC according to the EMC information.
- Use only original accessories with this device.
- The apical position is indicated on the screen with "00". To avoid over-instrumentation, it is suggested to subtract 0.5mm from the reading when determining the working length for shaping.
- Always dry the cavity entrance with a cotton pellet in order to obtain an accurate measurement.
- Users are responsible for the operational control, maintenance and continual inspection of this product.
- The battery can be replaced, please contact our distributor if a replacement battery is required.

#### Contraindication

The Apex Locator is not recommended for use:

- In patients who have a pacemaker or other implanted electrical devices or have been cautioned by their physicians against the use of small electric appliances such as shavers, hair dryers etc.
- In Patients allergic to metals.
- Children.

## **Package Contents**

Apex Locator is composed of a control unit, AC adapter, Plug adaptor, measuring wire, lip hook, file clip, file probe.

## **Accessories**




## **Accessories list**

No.	Name	Quantity	Consumable?	Can be sterilized?
а	File clip	4 PCS	YES	YES
b	Measuring wire	1 PCS	YES	NO
С	Battery	1 PCS	NO	NO
d	Lip hook	4 PCS	YES	YES
е	File probe	4 PCS	YES	YES
f	AC adapter	1 PCS	NO	NO


## Component

#### **Control Unit**

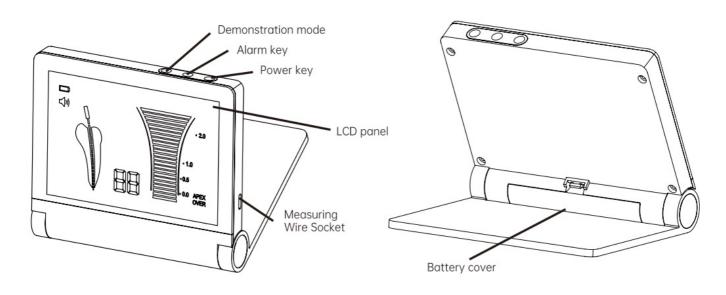

LCD Panel	Displays the position of the file tip, remaining battery level and alarm sound volume		
Power key	When the Power key is pressed, the power turns on sounding an alarm, then the LCD panel lights up, press the Power key again, the power and the LCD panel turns off.		
Alarm key	When the Alarm key is pressed, the alarm audio volume can be adjusted (rot ation of OFF->Low->Medium->High).		
Measuring wire socket	Socket to which the measuring wire is inserted.		
Battery cover	Secures battery in place.		
Charging light	The light will remain on when battery is charging		
Demonstration mode	Press the button to demonstrate the measurement process		
Charging socket	Socket to which the AC adapter is inserted		

## **LCD Panel**

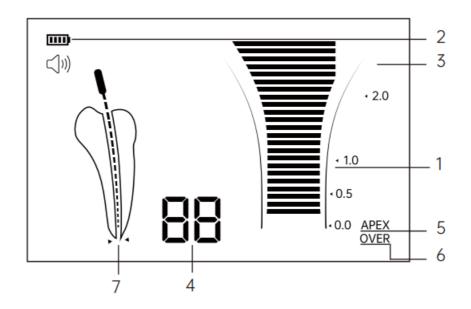

Displays the approximate position of the end of the file		
Batter indicator	Displays the remaining battery level. When the Battery indicator flashes, immediately recharge the device with AC adapter	
Alarm indicator	Displays the alarm audio volume (rotation of OFF ■4 ->Low ■4 ->Medium ■4 > ->High ■4 >) )	
Number display	Displays the present position from the end of the root canal in numerical value. When the value displays "10" to "05", an alarm corresponding to each value sounds. When the value reaches "04" to "00", a short alarm sounds with "APEX" displayed on the LCD panel. When the values reaches "-1" to "-3", a shorter alarm sounds with "OVER" displayed on the LCD panel	
Apex indicator	Turns on when the value representing the present position of the file end reaches "00"	
OVER Display	Turns on when the value representing the present position of the file end reaches "-1"	
Measurement indicator	Indicates the files position in the root canal	


Displays the approximate position of the end of the file

4 is not a value to show the actual distance from the end of the root canal in the unit of mm. It is to be used as indication for measurement.

## Installation

Bar araph

- 1. Connect the measuring wire: securely insert the plug of the measuring wire into the Measuring wire socket on the unit.(Fig.2)
- 2. Connect the file clip: connect the plug of the file clip to either plug of the measuring wire. (Fig.3)
- 3. Connect the lip hook: connect the lip hook to the other plug of the measuring wire. (Fig.3)

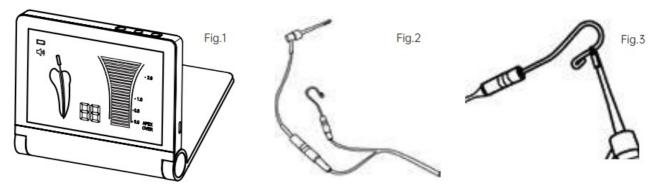

- 4. Make the lip hook touch the bent section of the file clip (Fig.4), the screen will display"OVER",(as show in Fig.5 c), otherwise, it means the file clip or the measuring wire is damaged, and should be replaced.
- 5. Display explanation
  - "10 to 05", Green bar graph and/or low-frequency sound: File has reached the front region of the Apex;
  - "04 to 00", Yellow bar graph and/or middle-frequency sound: File is very close to the Apex;
  - "-1 to -3", Red bar graph and/or high-frequency sound: File has exceeded the Apex.

#### **Display Screen explanation**







- The file has reached the front region of the apex
- The file is very close to the apex
- · The file has already exceeded the Apex

#### **Demonstration mode**

Demonstration mode tracks the movement of the file

- Pull out the measuring wire and adapter.
- Turn on Apex Locator
- Press " button for 2 seconds to enter the demonstration mode
- Demonstration mode will be exited by finishing the demonstration or pressing the demonstration mode button.

## Operation


#### **Preparation**

- 1. The use of apex locators alone, without a preoperative and postoperative radio graph, is not a recommended practice as apex locators may not work properly in all conditions. It is recommended to take an X-ray prior to the use of the Apex Locator, to compare the information given by both devices.
- 2. The dentist should have a good understanding of the tooth and root canal in question
- 3. The root cavity should be sufficiently exposed and the root should be unblocked.
- 4. The largest file that can correctly reach the apex should be selected.
- 5. Avoid contact between the file, file clip, and the gingiva or any metal crown and bridge appliances. If the crown of the tooth is broken and there is a possibility of the gingiva contacting either the file, file clip or a probe, an incorrect reading may occur. An isolating barrier around the rim of the broken tooth must be created before proceeding with the Apex location.
- 6. Dry canals should be treated with an irrigation solution such as saline or hydrogen peroxide. Access to the cavity should then be air-dried or wiped dry with a cotton pellet.
- 7. If there is bleeding from the root canal or apical foramen, it must be stopped before a correct measurement can be taken.
- 8. Canals must be cleaned of all remnants after root canal treatment before measurements can be taken.
- 9. Accessories such as file clips, lip hooks, and file probes should be clean and free from any chemical disinfectant or medical solution residue.
- 10. Mark the treated tooth and record the information on the patient's medical records. Be careful to make sure the tooth is marked in the practical and most resilient part of the tooth.
- 11. The root canal must be cleared of any pulp or necrotic tissue and there must be no inflammation or infected material surrounding the apex.

- 12. The following cases are not for use with the apex locator.
  - The measurement length of the canal may be shorter than its real length due to root hypoplasia and thus a true reading is not possible
  - · A crack in the root may allow electric leakage thus affecting the accuracy of the reading
- 13. An X-ray shot at a difficult or unusual angle may sometimes cause the illusion that the file tip has not reached the apex. Results of the Apex Locator and X-ray may possibly not correlate falsely indicating that the file tip has not reached the root canal tip. (Fig. 6)

## **Operating Procedure**

- 1. Grip the file inserted in the root canal with the file clip. Grip the upper part (near the handle) of the file's metallic part. (Fig.7)
- 2. Hang the lip hook on a corner of the patient's mouth. (Fig.8)

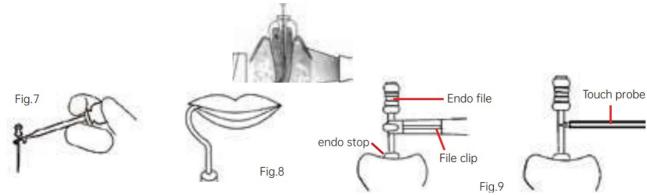

- 3. Insert the file into the canal and push it slowly towards the Apex.A continuous alarm will sound when the file is less than 2mm from the apex. Gloves must be worn to avoid contact of the operator's skin and metal shank of the file.
- 4. The "APEX" will be indicated when the screen displays "00" but as previously stated 0.5mm should be subtracted from the reading so as to not over instrumentate.
- 5. When the file reaches this point, adjust the endo stop and remove the file. By measuring the distance between the endo stop and the tip of the file, the working length of the canal can be determined.
- 6. The file probe may also be used instead of the file clip to touch the file when working on the posterior to determine the working length of the canal. (Fig.9)
- 7. After use, press the Power key for approximately one second to turn off the power (the alarm sounds and the LCD panel turns off). The device will shut down automatically after 5 minutes if unit is not in operation.
- 8. Remove the file from the file clip.
- 9. Remove the lip hook and file clip from the measuring wire.
- 10. Remove the measuring wire from the control unit.

**CAUTION:** Never hold the measuring wire when removing the lip hook and file clip from the measuring wire. Always hold the connector section.

#### **Audio Alarm Volume Control**

Alarm audio volume can be adjusted to "OFF ■ 1", "Low ■ 1", "Medium ■ 1", and "High ■ 1")".

1. press the Alarm key.

- 2. The Alarm Indicator on the LCD panel and the sound volume will change.
- 3. Each time the key is pressed the sound volume changes.

#### **NOTICE**

The last setting is stored when the control unit is switched off.

#### **Maintenance**

#### Cleaning

Preparation prior to cleaning

- 1. Remove the file clip and lip hook from the measuring wire.
- 2. Remove the measuring wire from the control unit.
- 3. Check for damage on each cord or deformation on each connector.

#### Cleaning

- 1. Rinse accessories thoroughly with clean suitable water, then wipe clean with alcohol-immersed cotton or cloth.
- 2. Repeat until accessories are visibly clean.

#### Note:

- Clean and sterilize the accessories before each use to prevent any contamination. This includes the first use as well as any subsequent use.
- Accessories that should be cleaned include: measuring wire, file clip, lip hook, file probe.
- · Do not use highly acidic water to immerse or clean .

## **CAUTION:**

Non observance of the following precautions could lead to the deterioration or failure of the accessories. Be sure to follow these precautions when cleaning the accessories.

- When cleaning the product never use solvent such as benzine or thinner.
- Do not use a chlorinated cleaner.
- Do not clean the product with an ultrasonic cleaning apparatus.
- For your own safety, please wear personal protective equipment (gloves, glasses, mask)
- After cleaning the measuring wire, make sure to dry the connector part of the measurement wire.

#### Sterilization of the file clip, lip hook and file probe

Note: Sterilize file clip, file probe and lip hook by autoclave sterilization, other parts Apex Locator can not be sterilized.

#### **Autoclave Procedure**

- 1. Insert into an autoclave pouch.
- 2. Seal the pouch.
- 3. Sterilize at 134oC (273oF) for 10 mins and a minimum drying cycle of 30 mins.
- 4. The product should remain in a sealed pouch until required for use.

#### **CAUTION**

- The product must be cleaned before sterilization.
- Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product.
- Do not use Autoclaves exceeding 138oC during sterilization.
- We recommends sterilization according to EN13060, class B. Always follow the autoclave manufacturer's instructions for use.
- Do not touch the product immediately after autoclaving as it will be very hot and must remain in a sterile condition.
- Reprocessed products should be stored, and protected from dust with minimum exposure to germs in a dry, dark and cool place.
- Autoclave sterilization is the only agreed method to correctly sterilize this product. The validity of other sterilization methods is not confirmed or guaranteed.
- Resistance to reprocessing: file clip: 200 cycles, other accessories have no cycle limitation, but should be replaced when no longer in usable condition. (reprocessing cycles include cleaning and sterilization).

## **Charging Battery**

Do not use the Apex Locator whilst the indicator is flashing. Charge the battery as below:

- Connect the AC adapter to the Charging Socket of the device, then insert the AC adapter plug into a supply power socket.
- 2. The Apex Locator will take approximately 2-3 hours to fully charge.

#### **Attention**

- a. Orange light indicates charging and will turn green once the unit is fully charged.
- b. Please do not use the device when it is charging.
- c. We provide a plug adaptor for certain target markets, if the AC adapter can not match the supply power socket, you can insert the plug of AC adapter into the Plug adaptor, then insert the Plug adaptor into the supply power socket.

#### **CAUTION**

- · Avoid shorting the battery.
- Do not disassemble or tamper with the battery.
- Use the AC adapter provided by the manufacturer (complies with IEC 60601-1) to charge the device, never use a modified or damaged charger.
- Batteries will discharge over time if Apex Locator is not used. Always recharge if the unit has not been used for extended periods.
- Only the NCM 14500 DC 3.6V/850mAh Rechargeable Li-ion Battery Cells which have passed the IEC 62133 certification can be used.
- If you need to replace battery please contact us or your dealer.

#### How to install the battery:

Open the battery housing

- Insert battery connector into the square notch
- Make sure it is firmly installed by gently pulling the battery
- · Reinstall the battery housing

Note: the square notch is an anti-mistake design, if the polarity is reversed, the battery can not be installed.

## **Regular Maintenance Checks**

Regular maintenance should be performed every 3 months as per the below chart. If any abnormalities are found, immediately contact our authorized dealer or us.

Points to check	Process
ON/OFF operation	Check that the power turns ON and OFF correctly
Remaining battery level	Check that the Battery indicator does not flash. If the displayflashes, recharge the battery following the instruction in "10.Charging battery"
Alarm sound volume	Press the Alarm key and check that the alarm audio volume changes. (rotation of OFF->Low->Medium->High)
Connector part	Check for debris or corrosion on the lip hook or connector terminals of the cable

Product operation	Check with the tester, that the cable and the control unit operate properly, following the instruction in "6. Check before Treatment and Installation"
-------------------	--

· See trouble shooting chart if problems are discovered

## **Trouble Shooting**

When a problem is detected, check the following again before requesting a repair

Malfunction	Cause	Solution	
	Battery is not inserted	Insert battery	
The power does not tu	Battery is not correctly insert ed	Correctly insert the battery	
	The battery level is low	Recharge the battery	
Root canal length mea surement cannot be performed	The measuring wire or other connectors are not properly c onnected.	Securely insert the connector Connect the lip hook and file clip and make the lip hook touches the bent section of file clip to check whether the device has been connected correctly.(refer to 6. Check before Treatment and Installation.)	
Alarm sound volume is low  The alarm sound volume is a djusted to "OFF".		Check the alarm sound volume.	
The LCD panel does n ot display	The battery level is low	If the LCD panel does not display after the battery is ch arged, failure of the LCD panel is suspected	

	The lip hook is not firmly in cont act with the membrane of the p atient's oral cavity	Adjust the lip hook position so that it correctly contacts the membrane in the oral cavity
	Perforation of the canal or an a djacent surface has caries	Remove the file, close the perforation and repair the carie s, then repeat the apex detection procedure, carefully inserting the file in canal
	Large lateral canal	Try continuing the procedure by gently advancing the file
	The file is in contact with the gingiva	When the file contacts the gingiva the entire bar graph will light be lit up. Check if the file is in contact with gingiva
Bar graph is not stable	The file is in contact with a metal prosthesis.	When the file contacts a metal prosthesis, the measured c urrent flows to the gingiva or periodontal tissues and the b ar graph moves. Check if the file has contacted a metal pr osthesis
	Current leakage to the gingiva i s occurring due to a major colla pse of the crown	Form a matrix around the tooth to prevent current leakage to the gingiva

Bar graph is not stable or is damaged		Replace or clean the file clip	
	The root canal is closed	Bar graph operates correctly when the file reaches the a pical constriction. In this case, always check in combinati on with X-ray photography	
Bar graph does not mo ve	The inside of the root canal is extremely dry.	Moisten the root canal with a saline solution	
	Bad electrical contact	Perform the cable connection test as described in 6. Che ck before Treatment and Installation	
	The connection hook of file clip is not properly connect ed to the file	Place the connection hook on the metal part of the file below the plastic handle	
Bar graph does not mo ve	In the case of retreatment: residue from old filling mate rial may be blocking the root canal	Remove old root filling material residues prior to measuring	

	Root canal may be blocked by r emnants of medical products  (e.g. calcium hydroxide)	Rinse the root canal with NaCl solution. Dry the access cavity with a cotton pellet/air-blower	
Bar graph does not mo ve	The selected file is too small for a large root canal	Use the largest file possible for the canal to produce the most accurate result	
	Electronic malfunction	Contact your distributor or us	
	Short circuit due to excess liqui d(irrigation solution, saliva or bl ood) in the pulp chamber	Dry the access cavity with a cotton pellet/air blower. In case of excess bleeding, wait until it has stopped	
Display indications are incorrect, i.e. unit displ ays that "APEX" is rea ched before it has	A direct contact of the file with the gingiva proliferations or metal restorations (crown, amalgam filling)	For isolation:  A) Do adequate preparation filling.  B) Use a rubber dam or isolate the file by placing 2-3 silicone stoppers on it	

If none of these are applicable or if the trouble is not fixed after the appropriate action has been taken, a failure in this product is suspected. Contact our Authorized Dealer.

# **Specifications**

Model	Apex Locator
Input (charge the battery)	DC 5V 1A

AC Adapter: UES06WZ-XX	XYYYSPA	100-240V AC 50Hz/60Hz		
Battery: NCM 14500 Rechargeable Li-ion Battery		DC 3.6V/850 mAh		
Working Voltage		DC 3.6V		
Working Current		DC: 0.13-0.14A		
Rated Power		£0.5W		
Measurement Voltage		AC 200mV		
Measurement Current AC		AC 100μA		
Screen		5.1" LCD ( 110 mm x 70 mm )		
Display		Reflective colour LCD display		
Alarm		The beeper will sound when the file is less than 2 mm from the apex		
	Dimensions	L130 mm × W112 mm × H23.5 mm		
Control Unit	Weight	370 gms ( including battery )		
Use Environment	Temperature :5°C - +40°C	Humidity: 30%-75% RH	Atmospheric pressure: 70kPa-106kPa	

## **Classification of Equipment**

- Type of protection against electric shock:
  - · Class II internally powered equipment
- Degree of protection against electric shock:
  - Type BF applied part
- Method of sterilization recommended by the manufacturer:
  - See "9-2 Sterilization of the film clip, lip hook, and file probe"
- Degree of protection against ingress of water as detailed in the current edition of IEC 60529:
  - Control unit: IPX0

- Degree of safety of application in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide:
  - Equipment is not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.
- · Mode of operation:
  - Continuous operation

## **Operation Principle**

The lip hook, file clip, and file probe are used as electrodes and are attached to the patient's mouth and the operating instrument such as a file. The motion of the instrument in the root canal causes animpedance variation between the pair of electrodes. The position of the apical foramen is detected by measuring the impedance variation using two different frequencies

## **Declaration of Conformity**

We declare that the dental apex locator complies with the following normative documents: EN 60601-1; EN 60601-1-2; EN 80601-2-60; EN 62304; EN 60601-1-6; ISO 10993.

## **Transportation and Storage Environment**

Keep away from environmental conditions including but not limited to harmful chemicals like acids and alkali. Temperature:  $-20^{\circ}\text{C} - +40^{\circ}\text{C}$ , Humidity: 10% - 93%, Atmospheric Pressure: 70kPa - 106kPa

## **Symbols**



(( <u>@</u> ))	Marking on the outside of Equipment or Equipment parts that include RF transmitters or that apply RF electromagnetic energy for diagnosis or treatment			
Z	Following the waste of electric and electronic equipment (WEEE) Directive (2002/96/EC) for product and accessory disposal			
IPX0	Symbol for "Protection from ingress of liquids into device"			
<b>†</b>	Type BF applied part Follow operating instructions			
•••	Manufacturer	-20°C	Temperature limitation	
EC REP	Authorized representative in the European community	①	Power key	
SN	Symbol for "Serial Number"	►II	Demonstration mode	
LOT	Batch code		Direct current	

## Warranty

Our products are warranted against manufacturing errors and defects in materials. We reserve the right to analyze and determine the cause of any problem. Warranty is voided should the product be not used correctly or for the intended purpose or has been tampered with by unqualified personnel or have had nonparts installed. Replacement parts are available for seven years beyond the discontinuation of the model.

## **Disposing of Product**

To avoid the health risks of operators handling of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon or a dentist is required to confirm the equipment is sterile. Following the waste of Electric and Electronic Equipment (WEEE) Directive (2002/96/EC) for product and accessory disposal. If you have any queries, please refer to the local authorities responsible for waste disposal.

#### **NOTES**

Please follow local authority guidelines for the disposal of batteries and used equipment. To protect the environment, dispose of empty batteries at appropriate collection sites according to national or local regulations. Dispose of batteries at public collection points in the EU countries-2006/66/EC

## **EMC Information (Electromagnetic Compatibility Information)**

Guidance and manufacture's declaration – electromagnetic emission			
The product is intended for use in the e the Apex Locator should be assured that	_	ment specified below. The customer or the user of environment.	
Emission test	Compliance	Electromagnetic environment – guidance	
RF emissions CISPR 11/EN55011	Group 1	The product uses RF energy only for its internal function. Therefore, its RF emissions are very I ow and are not likely to cause any interference in nearby electronic equipment.	
RF emission CISPR 11/EN55011	Class B		
Harmonic emissions EN/IEC 61000- 3-2	Class A	The product is suitable for use in all establishm ents, including domestic establishments and th ose directly connected to the public low-voltage power supply network that supplies buildings u	
Voltage fluctuations/flicker emissions I EC 61000-3-3	Complies	sed for domestic purposes.	

Guidance and Manufacture's Declaration – Electromagnetic Immunity				
The product is intended for the Apex Locator should be	<del>_</del>	-	ified below. The customer or the user of nt.	
Immunity test	IEC 60601 test lev	Compliance level	Electromagnetic environment – guida nce	
			The floors should be wood,	
Electrostatic discharge (ES D) IEC 61000-4-2	±6 kV contact ±8 k V air	±6 kV contact ±8 k V air	concrete or ceramic tile. If floors are c overed with synthetic material, the rel ative humidity should be at least 30%.	
Electrical fast transient/ bur t IEC 61000-4-4	±2 kV for power su pply lines ±1 kV for input/ out put lines	±2kV for power sup ply lines	Mains power quality should be that of a typical commercial or hospital envir onment.	
Surge IEC 61000-4-5	± 1 kV line(s) to lin e(s) ± 2 kV line(s) to ea rth	±1 kV differential m ode	Mains power quality should be that of a typical commercial or hospital envir onment.	
Guidance and Manufacture	s Declaration – Electro	magnetic Immunity		
The product is intended for the Apex Locator should be			ified below. The customer or the user of nt.	
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – gui dance	

Compliance level

dance

Immunity test

Voltage dips, short interr uptions, and voltage vari ations on power supply i nput lines IEC 61000-4-1 1	<5% UT (>95% dip in UT)for 0.5 cycle 40% UT (60% dip in UT)for 5 c ycles 70% UT (30% dip in UT)for 25 cycles<5% UT (>95% dip in UT)for 5 sec	<5% UT(>95% dip in UT) for 0.5 cycle 40% UT(60% dip in UT) for 5 cycles 70% UT(30% dip in UT) for 25 cycles<5% UT (>95% dip in UT) for 5 s ec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the A7 r equires continued operation during power mains interruptions, it is recommen ded that the A7 be powered from a n uninterruptible power supply or a battery.
Power frequency (50Hz/60Hz)magnetic field IEC 61000-4-8	3 A/m	3.15 A/m	Power frequency magnetic fields s hould be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE UT is the a.c. mains voltage before application of the test level.

# Guidance and Manufacture's Declaration – Electromagnetic Immunity

The product is intended for use in the electromagnetic environment specified below. The customer or the user of the Apex Locator should be assured that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance	
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% UT (>95% dip in UT)for 0.5 cycle 40% UT (60% dip in UT)for 5 cycles 70% UT (30% dip in UT)for 25 cycles<5% UT (>95% dip in UT)for 5 sec	<5% UT(>95% dip in UT)for 0.5 cycle 40% UT(60% dip in UT) for 5 cycles 70% UT(30% dip in UT) for 25 cycles<5% UT (>95% dip in UT)for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the A7 requires continued operation during power mains interruptions, it is recommended that the A7 be powered from an uninterruptible power supply or a battery.	
Power frequency (50Hz/60Hz)magnetic field IEC 61000-4-8	3 A/m	3.15 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.	
NOTE UT is the a.c. mains voltage prior to application of the test level.				

			$d=12\sqrt{P}$ 80 MHz to 800 MHz $d=2,3\sqrt{P}$ 800 MHz to 2,5 GHz
diated RF IEC 00-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m 80 MHz to 2 .5 GHz	Where P is the maximum output power rating of the transmitter in watts(W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey (a), should be less than the compliance level in each frequency range (b) Interference may occur in the vicinity of equipment marked with the following symbol:

- **NOTE 1** At 80 MHz and 800 MHz, the higher frequency range applies.
- **NOTE 2** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.
- a: Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcasts, and TV broadcasts cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the product is used exceeds the applicable RF compliance level above, the product should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the product.
- b: Over the frequency range of 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended Separation Distances Between Portable

and Mobile RF Communications Equipment and the Product

The product is intended for use in an electromagnetic environment in which radiated RF disturbances are contro lled. The customer or the user of the product can help prevent electromagnetic interference by maintaining a mi nimum distance between portable and mobile RF communications equipment (transmitters) and the product as r ecommended below, according to the maximum output power of the communications equipment.

	Separation distance according to frequency of the transmitter (m)		
Rated maximum output power of transmitter (W)	150 KHz to 80 MHz $d = 12\sqrt{P}$	80 MHz to 800 MHz $d = 12\sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3\sqrt{P}$

0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

**NOTE 1** At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. **NOTE 2** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

## After service and warranty instruction

- 1. Period validity: Five years' free repair for the main unit, and one year's free repair for the handpiece from the date of purchase. Lifetime maintenance.
- 2. Range of warranty: Within the warranty period of validity, we are responsible for any troubles caused by quality problems or product technique and structure.
- 3. The following are beyond our warranty:
  - 1. The damage caused by disobeying the operation instructions or lack of the needed condition.
  - 2. The damage is caused by unsuitable operation or disassembly without authorization.
  - 3. The damage caused by unadvisable transportation or preservation.
  - 4. There isn't the seal of the distributor or the warranty card isn't filled in completed.

#### **After service and Warranty Instruction**

- Name of Customer
- Address
- Post Code
- Tel
- E-mail
- Purchase Date
- Distributor
- Model
- · Product No.
- · Handpiece No.
- · Production Date
- <a href="https://www.alandental.com">https://www.alandental.com</a>

#### **Documents / Resources**



# **REFINE A7 Apex Locator** [pdf] Instruction Manual A7, A7 Apex Locator, Locator, A7 Locator, Apex Locator

## References

• User Manual

## Manuals+, Privacy Policy

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.