

REHASENSE SERVER Light Weight Rollator



# REHASENSE SERVER Light Weight Rollator User Manual

[Home](#) » [REHASENSE](#) » REHASENSE SERVER Light Weight Rollator User Manual 

## Contents

- 1 REHASENSE SERVER Light Weight Rollator
- 2 Product Usage Instructions
- 3 Frequently Asked Questions
- 4 General Information
- 5 Delivery & general product description
- 6 Security requirements
- 7 Product details
- 8 Usage
- 9 Cleaning
- 10 Label Example
- 11 Technical data
- 12 Transport description
- 13 Maintenance
- 14 Warranty
- 15 Documents / Resources
  - 15.1 References
- 16 Related Posts



**REHASENSE SERVER Light Weight Rollator**



## Specifications

- **Product:** Light Weight Rollator
- **Intended Usage:** Assisting walking for individuals with mobility problems
- **Features:** Push handle, Brake handle, Seat, Locking mechanisms, Shopping bag, Turning wheel

## Product Usage Instructions

### General Information

This user manual provides important guidelines for the correct and safe use of the rollator. It is crucial to read and understand this manual before using the product to ensure safety.

### Intended Usage, Indications & Contraindications

- **Intended Usage:** The rollator is designed to assist individuals with mobility issues in standing, walking, and sitting independently. It should only be used for its intended purpose.
- **Indications:** Suitable for individuals requiring stable support during walking due to various health conditions.
- **Contraindications:** Not suitable for individuals with severe balance issues or insufficient muscle strength to walk even with support. Usage should be recommended by a healthcare professional.

### Delivery & General Product Description

- The rollator is delivered folded in a box for convenient transportation. Contents include a complete folded rollator and a user's manual.

### Security Requirements

- All users must read and follow the user manual to prevent any damage or injuries. In case of a serious incident, contact the dealer or Rehasense immediately.

## **Product Details**

- Push handle
- Brake handle
- Seat
- Locking mechanisms
- Shopping bag
- Turning wheel

## **Usage**

Refer to the provided image for an overview of the rollator's components.

### **Follow these steps to prepare the rollator for use:**

1. Unfold the rollator by pushing the seat tube.
2. Adjust the push handle height using the locking knob.
3. To fold, pull the lock, and release the pulling strip.
4. Ensure all parts are securely in place before use.

## **Frequently Asked Questions**

### **Q: Can the rollator be used outdoors?**

**A:** Yes, the rollator can be used both indoors and outdoors, providing support on various terrains.

### **Q: How do I clean the rollator?**

**A:** Use a damp cloth with mild soap to clean the rollator. Avoid harsh chemicals that may damage the product.

## **General Information**

**Dear Customer,**

- This user manual contains a description of the medical device and important guidelines to ensure the correct and safe use of the product. It is important to read this manual carefully before use.
- It is especially important to read the safety requirements and follow these.
- REHASENSE continuously develops its products and reserves the right to change the specifications and functions of products without notice.
- If you have any queries please contact your dealer or Rehasense directly. Contact information is located on the last page of this manual.

## **Intended Usage, Indications & Contraindications**

- **Intended usage:** The rollator has been designed as a tool to assist walking for people who have mobility problems, for example with standing, walking, or sitting independently.
- Thanks to the rollator these people can stand, walk, and rest easily and they can fulfill most of their daily tasks and routines without assistance from others. Any other usage is prohibited.
- **Indications:** The device is specifically designed for individuals who, due to a wide variety of possible health problems, require additional stable support during walking.
- **Contraindications:** The device must not be used by users who have serious problems with body balance or whose muscle strength is too low to be able to stand and walk, even with the extra support of a rollator.
- The need and possibility of usage of a rollator by the user should always be assessed and indicated by a doctor or physiotherapist.

## **Delivery & general product description**

- The rollators are supplied in boxes and come folded, for easier transportation.

### **The contents are:**

- 1 complete, folded rollator
- 1 user's manual
- The Server is a four-wheel, lightweight rollator with a cross-foldable frame. The product is made of aluminum, so it is light and easy to store or transport.
- The rollator offers great maneuverability, and stability and its push handles are height adjustable for a good user fit.
- The product consists of a powder-coated foldable structure frame made of aluminum round tubing and a foldable cross. It is also equipped with a fabric seat and a shopping bag which allows the user to rest and to carry small items.

## **Security requirements**

Make sure that this user manual is read by all people who use the device. The manufacturer does not take any responsibility for damages and/or injuries caused by the fact that the user's manual has not been followed.


### **Limitations for usage**

- The device must not be loaded with more than 150 kg.
- The device's bag has a load limitation of 5 kg.
- The device is devised to be used indoors and/or outdoors on flat surfaces.
- The device is not intended to be self-propelled while seated
- The brakes **MUST** be in the locked position **BEFORE** using the seat.


### **Avoiding accidents**

- Use the device only for the purpose it is intended for.
- Use the device only in good technical condition.

- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- During different adjustments that are possible on the device, the user must take care not to trap their fingers or other body parts between two moving elements of the device
- The device must only be used on a stable surface.
- All wheels MUST be in contact with the floor at ALL TIMES during use. This will ensure the rollator is properly balanced and should avoid accidents.
- When using the rollator in a stationary position the hand brakes MUST be locked.
- IF ANY SERIOUS INCIDENT OCCURES concerning THE DEVICE PLEASE CONTACT YOUR DEALER OR REHASENSE DIRECTLY (contact details at the end of the user manual).
- Immediate actions should be taken according to the requirements of the Regulation of the European Parliament and of the EU Council 2017/745 on medical devices of April 5, 2017.
- Be careful when you see this symbol on the device. Fingers or other body parts of a user and/or assistant can be trapped between the device's parts during the folding, unfolding, or adjusting of the device.
- Be extremely careful when adjusting the device. Observe the dimensions of gaps between the device's elements to avoid trapping your fingers or other body parts.

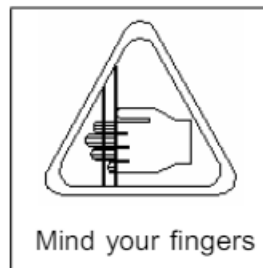

## Product details

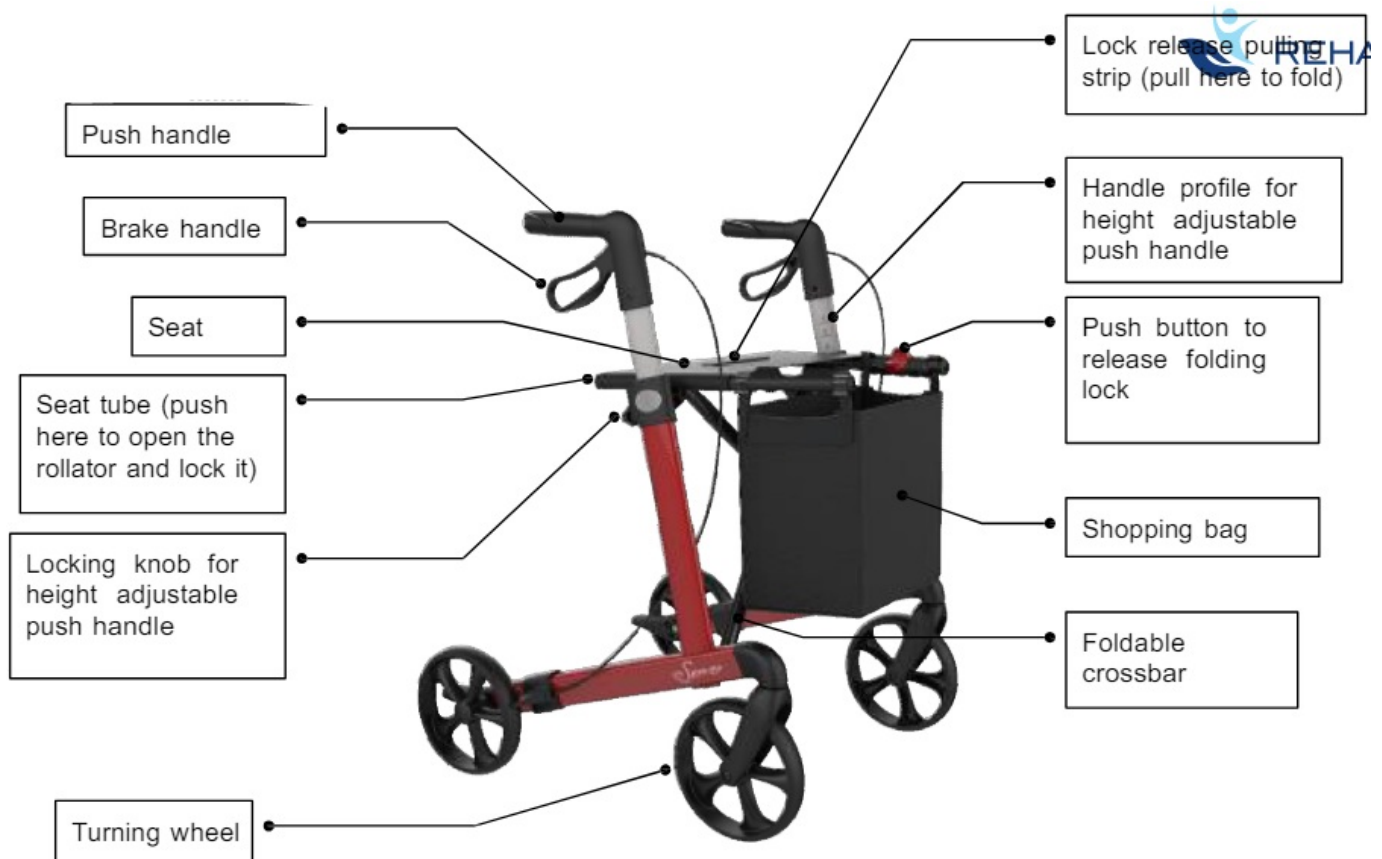

## Usage

- Looking at the picture above the device should be prepared for use in an instinctive way.
- However, we recommend you follow the steps below.

### Initial Check

- In the first instance take the device out of the box and check (using the picture above) that nothing is missing.
- If you discover at this stage that some part is missing or damaged do not continue to assemble the product.
- Contact your supplier of the product right away.

### Opening of the rollator

- Push the red button which locks the seating tubes
- Hold both seat tubes and simultaneously push them down until you hear a “click” sound of the locking mechanism of the frame.
- Try to pull the seat tubes back up to fold the frame. If this is impossible to do the frame is opened and secured correctly. See the OPEN position in the picture above.



- After unfolding or assembling the rollator always ensure that the seat support is securely locked by the pulling strip mechanism for securing the lock and that the frame does not move anymore.
- Always ensure that both push handle tubes are securely locked in the main support tube with the two wing knobs to prevent the accidental sliding in of the push handle tubes when forced.
- Always test to see that accessories are properly and safely locked in place BEFORE using.
- Failure to follow these warnings can result in personal injury.

### Folding

- Lift the pulling strip on the seat to fold the rollator.
- Fold the rollator until the lock closes.

### Adjusting the height of the push handles

- Release the handle locking knob,
- Make adjustments to your desired height.
- Tighten the knob.
- Both push handles should be adjusted to the same height.
- The correctly adjusted handles should be to the height of the user's hips. Never leave the device without the brakes on.

## **Using the hand brakes**

- TO STOP THE ROLLATOR simply pull the brake handles up towards the hand grips.
- TO REMAIN STATIONARY hold the brake handles in the "UP" position.
- TO START MOVING release the brake handles. They will drop down automatically and the brakes will be released.
- You should always engage the breaks when you don't plan to move the device.
- Never leave the device without the breaks engaged when someone is sitting in it!
- Never leave the device on slopes without the breaks engaged!
- Never sit too much to the rear. Do not sit (partly) above the shopping bag.
- Always sit straight up when seated on the rollator with your forearms or hands lightly resting on the push handles.
- Never lean backward when seated on the rollator

## **Locking/Unlocking the hand brakes**

- To lock the brakes push the brake handles down until you can hear the "click" sound.
- To unlock the brakes pull the brake handles up until you feel them loosen.
- When using the rollator in a stationary position, the hand brakes **MUST** always be locked.

## **Cleaning**

### **General:**


- Clean the device regularly and keep it clean.
- Lighter dirt and dust can be removed with a wet cloth.
- Larger parts of dirt can be removed with a non-corroding, mild washing agent, and warm water.
- Do not use harsh abrasives or bleach-based cleaners, as this may cause damage or discoloration.
- Dry parts with a soft cloth.

### **Wheels:**

- Clean wheels with warm water and a mild cleanser. Dry with a clean rag.
- Lubricate with multi-purpose oil periodically or if wheels begin to squeak.

## **Label Example**

**The product label is placed on one of the bottom tubes of the main frame:**

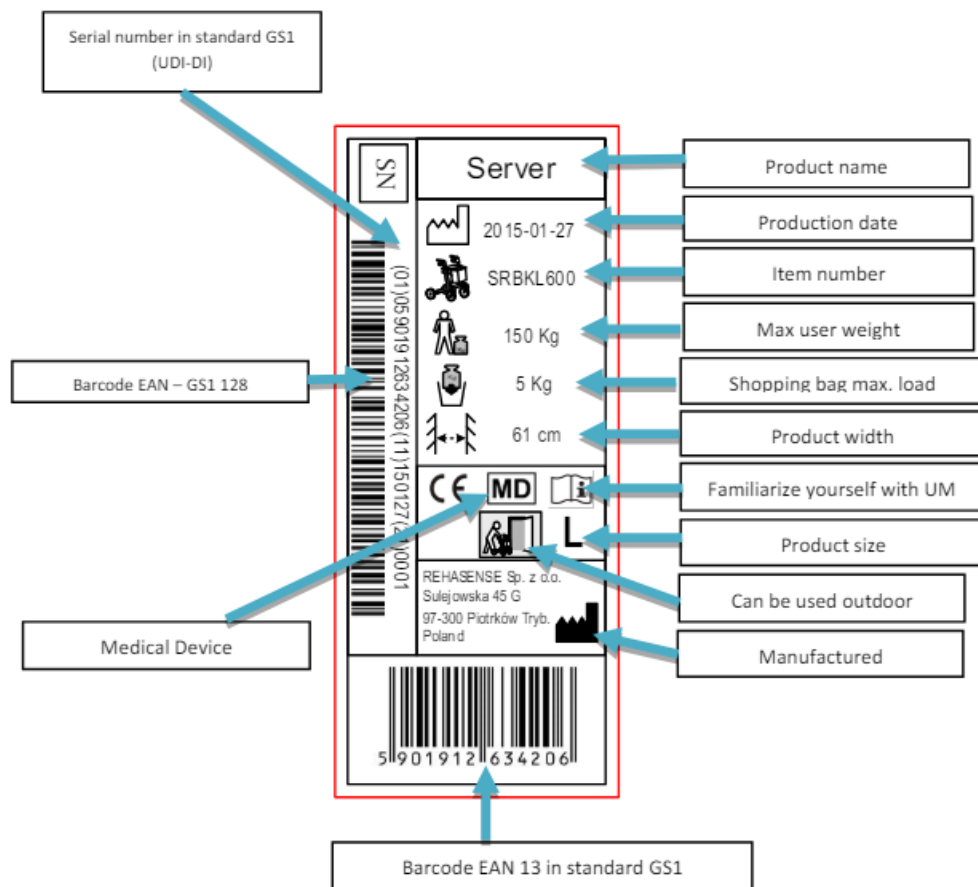

## Technical data

“SERVER” lightweight rollator	
Max. load	150 kg
Max. load of the bag	5 kg
Total weight with accessories	7 kg & bag 0,3 kg
Length	68 cm
Width	Opened 60 cm / Folded 24 cm
Seat Width	60 cm
Height of hand grips (adjustable)	S, M, L resp.: 63 – 79 ; 66 – 86; 74 – 102 cm
Height of seat	S, M, L resp.: 50, 55, 62 cm
Brakes	2 x handbrake with locking function

## Transport description

- The rollator is an easily foldable product, convenient for transportation. If it is necessary to transport it we recommend folding it as stated in the description above.
- The rollator should be transported in a way that it is stable and there is no risk of sudden and dangerous movements of the transported device.



## Maintenance

- Ensure that every part of the rollator is secure at ALL times. Check all screws for tightness. If not, tighten if necessary. Replace any broken, damaged, or worn items immediately.

## Re-use

- The walker is suitable for re-use. Before passing on to another user it must be subject to an inspection by the specialist dealer.
- **The following points have to be checked:** Visual inspection of the frame  
Secure seating of nuts and bolts  
Condition of the plastic elements
- Condition of wheels and ball bearings
- Effectiveness and condition of the brakes
- When one of the parts is damaged or broken it must be replaced by a new one. It is recommended to replace the wheels whenever a change. All loose bolts and nuts must be tightened.
- The product must be thoroughly cleaned and disinfected before reuse.
- **WARNING!** It is forbidden to re-use the rollator when the frame is bent or broken. The instruction manual must accompany the rollator with every change.


## How to deposit the product

- The device should not be deposited with normal garbage. It should be delivered to the deposit station for reuse or recycling.

## Warranty

- The guarantee is not valid by mis-intended use of the product.
- The guarantee excludes damage caused by incorrect operation, accidents, and/or through normal wear and tear.
- Any modifications to the product will cause a loss of guarantee. Mechanical damages are not covered by this warranty.
- For repairs during the guarantee period, please contact your local dealer

## Documents / Resources

	<p><a href="#">REHASense SERVER Light Weight Rollator</a> [pdf] User Manual SERVER Light Weight Rollator, SERVER, Light Weight Rollator, Weight Rollator, Rollator</p>
---	--

## References

- [User Manual](#)

## **Manuals+. Privacy Policy**

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.