



ResMed Pixi Hint Fitting Mask User Guide

[Home](#) » [ResMed](#) » ResMed Pixi Hint Fitting Mask User Guide 

RESMED

Pixi Hint Fitting Mask
User Guide

Fitting Guide



Pixi Hint! If fitting the mask for the first time, we recommend the child be sitting in an upright position to ensure the correct fit.



- Hold the mask gently against the child's face, checking that the nose is fully enclosed by the cushion and the mouth is clear.



- Pull the headgear over the head and move the middle strap down until it sits across the middle of the back of the head.



- Keeping your hand on the frame to maintain the correct headgear position, bring the lower headgear strap behind the back of the neck.



- Attach the lower headgear strap to the mask by holding the tab and hooking the quick release clip onto the lower headgear arm.



- If necessary, adjust the straps by removing the Velcro® and gently tightening the headgear straps until comfortable. Do not overtighten.
- For optimal results, adjust the straps starting at the bottom strap and work upwards.



- Connect one end of the air tubing to the swivel and the other end to the airflow device.
- Turn on the airflow device.

Contents

[1 Tip](#)

[2 Documents /
Resources](#)

[3 Related Posts](#)

Tip



- When fully fitted, ensure the top strap is not overtightened and vertically aligned with the top of the ears; the middle strap should sit across the back of the head horizontally aligned with the top of the ears.



For details, refer to the
Pixi Paediatric Mask User Guide.

Combine the Pixi mask

with the latest Stellar or VPAP devices for a therapy system that delivers greater comfort, reliability, and effective care.




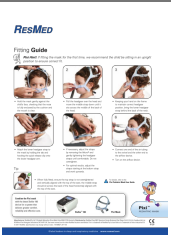
Manufacturer: ResMed Pty Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia. Distributed by: ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA.

EC REP ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK. See www.resmed.com for other ResMed locations worldwide.
Pixi is a trademark of ResMed Pty Ltd and Velcro is a trademark of Velcro Industries B.V. © 2011 ResMed.



Specifications may change without notice. 1014891/1 2011-07 0120
Global leaders in sleep and respiratory medicine www.resmed.com

Documents / Resources

	ResMed Pixi Hint Fitting Mask [pdf] User Guide Pixi Hint Fitting Mask
	ResMed Pixi Hint Fitting Mask [pdf] User Guide Pixi Hint Fitting Mask

