



## ResMed Swift Fx Nano Mask User Guide

[Home](#) » [ResMed](#) » ResMed Swift Fx Nano Mask User Guide 

# RESMED

### Fitting Guide

Ensure the bottom of the cushion opening aligns with the bottom of your nose then place the mask over your nose.



Then pull the headgear over your head so it sits midway between ears and eyes.



Adjust the top strap first by pulling the strap through the buckle, one notch at a time, until the mask is comfortably over your nose.  
Do not overtighten.



The back strap should sit just above the ears at the back of the head.  
Adjust both fasteners on the fabric back strap at the same time.  
Ensure mask is comfortably sealed.



Position the back strap over or under your hair to suit your hairstyle preference.



Adjust the nasal cushion until it sits comfortably on your nose.

Connect the free end of the short tube assembly to the air tubing of your device, then turn it on.



Check for leaks and adjust if needed. If any air is leaking out then readjust the back strap before readjusting the top strap.

Your mask is now ready to use.

#### **Optional Soft Wraps**



Wrap each of the soft wraps around either side of the headgear, folding the top fastener over the lower one. The soft wraps should be positioned on your cheeks, with the fasteners on the outside.

**Note:** For additional information, refer to the Swift FX Nano / Swift FX Nano for Her Nasal Mask User Guide.

**Swift™ FX Nano for Her  
NASAL MASK**



**Distributed by:**

ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA.  
ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire  
OX14 4RY UK. See [www.resmed.com](http://www.resmed.com) for other ResMed locations worldwide.  
Swift is a trademark of ResMed Pty Ltd and is registered in U.S. Patent and  
Trademark Office. © 2013 ResMed.  
Specifications may change without notice. 1015935/1 2013-01  
**ResMed Pty Ltd**



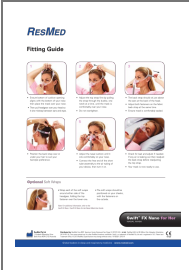
1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia  
Global leaders in sleep and respiratory medicine [www.resmed.com](http://www.resmed.com)

Contents

1 Documents / Resources

2 Related Posts

**Documents / Resources**

	<p><a href="#">ResMed Swift Fx Nano Mask</a> [pdf] User Guide</p> <p>Swift Fx Nano Mask</p>
---	---