



RESMED Swift LT Nasal Pillows System User Guide

[Home](#) » [ResMed](#) » RESMED Swift LT Nasal Pillows System User Guide 

RESMED

Swift™ LT
NASAL PILLOWS SYSTEM

Contents

- [1 Fitting Guide](#)
- [2 Optional](#)
- [3 Documents / Resources](#)
 - [3.1 References](#)
- [4 Related Posts](#)

Fitting Guide



- Holding the pillows at your nose, pull the headgear over your head.
- The headgear should sit midway between the ears and eyes.



- Adjust the top straps by pulling them evenly until comfortable.
- Do not overtighten.



- Repeat Step 2 with the back straps.
- Do not overtighten.



- Using the rotating barrel, position the nasal pillows at the most suitable angle.
- Connect the Swift LT short tube to the device air tubing.
- Turn on the device.



- The Swift LT is now fitted and ready for use.

Optional



- For alternate tubing positions, use the tube retainer.
- To attach, push it into the top headgear buckle.
- Clip-in either the short tube or device air tubing, depending on flexibility needs.
- The tubing can be positioned to either side of your face or centered.
- If the pillows pull on your nose, adjust the tubing at the tube retainer.



For details, refer to the Swift LT Nasal Pillows System User Guide.

ResMed Corp Poway, CA, USA +1 858 746 2400 or 1 800 424 0737 (toll free), ResMed Ltd Bella Vista, NSW, Australia +61 (2) 8884 1000 or 1 800 658 189 (toll free). Offices in Austria, Brazil, China, Finland, France, Germany, Hong Kong, India, Japan, Malaysia, Netherlands, New Zealand, Norway, Singapore, Spain, Sweden, Switzerland, United Kingdom (see website for details). Protected by patents: AU 785376, DE 20321468.4, DE 20321469.2, DE 20321470.6, DE 20321471.4, DE 20321472.2, EP 1314445, NZ 553756, NZ 553757, NZ 553822, NZ 553824, NZ 553825, US 6581594, US 6823865, US 7159587, US 7318437. Other patents are pending. Protected by design registrations: AU 301943, CN 200430102469.3, JP 1243026, NZ 407161, US D552731. Other designs are pending. Swift is a trademark of ResMed Ltd and Swift is registered in U.S. Patent and Trademark Office.

© 2008 ResMed Ltd. Specifications may change without notice. 1012455/1 08 03

Global leaders in sleep and respiratory medicine

www.resmed.com

Documents / Resources

	RESMED Swift LT Nasal Pillows System [pdf] User Guide Swift LT Nasal Pillows System
	ResMed Swift LT Nasal Pillows System [pdf] User Guide Swift LT, Nasal Pillows System, Swift LT Nasal Pillows System
	RESMED Swift LT Nasal Pillows System [pdf] User Guide Swift LT, Nasal Pillows System, Swift LT Nasal Pillows System

References

- [Sleep apnea and COPD - learn about symptoms and treatment | ResMed](#)