




Roche ACCU-CHEK Instant Wireless Blood Glucose Monitoring System User Guide

[Home](#) » [Roche](#) » Roche ACCU-CHEK Instant Wireless Blood Glucose Monitoring System User Guide 



ACCU-CHEK Instant Wireless Blood Glucose Monitoring System User Guide



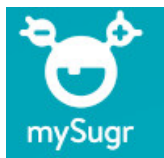

WARNING

The Accu-Chek Instant meter is intended for patient self-monitoring by an individual person on the target range on App.

Contents

- 1 PAIRING THE ACCU-CHEK INSTANT METER TO THE MYSUGR® APP
- 2 CHANGING THE TARGET RANGE ON YOUR ACCU-CHEK INSTANT METER
- 3 LOG AN ENTRY
- 4 Documents / Resources
 - 4.1 References
- 5 Related Posts


PAIRING THE ACCU-CHEK INSTANT METER TO THE MYSUGR® APP



On the mySugr homepage go to the “connections” option on the bottom right of the screen and select “Accu Chek Instant” then select the “Connect now” button.

Turn the meter off if it
ou see the Bluetooth (

3

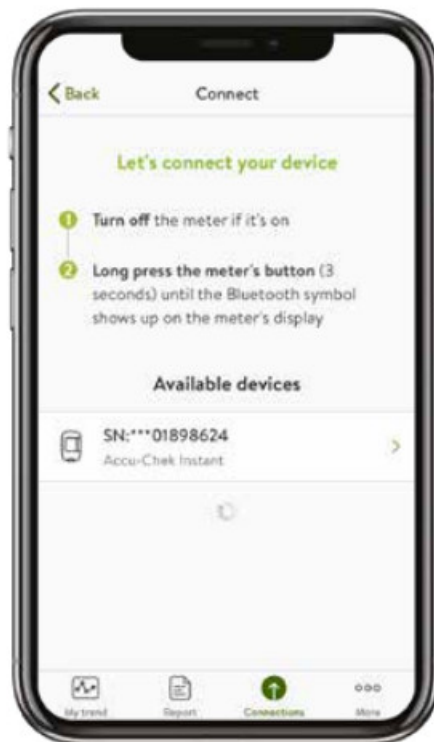

4



You will then see the serial number of the Accu-Chek Instant meter displayed in the mySugr app. Select the meter to confirm pairing.

Enter the pin number

5



6



After you have entered the Pin select “pair” and the pairing will be completed.

You have now successfully paired the Accu-Chek Instant meter with the mySugr app. You will need to perform a test to ensure the meter is working correctly.

Download the mySugr app to set up your account.




CHANGING THE TARGET RANGE ON YOUR ACCU-CHEK INSTANT METER

Once your Accu-Chek Instant meter has been paired to the mySugr app, you are able to change the target ranges on your meter via the mySugr app.


Start by opening the mySugr app.

To confirm the new TRI on the Instant meter, please perform another BG test.

1

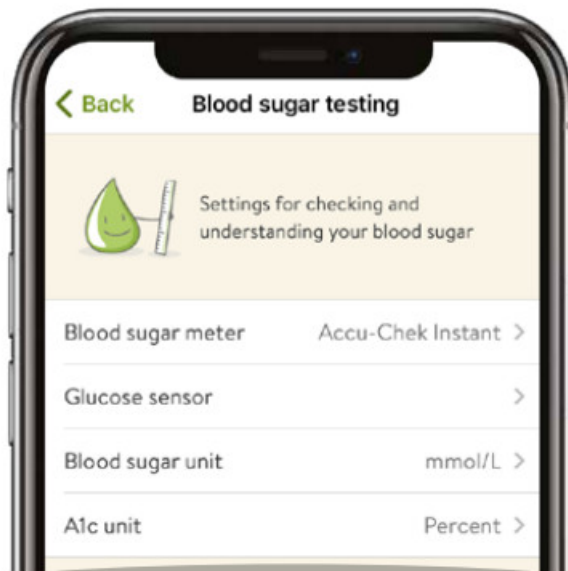

2

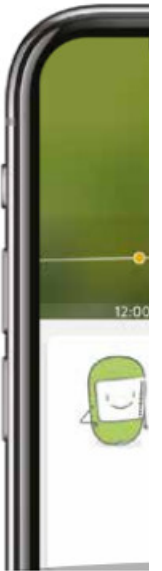

Select “More” on the bottom right of the home screen then select “Profile Settings” then “Blood sugar testing”.

Select your “Hyper” “Target range” and “Hypo” and scroll up or down to select the required range.

Once settings have been saved, a confirmation message will appear.

3




You can also receive a confirmation notification when your meter has been updated to match the target range on the app.

LOG AN ENTRY



mySugr Bolus Calculator is licensed for people with diabetes over the age of 18 years.

mySugr logbook is licensed for people with diabetes over the age of 16 years.

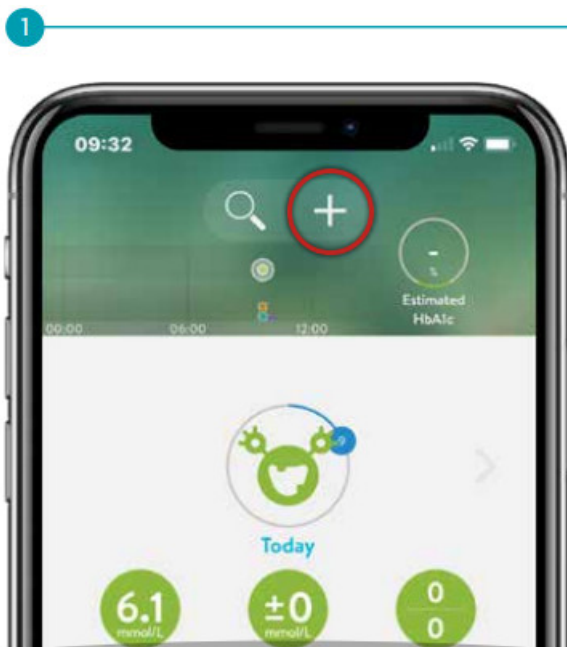



Tap the 'Plus' button on your screen.

Set a reminder.

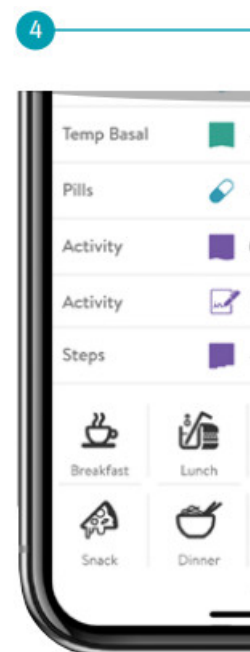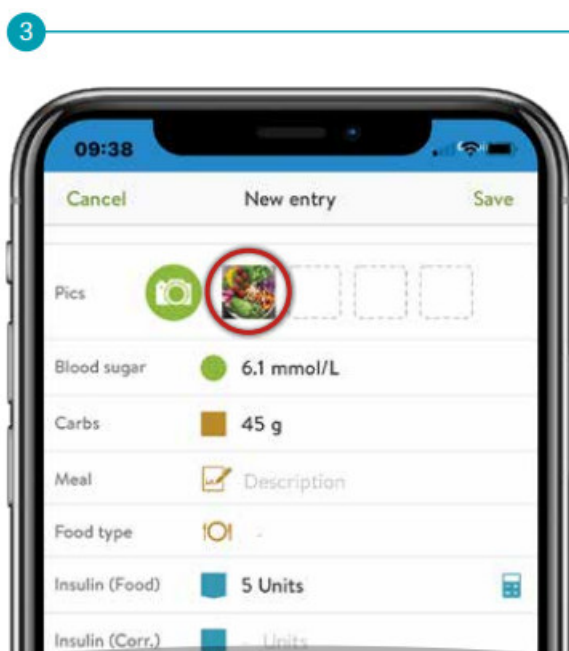

Take a picture of your food.

Enter values for your blood glucose, carbs,
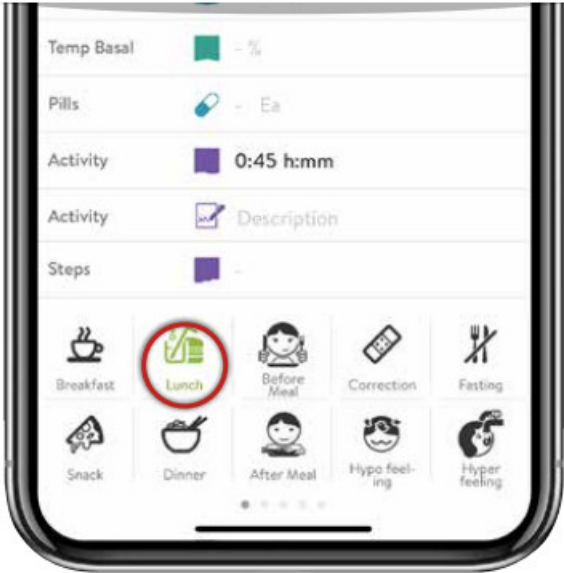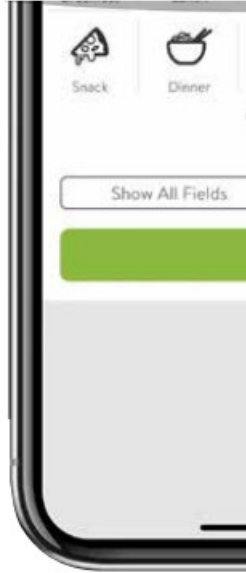
| | |
|--|---|
| <div data-bbox="268 96 304 136">5</div>  <div data-bbox="124 779 526 808">Select the tags you want to save.</div> | <div data-bbox="1262 96 1299 136">6</div>  <div data-bbox="997 779 1181 808">Save the entry.</div> |
|--|---|

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks by Roche is under license.

© 2021 Roche Diabetes Care Limited. All rights reserved.

ACCU-CHEK, ACCU-CHEK INSTANT, and MYSUGR are trademarks of Roche.

All other trademarks or brand names are the property of their respective owners.

Roche Diabetes Care Limited, Charles Avenue, Burgess Hill, West Sussex, RH15 9RY, UK.

Company Registration Number 9055599


For use in the UK & Ireland only

Date of preparation: February 2021

Material Number: 09426507001

www.accu-check.co.uk www.accu-check.ie

Documents / Resources

| | |
|---|---|
|  | <p>Roche ACCU-CHEK Instant Wireless Blood Glucose Monitoring System [pdf] User Guide ACCU-CHEK Instant, Wireless Blood Glucose Monitoring System, ACCU-CHEK Instant Wireless Blood Glucose Monitoring System</p> |
|---|---|

References

- [Accu-Chek UK | Diabetes Care Management: Devices, Apps & Tools](#)
- [Accu-Chek UK | Diabetes Care Management: Devices, Apps & Tools](#)