



Roche cobas h 232 System User Guide

[Home](#) » [Roche](#) » Roche cobas h 232 System User Guide 




Quick Reference Guide

Contents


- [1 IQC Checking](#)
- [2 Routine Testing](#)
- [3 Documents / Resources](#)
- [4 Related Posts](#)

IQC Checking

IQC timeout at 8 AM daily

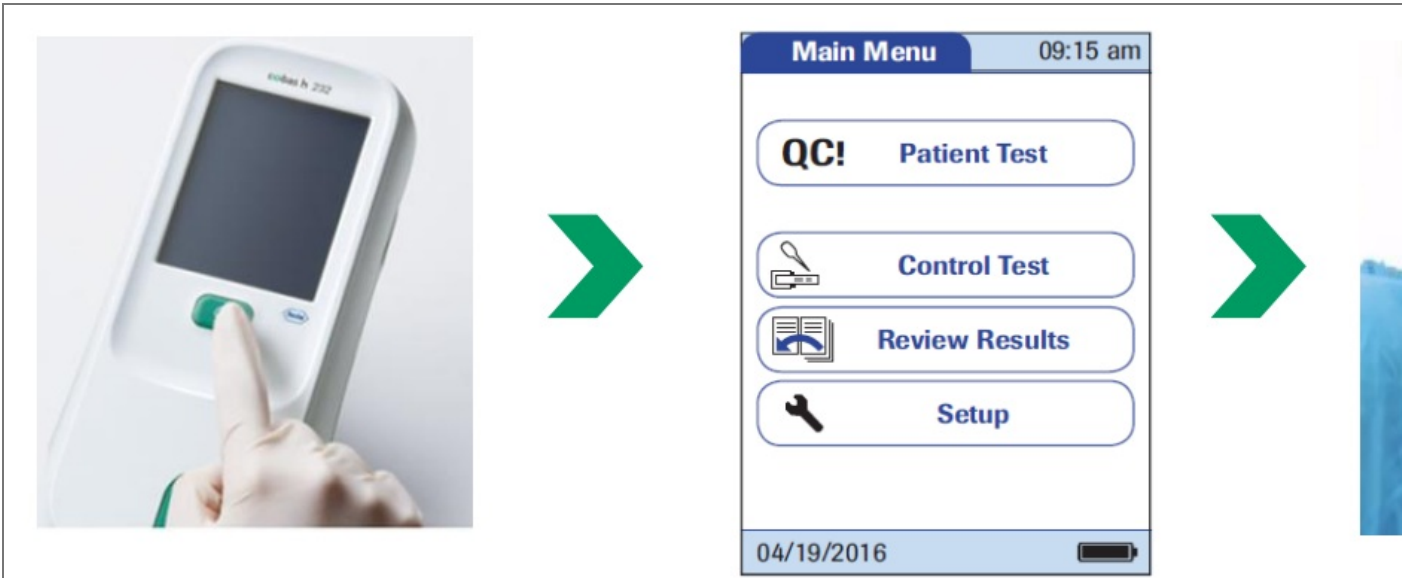

Power on the h 232	Press Control Test	Insert the IQC test strip
--------------------	--------------------	---------------------------

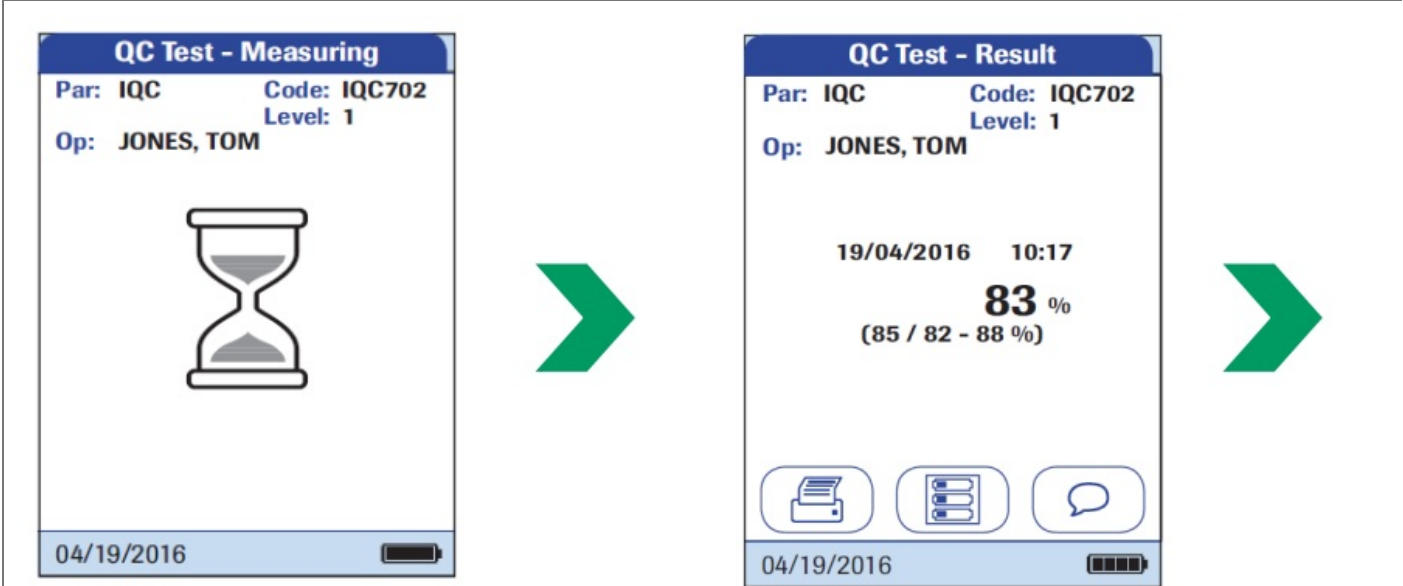


The measurement begins automatically	The result is displayed as Pass or Fail	Return the test strip to the container
--------------------------------------	---	--

■ Daily Cleaning (70% alcohol)



A) Clean the meter surface with moistened tissue B) Remove and wipe the green cover	Clean the inside area with cotton swab	* Be gentle & cautious on the side
---	--	------------------------------------

Routine Testing

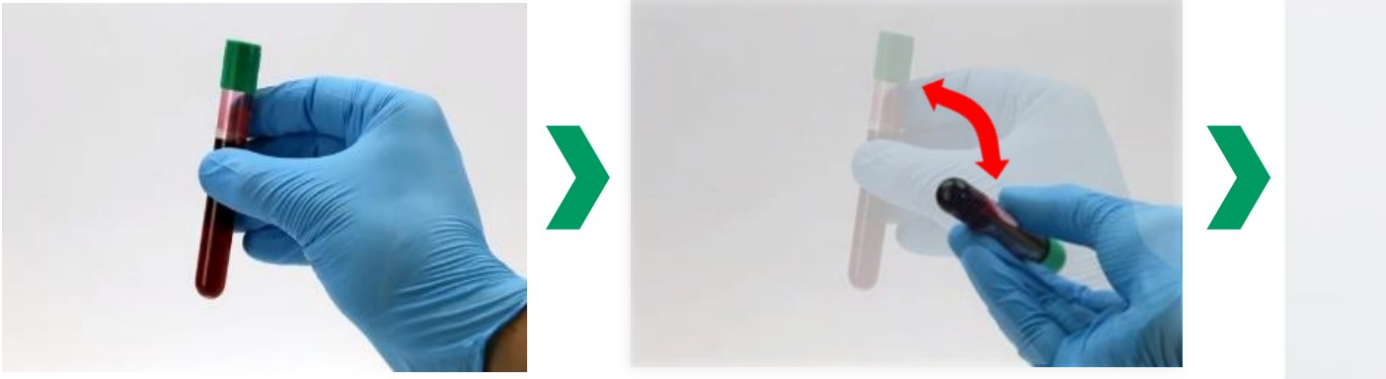	
Use heparinized venous whole blood only	Gently swirl the tube for 10 times
	
Press the "Patient Test" Button	Enter or scan the patient ID
	
The test strip will warm up automatically	Draw exactly 150 µL sample using a Roche CARDIAC Pipette in between the 2 blue arrows



* Hold the pipette vertically over the sample application area. Slowly apply the sample to avoid spillage and losing of samples.

The Roche CARDIAC Pipette tip should not touch the sample application area of the test strip.



<https://vimeo.com/757443233>

Technical Support:

+852 5729 6447 (Max Tsang)


+ 852 5729 3919 (Jeffrey Chan)

Roche Diagnostics (Hong Kong) Limited

Video Guide



Documents / Resources

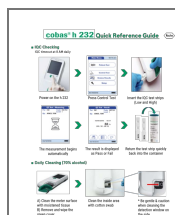

[Roche cobas h 232 System](#) [pdf] User Guide
cobas h 232 System, cobas h 232, 232 System, cobas