



Roche P-N 09804536001 Connect the power supply 4 Optionally Analyzer Instruction Manual

[Home](#) » [Roche](#) » Roche P-N 09804536001 Connect the power supply 4 Optionally Analyzer Instruction Manual 

Contents

- [1 P-N 09804536001 Connect the power supply 4 Optionally Analyzer](#)
- [2 P-N 09804536001 Connect the power supply 4 Optionally Analyzer](#)
- [3 Parts list](#)
- [4 Set up the analyzer](#)
- [5 Log on to the analyzer](#)
- [6 Shut down the analyzer](#)
- [7 User documentation](#)
- [8 Documents / Resources](#)
 - [8.1 References](#)

P-N 09804536001 Connect the power supply 4 Optionally Analyzer



cobas® liat analyzer
Quick Start Guide

P-N 09804536001 Connect the power supply 4 Optionally Analyzer

This Quick Start Guide is a reference for the cobas® liat analyzer setup. Read the entire cobas® liat system User Guide before using it, or running any cobas® liat assay.

Unpacking the analyzer

IMPORTANT! Retain the shipping container and packing materials in case you need to return the analyzer to Roche for any reason.

- Unpack the analyzer from the shipping container.
- Do not apply any force to the door on the top of the unit or to the barcode scanner window.
- Do not hold or lift the analyzer from the sample entry door or the barcode scanner window.
- Handle with care. Use both hands to support the analyzer along the sides of the base.
- Unpack the power supply and other accessories.
- Inspect the outer case, front panel buttons, screen, and rear ports for any shipping damage. Check all other items for shipping damage. Report any problems to your local Roche representative.

Incident reporting

- Inform your Roche representative and your local competent authority about any serious incidents which may occur when using this product.

Parts list

Documentation

- cobas® liat analyzer Cleaning Tool Guide
- cobas® liat system Short Instructions
- cobas® liat analyzer Quick Start Guide

Hardware parts

- Touch screen stylus
- Power cable
- Power supply
- cobas® liat cleaning tool kit
- cobas® liat user ID card kit
- cobas® liat analyzer



Set up the analyzer

No special setup is required, all software and default settings are preloaded prior to shipment.

Analyzer location and operating environment

- Place on a level, vibration-free and non-reflective surface, away from direct sunlight.
- Allow at least 10 cm (4 in) rear clearance for ventilation and at least 6 cm (2.5 in) front clearance for barcode scanning.
- Operating temperature is between 15°C and 32°C (59-90°F), and relative humidity is between 15% and 80%. Operating altitude is up to 2000 metres Mean Sea Level (MSL).
- Locate the analyzer near a grounded mains outlet to avoid the power cable being a trip hazard.



Connect the power supply

CAUTION! Using an inappropriate power supply may cause malfunction.

Only use the power supply provided, connected to a voltage of 100-240V AC, 50-60 Hz.

- Insert the power supply cable into the analyzer power connector located at its rear.
- Connect the power cable to the power supply. Plug the other end of the cable into a properly grounded mains outlet.




Optionally, connect to network

Connecting the analyzer to the network enables, for example, remote software and assay updates, automatic time/date synchronization (SNTP), and host connectivity.

- Connect the Ethernet jack on the rear of the analyzer to an appropriate wall jack using a standard Ethernet data cable with RJ-45 connectors.

Refer to the User Guide for information on defining the network environment.

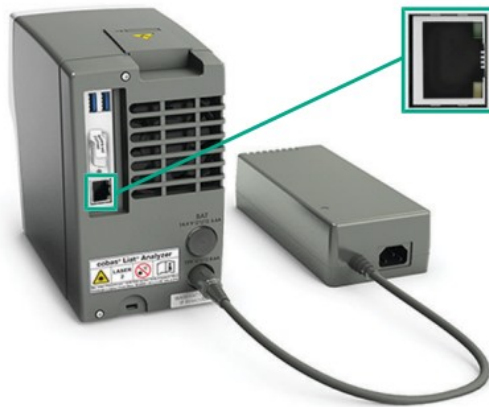

Power on the analyzer

This initiation process takes a few minutes from power up to display of the log on screen.

- Press and release the power button.
- The analyzer starts initializing. Wait for initialization to complete.
NOTICE Do not insert an assay tube or other items into the analyzer during initialization. Doing so will damage the assay tube and the analyzer.
- The Log on screen is displayed when the analyzer is ready for operation.




Log on to the analyzer

- Choose the Log on button.
- If you first log on to the analyzer after unboxing, enter the predefined user ID “ADMIN” and choose the Enter button.
- If you first log on after a restart, enter your user ID and choose the Enter button.
- If you log on to the analyzer on a routine basis, either enter your user ID and choose the Enter button, or choose the Scan button, and scan the barcode on your User ID card.

WARNING! Do not stare into the laser beam. The analyzer is a class 2 laser product.



- Enter your password or when prompted, change your password.
- On the first log on after unpacking the administrator sets first the Time zone and then the Date and time. It is recommended to keep this sequence! (Settings > System > Date and time).

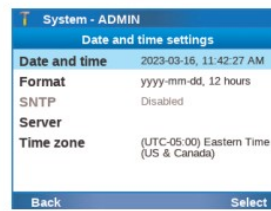

Run an assay


Refer to the cobas® liat system User Guide and the package insert or Instructions for Use of the assay for instructions on running an assay.



Shut down the analyzer

💡 Always use the shutdown function within the software. Improper shutdown can damage the analyzer!

- To log off from the analyzer, choose on the Main screen the Log off button.
- On the Log on screen, choose the Shut down button.
- Follow the instructions on the screen.
- When prompted, power off the analyzer by holding the power button until both the power indicator and the touch screen turn dark.

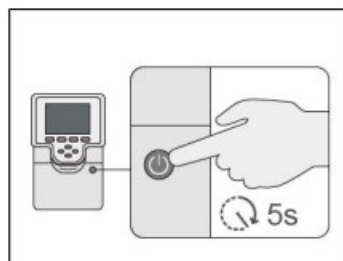

User documentation

Electronic user documentation can be downloaded using the eLabDoc e-service on Roche DiaLog:

<https://dialogportal.roche.com>

Alternatively, scan the following QR code:



https://pim-eservices.roche.com/eLD/web/pi/en/documents?searchTerm=cobas%20liat%20quick%20reference%20guide&noDefaults=true&ffSystems=9274&ffVersions=Documents_LatestVersion&searchType=Metadata&sourceSearchType=Metadata

For more information, contact your local affiliate or Roche Service representative.

Roche Support | Publication information

To obtain technical assistance, contact your local Roche representative.

Affiliates: <https://www.roche.com/worldwide>

Patents: <http://www.roche-diagnostics.us/patents>



Roche Molecular Systems, Inc.
1080 US Highway 202 South
Branchburg, NJ 08876 USA



Made in Switzerland



Roche Diagnostics GmbH Sandhofer Strasse 116 68305 Mannheim Germany

© 2015-2023 Roche Molecular Systems, Inc.

COBAS and LIAT are trademarks of Roche.


All other product names and trademarks are the property of their respective owners.

Revision history





Publication version 12.0 Revision date June 2023

VV-07724-05

Documents / Resources

	<p>Roche P-N 09804536001 Connect the power supply 4 Optionally Analyzer [pdf] Instruction Manual</p> <p>P-N 09804536001 Connect the power supply 4 Optionally Analyzer, P-N 09804536001, Connect the power supply 4 Optionally Analyzer, power supply 4 Optionally Analyzer, supply 4 Optionally Analyzer, Optionally Analyzer, Analyzer</p>
---	--

References

-  [Patents](#)
-  [navify® Portal](#)
-  [navify® Portal](#)
-  [Roche | Worldwide](#)
- [User Manual](#)

[Manuals+](#), [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.