



RR Mechatronics Starrsed X-Clean Westergren Pipettes Instructions

[Home](#) » [RR Mechatronics](#) » RR Mechatronics Starrsed X-Clean Westergren Pipettes Instructions 



REF QRR010946
Starrsed X-Clean
PACKAGE INSERT

Contents

- [1 Intended purpose](#)
- [2 Instructions For Use](#)
- [3 Chemical/Physical data](#)
- [4 Transport/Storage/Packaging](#)
- [5 Safety](#)
- [6 Ecological information](#)
- [7 Documents / Resources](#)
- [8 Related Posts](#)

Intended purpose

Product for the automatic cleaning of the Westergren pipettes in Starrsed ST.

Instructions For Use

The product is for use on the indicated Starrsed analyzers only. The product is ready to use, refer to the instrument manual for detailed instructions.

1. Open the box by tearing off the perforated part on top of the box.

2. Remove the container screw cap and pull the neck of the bottle pack out of the cardboard box.
3. Refill the onboard container on the instrument according the instructions in the instrument manual.

Chemical/Physical data

Contains main surfactants (0.5 % by weight in total) as active ingredient and some excipients to boost the cleaning performance (<0.25 % by weight in total) and the preservative sodium azide (< 0.5 ‰ by weight).
The latest version of the Safety Data Sheet is available on the website www.rrmechatronics.com

Transport/Storage/Packaging

Stability during transport: 5 – 60 °C, with a maximum of 14 days.
Store in dry conditions in well sealed containers at temperatures of 5 till 35°C, preferably at 18 till 30°C.
Starrsed X-Clean is supplied in 5 Liter containers, LxWxH 19x19x19 cm.

Safety

Any serious incident that has occurred in relation to this product must be reported to the distributor and/or manufacturer and the competent local authorities.

Shelf life

After production: 2 years, but no longer than the expiry date.
In-use stability: Maximum of 3 months at advised temperature of 5 till 25°C.

Ecological information

Generally not hazardous for water.

Disposal

Recommendation Must not be disposed of together with household garbage. Do not allow product to reach sewage system. Disposal must be made according to official regulations.
Recommended cleaning agent Water, if necessary with cleaning agent.



RR Mechatronics Manufacturing B.V.
De Corantijn 13, 1689 AN Zwaag, The Netherlands
www.rrmechatronics.com

Rev. 5

Revision due to IVDR-registration, intended purpose modified
Website added under manufacturer address
Component description and safety chapter added

Package Insert / Version: 05-EN 26-04-2022

Supplier
Mechatronics Instruments B.V.
P.O. Box 225
1620 AE Hoorn
The Netherlands
Phone: +31 229 291 129
Fax: +31 229 241 534

Documents / Resources



[RR Mechatronics Starrsed X-Clean Westergren Pipettes](#) [pdf] Instructions
Starrsed X-Clean Westergren Pipettes, Westergren Pipettes, Starrsed X-Clean, Pipettes, X-Clean