
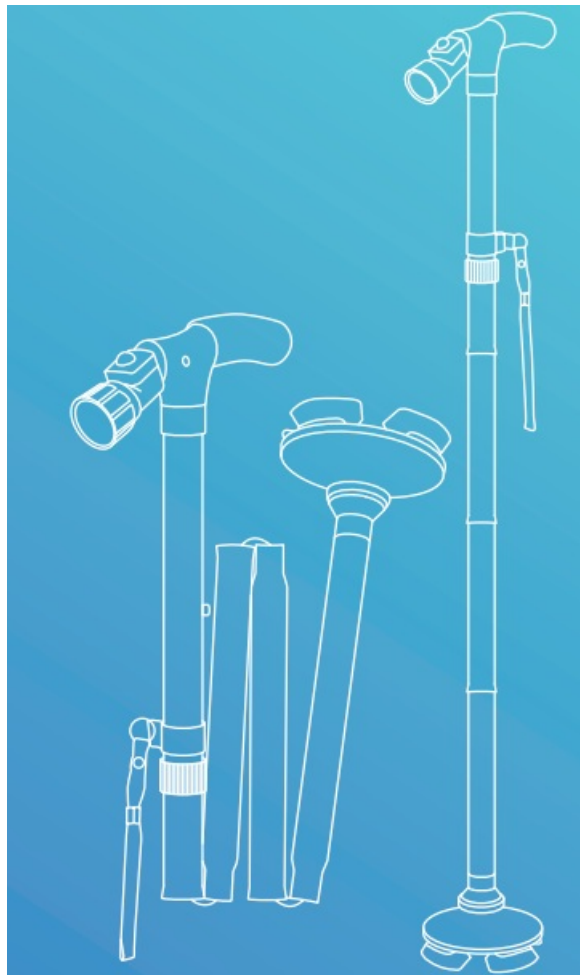




SAFE AGE UM A1 C LED Folding Cane User Manual

[Home](#) » [SAFE AGE](#) » **SAFE AGE UM A1 C LED Folding Cane User Manual** 

SAFE AGE UM A1 C LED Folding Cane




Contents

- [1 A QUICK START](#)
- [2 B CANE HEIGHT ADJUSTMENT](#)
- [3 C FIND YOUR CORRECT CANE HEIGHT](#)
- [4 D OPERATE THE LED LIGHT](#)
- [5 E CHANGING BATTERIES](#)
- [6 Documents / Resources](#)
 - [6.1 References](#)
- [7 Related Posts](#)

A QUICK START


1. Unbutton the wrist strap that is holding the cane together.
2. Hold the handle and release the folded sections. The cane will automatically snap into place.
3. Untwist the LED light cap and remove the battery protection tab to use the LED Light (see changing the batteries section for help).

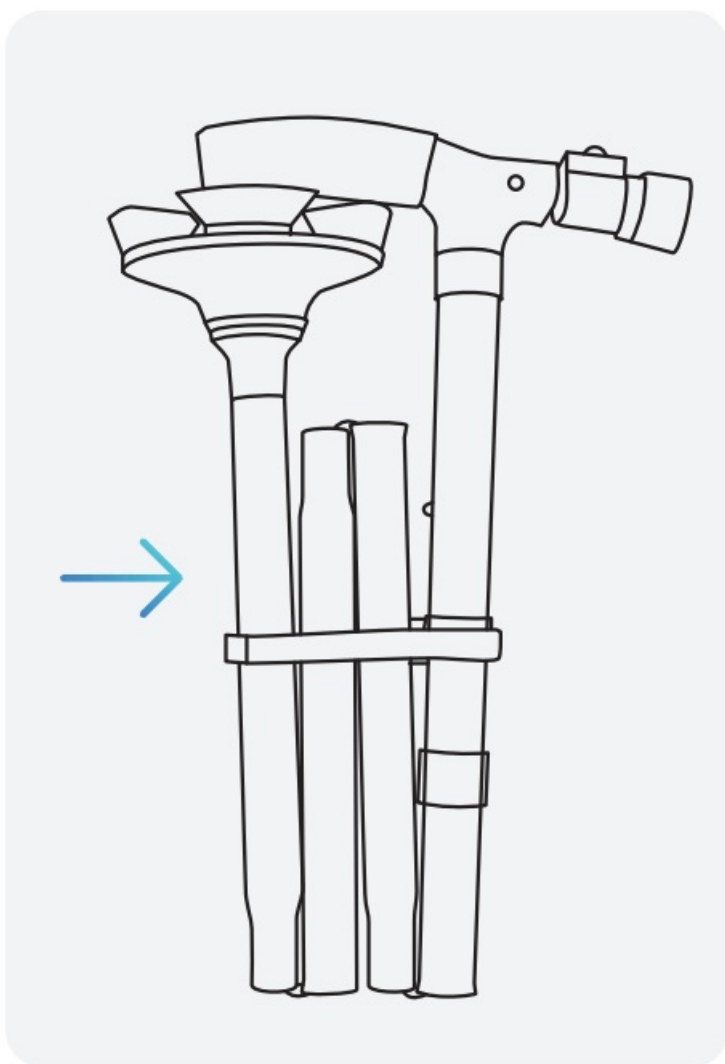

B CANE HEIGHT ADJUSTMENT

1. Loosen the locking latch.
2. Adjust brass pin to desired position.

3. Tighten the locking latch.

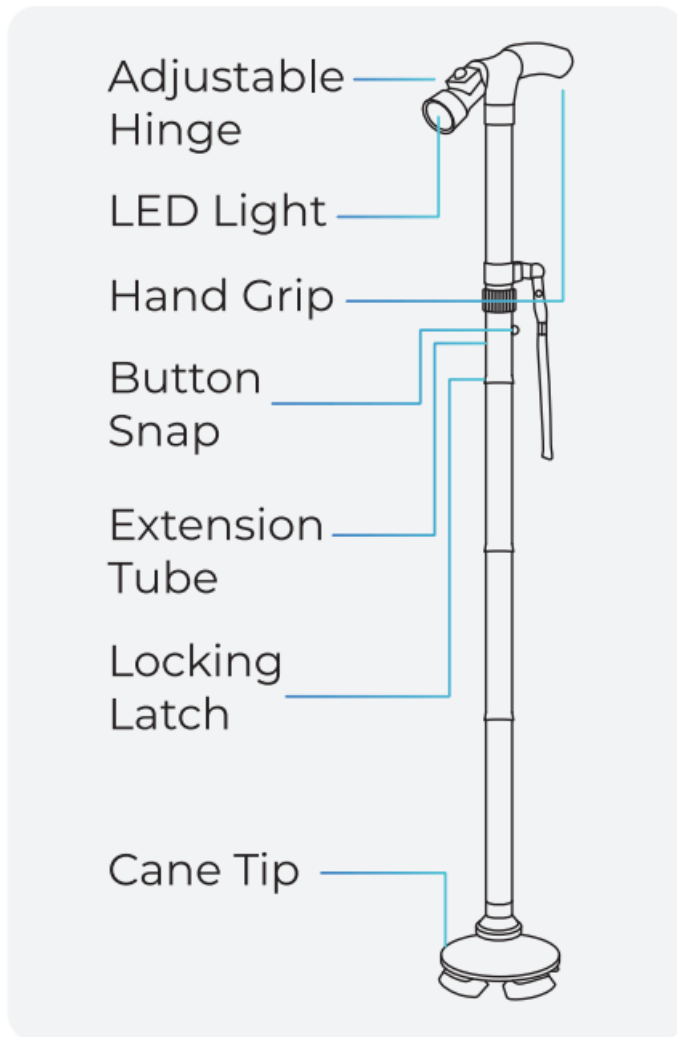

C FIND YOUR CORRECT CANE HEIGHT


Let your arm hang naturally by your waist. Adjust the cane height so the handle touches your wrist.

D OPERATE THE LED LIGHT

Remove the paper before use the LED Light

E CHANGING BATTERIES

1. Twist cap counterclockwise and remove the old batteries.
2. Insert 4 new LR44 Batteries negative side first into the black casing. Place black casing back into the battery holding slot negative side first.
3. Twist cap clockwise to lock into place.

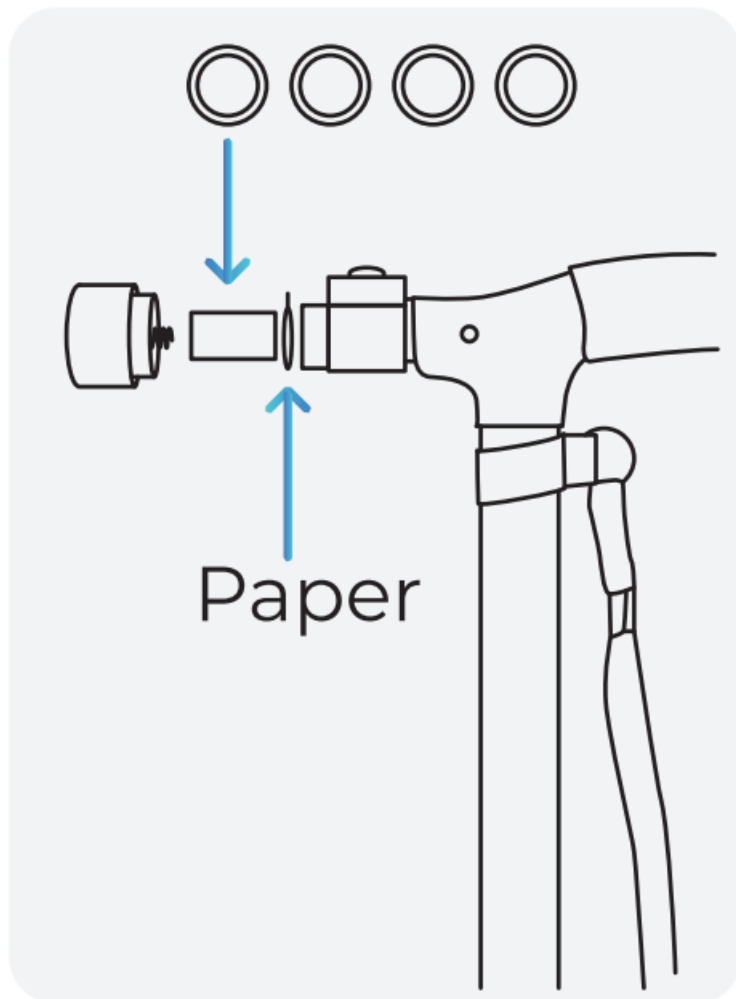

Note: Twist the cap all the way on to ensure the batteries are engaged.


WARNING



- Make sure the brass pin is locked into place before use.
- **Weight Capacity:** 114 KG



Documents / Resources

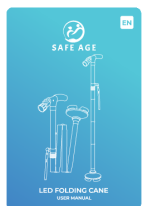

[SAFE AGE UM A1 C LED Folding Cane](#) [pdf] User Manual
UM A1 C LED Folding Cane, UM A1 C, LED Folding Cane, Folding Cane, Folding Cane

References

- [User Manual](#)