



# SAS C61W G4 Preventative Maintenance Checklist Instruction Manual



[Home](#) » [Sas](#) » SAS C61W G4 Preventative Maintenance Checklist Instruction Manual 



## Contents

- [1 Preventative Maintenance Checklist](#)
- [2 Documents / Resources](#)
  - [2.1 References](#)
- [3 Related Posts](#)

## Preventative Maintenance Checklist

	Completed by	Date
<p>This document outlines the cleaning and preventive maintenance procedures needed to be maintained on your Hydrim C61W G4.</p> <p>To see detailed instructions on each step please scan the QR code below titled “<b>Hydrim C61W G4 Maintenance</b>”</p>		
<p><b>Replacing the HIP Cleaning Solution:</b></p> <p>The HYDRIM C61W G4 uses a pre-measured container of HIP Cleaning Solution. Use only the recommended cleaning solution and read the MSDS before inserting into the unit.</p>		
<p><b>Refilling the Water Softening Salt Reservoir:</b></p> <p>When a red X appears on the detergent and water softener level indicator, press on the icon to determine if the problem lies with the water softener or cleaning solution level. If the water softener must be refilled follow the steps in the instructions.</p>		
<p><b>Changing the Air Filter:</b></p> <p>When the message “Replace air filter” appears, the HEPA filter is in need of changing. Follow the steps in the instructions.</p>		
<p><b>Wipe Down Hydrim:</b></p> <p>To clean outer surfaces and the decal covering the touchscreen, use a soft cloth moistened with soap and water or a surface disinfectant like OPTIM1 Disinfectant Cleaner. Do not use harsh cleaning chemicals.</p>		
<p><b>Daily Maintenance:</b></p> <ul style="list-style-type: none"> <li>Wipe Down Outer Surfaces</li> <li>Filter Maintenance</li> <li>Wash Arm Maintenance</li> <li>Door Lock Check</li> <li>Wash Arm Rotation Check</li> <li>Door Seal Check</li> <li>Load Carrier Check</li> <li>Chamber Filter Check</li> </ul> <div style="text-align: center;">   </div> <p><a href="https://qrco.de/bdz7y9">https://qrco.de/bdz7y9</a></p>		
<p><b>Annually – By a SciCan Approved technician –</b></p> <p>Annual maintenance schedule as outline in the service manual</p>		



**HIP Solution – C61W G4**  
<https://qrco.de/bdz828>



**Hydrim Water Softening Salt**  
<https://qrco.de/bdz8DI>



**Optim1 6" x7" Wipes**  
<https://qrco.de/bczLEe>



**Optim1 1 Gallon Bottle**  
<https://qrco.de/bde3re>



**Hydrim C61W Air Filter 01-1132775**  
<https://qrco.de/bdz9PU>




**Hydrim C61W Arm 01-1133025**  
<https://qrco.de/bdx0it>



**Hydrim C61W Chamber Scal 01-10//865**  
<https://qrco.de/bdx0fZ>



**Hydrim C61W Wash Drain Filter 01-1033105**  
<https://qrco.de/bdz9VS>

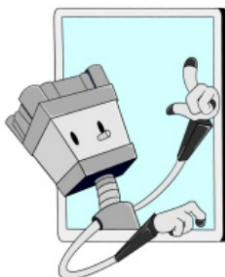


**FREE TECH SUPPORT**



**Sterilizer Autoclave Solutions**  
INFECTION CONTROL SPECIALISTS

**HYDRIM C61W G4**

Documents / Resources

	<p><a href="#">SAS C61W G4 Preventative Maintenance Checklist</a> [pdf] Instruction Manual C61W G4 Preventative Maintenance Checklist, C61W G4, Preventative Maintenance Checklist, Maintenance Checklist, Checklist</p>
---	--

References

-  [Meeting Place for Memorable Moments by MUSA](#)
- [User Manual](#)

[Manuals+](#), [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.