




Portable Digital Video Magnifier User Manual

[Home](#) » [Sharper Image](#) » Portable Digital Video Magnifier User Manual 

Portable Digital Magnifier

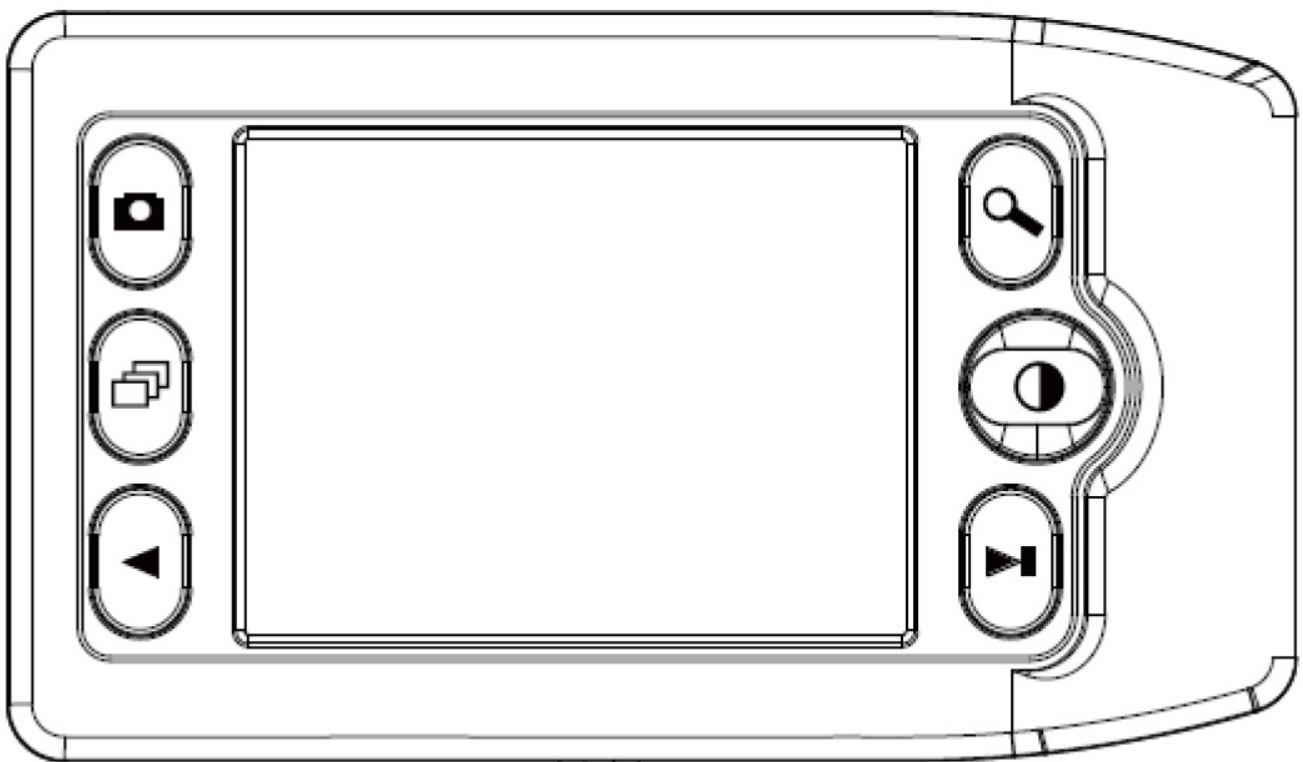

Welcome

Thank you for purchasing a Portable Digital Magnifier.

Please take a moment to read through this manual. Its contents will help you to get the best use of the device.


Precaution

Federal Communications Commission Notice

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

Overview

Open Box and Unpack Items

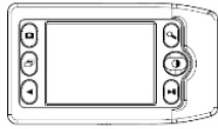

Portable Digital Magnifier




Clean fabric



Carrying bag



USB cable

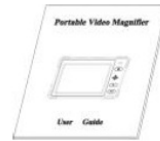


User's manual



AAA Alkaline Battery

Part Names & Functions

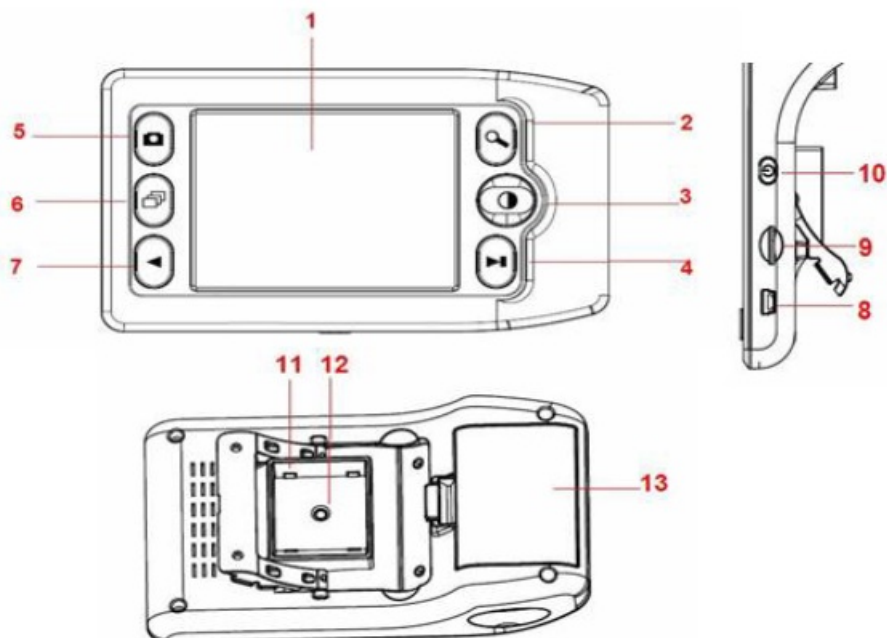

- | | |
|-----------------------------|------------------------|
| 1. 3.5-inch display | 8 USB port |
| 2. Magnify | 9 Micro-SD card slot |
| 3. Color Mode | 10 Power on/off |
| 4. Freeze/Unfreeze/Backward | 11 LED |
| 5. Snapshot | 12 Lens |
| 6. Playback | 13 Battery Compartment |
| 7. Forward | |

Safety and Maintenance

To keep your Magnifier in good condition, observe the following guidelines:


1. Read the instructions in this user's manual before attempting to use the unit.
2. Do not attempt to repair the Magnifier or open its case. This product contains no user serviceable parts.
3. Keep the Magnifier away from wet areas, rain, and other sources of liquids. Do not submerge the unit.
4. Always operate the Magnifier in a place where the temperature is between 32°F and 113°F.
5. Always store the Magnifier in a cold dry place at normal room temperature.
6. Do not attempt to force any connector into the unit's USB and Micro SD port. All appropriate connectors should fit easily into the port.


Getting Started

Loading Battery

Please use 3 A AA alkaline batteries. Slightly push and pull the battery door latch to open the battery compartment. Load the batteries according to the polarities shown inside the battery compartment, then snap close the battery door. Do not mix the old and new batteries during battery replacement.

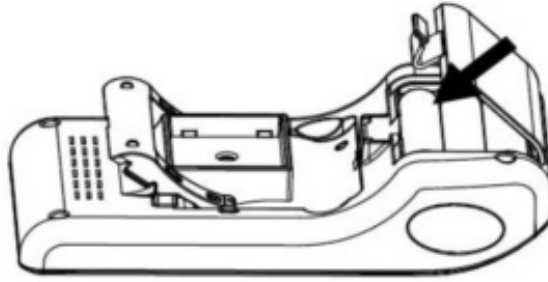


Notes: Remove the batteries when it is not in use for a long period of time. Rechargeable AAA size alkaline or NI-MH battery is acceptable. Do not mix different type of battery in the battery compartment.



When the battery is low, there will be low power indicator  shown on the lower right corner of the panel. When replacement of new fresh batteries is needed, use only Alkaline AAA size batteries. Do not mix the old with the new batteries during battery installation.


Reading with the Magnifier

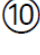To read with the Magnifier, operate as the following:

1. Turn on the unit by pressing the Power on/off switch  located on side.
2. Pull out the foot of the Magnifier until it sits at its maximum range.
3. Place the Magnifier flat against the text you want to read.
4. Slide the unit gradually to the right to read to the end of the line. Slide the unit down and back to the left to read the next line of text.
5. To change the level of magnification, press the Magnify button .
6. To change the colors used on the screen, press the Color Mode button  until you find the mode best suited for your vision needs.


Using the Magnifier

Color Mode selection


Press Color Mode button  to cycle through the Magnifier's 3 different color modes as follows:

- ◆ Full color
- ◆ Enhanced positive image (black on white)
- ◆ Enhanced negative image (white on black)


Magnification Adjustment

Press Magnify button  to cycle through the four level of magnifications 5X, 6X, 7.5X or 10X.




Freeze Image


Press Freeze/Unfreeze  button to switch between freezing and unfreezing the current image.

Capture Photo

Load a micro SD card (not included) into the micro SD card slot. Press Snapshot  button to capture a photo. The captured photos will be automatically stored in the micro SD card.

Playback


Press Playback  button to view captured photos. Press Backward  and Forward  button to move forward and backward.


Press Playback  button again will return the magnifier to reading mode.

Download Captured Pictures to PC


Connect the Magnifier with PC by USB Cable to USB port, turn the power on and then you can download the captured pictures to computer.

Low Battery Indication

If battery is low, there will be a  symbol (blank) on the lower right corner of the screen, indicating that you have to replace new batteries.

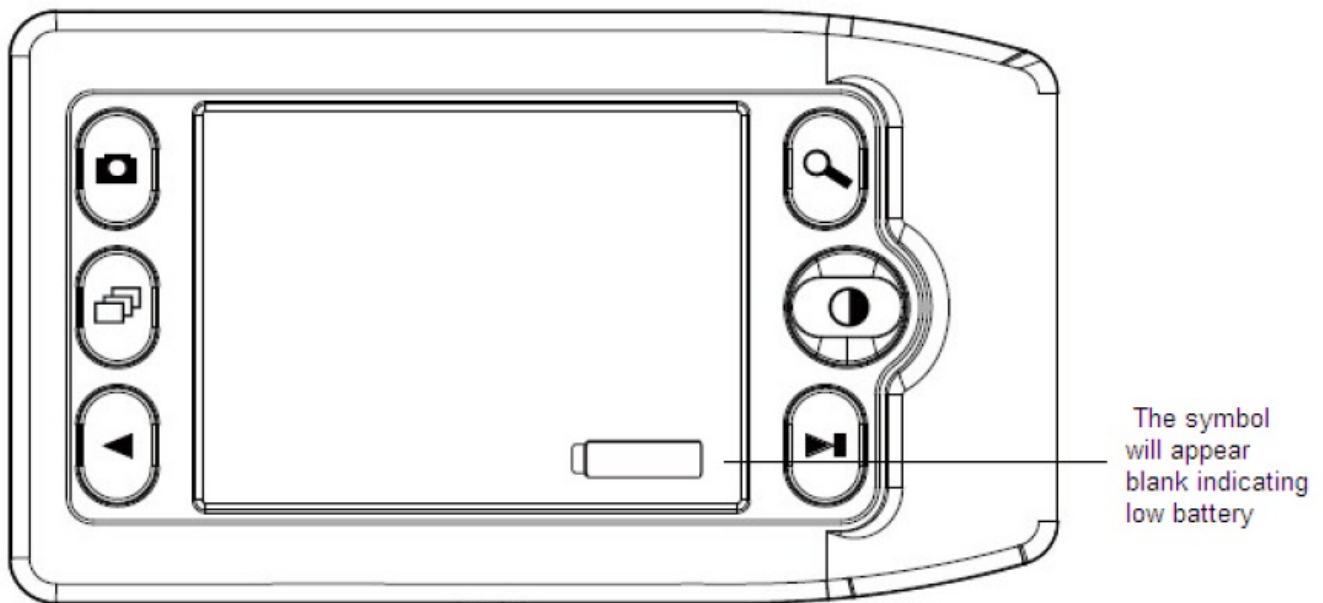

Power Off

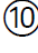Turn off the power of the unit by pressing the Power on/off switch  located at the side. To save battery power, please always turn off the Magnifier when not using it. The Magnifier will turn off automatically if it is not in use after 5 minutes.

Trouble Shooting

Problem	Solution
The screen is blank.	Ensure that the power is on. Ensure the battery has enough electricity. Verify that the item you want to magnify is Positioned correctly under the center of the screen
The unit does not turn on.	Change the battery with new ones. Check the polarity of battery installed
The image on the screen is dark and/or distorted.	Ensure that the unit is positioned flat on the page you are reading. Ensure that the foot of the magnifier flips out to its maximum position.
There are smudges or blemishes on the screen	Clean the screen and the Lens (located on the bottom of the unit).
Text is too small to read	Press the magnify button.

This symbol on the product or in the instructions means that your electrical and electronic equipment should be disposed at the end of its life separately from your household waste. There are separate collection systems for recycling in the EU.

For more information, please contact the local authority or your retailer where you purchased the product.



Portable Digital Video Magnifier User Manual – [Original PDF](#)

Portable Digital Video Magnifier User Manual – [Optimized PDF](#)