



Shofu EyeSpecial C-V Cross Polarizer User Manual

[Home](#) » [SHOFU](#) » Shofu EyeSpecial C-V Cross Polarizer User Manual 



- Read through the "Important Safety Instructions" for the Especial C-V before using this cross polarizer.
- This product is exclusively for the Especial C-IV and later models. It is incompatible with the previous models Eye Special C-II and C-III.

Contents

- [1 Intended use](#)
- [2 Preparation](#)
- [3 How to take a picture](#)
- [4 Documents / Resources](#)
 - [4.1 References](#)

Intended use

To make the surface texture or inner structure of natural teeth stand out in clear relief, such as MA melon structure or cracks, which can hardly be captured by the regular picture taking.

Preparation

Activate the cross polarization mode before you use the cross polarizer for the first time (activation is required only once).

1. Activate the cross polarization mode

Scan the QR code below to activate the cross polarization mode.

When the message “Change the optional setting?” appears, press “OK”.



QR code to activate the cross polarization mode

2. Save cross polarization mode as custom mode

Scan the QR code below to save the cross polarization mode as a custom mode.

When the message “Select the number” appears, select one of the three custom settings.

Thereby make sure the camera is in a shooting mode (one of the 9 preinstalled modes), since QR codes cannot be scanned in the custom mode.



for EyeSpecial C-IV



for EyeSpecial C-V

QR codes to save
as a custom mode

How to take a picture

1. Select the custom setting with cross polarization

Confirm, that the “Custom” indicator is displayed on the upper left of the screen before taking a picture.




2. Mount the close-up lens

Mount the close-up lens onto the camera lens and rotate it clockwise until it is securely attached.

3. Set the SHOFU Cross Polarizer

Set the SHOFU Cross Polarizer securely onto the close-up lens until you hear a click sound.

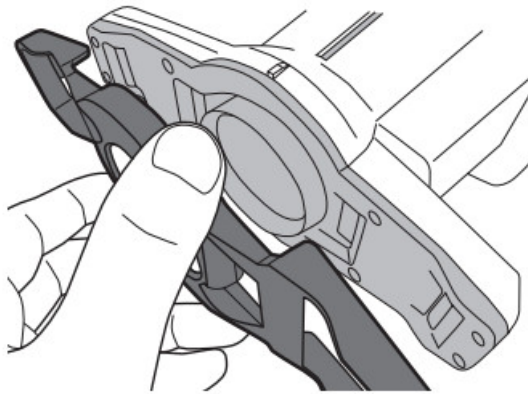


4. Take picture


Procedures to take pictures are the same as in Tele macro mode. Please refer to the User Manual of the Eye Special C-V.

5. Remove the SHOFU Cross Polarizer


To remove the polarizer, gently lift the catch on either side of the camera and press the polarizer outward.

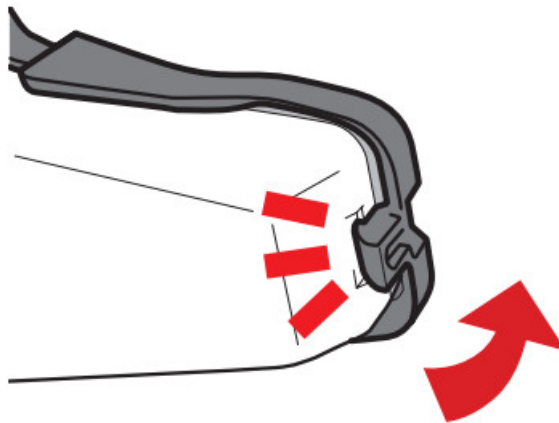

How to scan a QR code

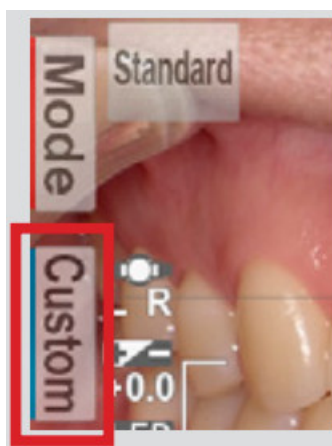

1. Press **Custom** or the F2 key in the shooting mode

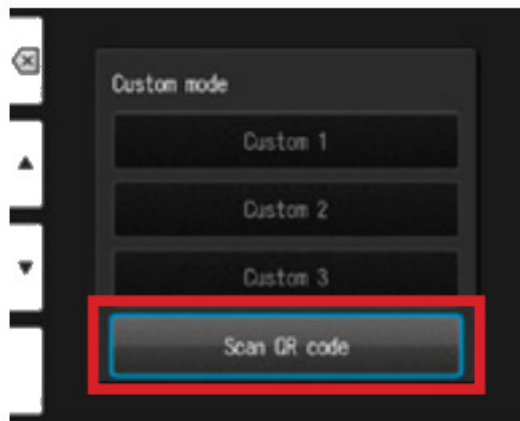

2. Select "Scan QR code"



3. Press the shutter button to start scanning



Manufacturer
SHOFU INC.


11 Kamitakamatsu-cho,
Fukien Higashiyama-ku,
Kyoto 605-0983, Japan

SHOFU DENTAL GmbH

An der Pant 70, 40885 Ratingen, Germany

www.shofu.com

08/2023

	<p>Shofu EyeSpecial C-V Cross Polarizer [pdf] User Manual</p> <p>EyeSpecial C-V Cross Polarizer, EyeSpecial C-V, Cross Polarizer, Polarizer</p>
---	---

References

- [User Manual](#)