

SIEMENS dNote Driver Pack User Guide


[Home](#) » [SIEMENS](#) » SIEMENS dNote Driver Pack User Guide 

Contents

- [1 SIEMENS dNote Driver Pack](#)
- [2 Product Usage Instructions](#)
- [3 INSTRUCTION](#)
- [4 Device Driver Support Full List](#)
- [5 Documents / Resources](#)
- [6 Related Posts](#)

SIEMENS

SIEMENS dNote Driver Pack

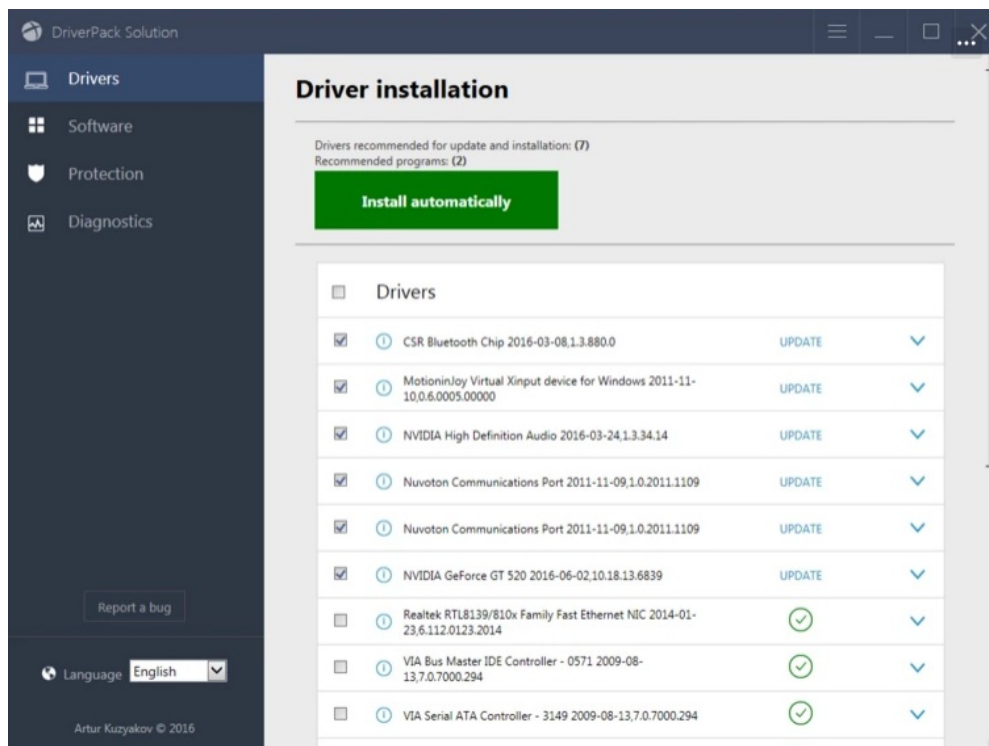

Product Information

The product is the dNote Driver Pack, version 3.5. It is a software package that provides device driver support for various medical devices. The driver pack is compatible with UniPOC version 1.6.4 and higher or UniPOC version 1.7.4 and higher. It also requires the latest release/patch of dNote version 3.2 or higher.

The dNote Driver Pack supports both unidirectional and bidirectional device drivers. The unidirectional device driver support includes Meditrac Patient Results, QC Results, Calibration Results, Proficiency Results (introduced with Driver Pack 2.5 device type specific), and Linearity Results (introduced with Driver Pack 2.5 device type specific).

The bidirectional device driver support includes Meditrac Patient Lists, Operator Lists, Location Lists, Reagent Lists, QC Lot Lists, Configurations per Device Type, and Comment Lists (introduced with Driver Pack 2.5).

Please refer to the UniPOC Driver Functionality Matrix spreadsheet in the Product Information Portal for specific functionality supported by UniPOC for each device type.

Product Usage Instructions

To install the dNote Driver Pack 3.5, make sure you have UniPOC version 1.6.4 or higher or UniPOC version 1.7.4 or higher installed on your system. Additionally, ensure that you have the latest release/patch of dNote version 3.2 or higher.

After confirming the compatibility requirements, download the appropriate driver pack installation file for your device type from the provided list:

- Piccolo – Download DLP_Piccolo.exe version 6.5.5.44013
- AfinionAS100 – Download DLP_AfinionAs100.exe version 6.4.92.41492
- Afinion2 – Download DLP_AfinionAs100.exe version 6.4.92.41492
- Alerel – Download DLP_Alerel.exe version 6.4.57.41037
- Emerald – Download DLP_CellDynEmerald.exe version 6.4.103.41037

- and so on...

Once you have downloaded the appropriate driver pack installation file, run the executable file to start the installation process. Follow the on-screen instructions to complete the installation.

Note: Installing the dNote Driver Pack 3.5 against incompatible versions of UniPOC or dNote will result in the installer package programmatically cancelling the installation.

INSTRUCTION

This document includes release note information for Version 3.5 of the dNote Driver Pack.

IMPORTANT NOTE: Driver Pack 3.5 can only be installed against UniPOC 1.6.4 and higher or UniPOC 1.7.4 and higher, and the latest release/patch of dNote 3.2 or higher. Attempting to install against any other versions will result in the installer package programmatically cancelling the install attempt.

Unidirectional Device Driver Support via Meditrac

- Patient Results
- QC Results
- Calibration Results
- Proficiency Results (introduced with Driver Pack 2.5 device type specific)
- Linearity Results (introduced with Driver Pack 2.5 device type specific)

Bidirectional Device Driver Support via Meditrac

- Patient Lists
- Operator Lists
- Location Lists
- Reagent Lists
- QC Lot Lists
- Configurations per Device Type
- Comment Lists (introduced with Driver Pack 2.5)

Please consult the UniPOC Driver Functionality Matrix spreadsheet in the Product Information Portal for specific functionality supported by UniPOC for each device type.

Device Driver Support Full List

Device Type	Driver Pack Install Name	Meditrac DLP Name	DLP Version1
Abaxis Piccolo Xpress	Piccolo	DLP_Piccolo.exe	6.5.5.44013

Abbott POC Afinion AS 100	AfinionAS100	DLP_AfinionAs100.exe	6.4.92.414 92
Abbott POC Afinion 2	Afinion2	DLP_AfinionAs100.exe	6.4.92.414 92
Abbott POC Alere I	Alerel	DLP_Alerel.exe	6.4.57.410 37
Abbott POC Cell Dyn Emerald	Emerald	DLP_CellDynEmerald.exe	6.4.103.41 037
Abbott ID Now	IDNow	DLP_IdNow.exe	6.5.22.433 12
Analyticon Urilyzer 100 / 100 Pro	Urilyzer100	DLP_Urilyzer100.exe	6.4.41.349 51
Analyticon Urilyzer 100 Pro POCT1a	Urilyzer100ProPoc1a	DLP_Urilyzer100ProPOCT1a.e xe	6.4.23.398 09
BD Veritor	BDVeritor	DLP_BDCloud.exe	6.4.34.410 37
Boule Medical Medonic M-series – M 16, M20, M16M, M20M, M16C, M20C, M16S, M20S, M32B, M32M, M32C, M32S	Medonic	DLP_Swelab_Alpha.exe	6.4.100.41 037
Boule Medical Swelab Alpha – Basic, Plus Basic, Standard, Plus Standard, Cap, Plus C ap, Sampler, Plus Sampler	Medonic	DLP_Swelab_Alpha.exe	6.4.100.41 037
Cepheid GeneXpert Omni	GeneXpertOmni	DLP_GenExpert.exe	6.5.43.438 90
Cepheid GeneXpert PCR	GeneXpert	DLP_GenExpert.exe	6.5.43.438 90
Cepheid GeneXpert Xpress	GeneXpert_Poc1a	DLP_GeneXpert_Poc1a.exe	6.5.29.430 99
Chirus Dxpess Reader	DXpress	DLP_DXpress.exe	6.4.29.167 46
CoaguSense Coag-Sense	CoagSense	DLP_CoagSensePT2.exe	6.5.12.440 12
Druker Diagnostics QBC Star	QBCStar	DLP_QBCStar.exe	6.4.56.410 37
Haemonetics TEG 6s	TEG6s	DLP_TEG6sHI7.exe	6.5.19.433 12
HemoCue 201DM Glucose	HemoCue	DLP_HemoCue.exe	6.4.0.2
HemoCue 201 DM Glucose RT	HemoCue	DLP_HemoCue.exe	6.4.0.2

HemoCue 201 DM Hemoglobin	HemoCue	DLP_HemoCue.exe	6.4.0.2
LumiraDX	LumiraDX	DLP_LumiraDx.exe	6.5.15.44205
Medtronic ACT Plus	ActPlus	DLP_ActPlus.exe	6.4.100.41037
Medtronic HMS Plus	Hms_Plus	DLP_Hms_Plus.exe	6.4.93.40298
Nova Biomedical NovaNet (Test Results & Operator Lists only)	NovaNet-Operator (Operators) NovaNet-Result (Test Results)	DLP_NovaNetOperatorResultU niPOC.exe	6.5.46.44689
Nova Biomedical StatSensor Creatinine		via NovaNet only	
Nova Biomedical StatStrip Glucose Meter		via NovaNet only	
Nova Biomedical StatStrip Lactate		via NovaNet only	
Nova Biomedical StatStrip Ketone		via NovaNet only	
Nova Biomedical Prime Plus		via NovaNet only	
Philips BiliChek	BiliChek	DLP_BiliChek.exe	6.4.7.16746
Quidel Sofia, Sofia 2	Sofia	DLP_Sofia.exe	6.5.25.43762
Quidel Triage Meter Pro	TriageMeterProTmsi	DLP_TriageMeterProTmsi.exe	6.4.109.41435
Radiometer ABL80 Flex	ABL_8xx_H	DLP_ABL_H_FW_2_1.exe	6.5.13.42574
Radiometer ABL80 Flex CO_OX	ABL_8xx_H	DLP_ABL_H_FW_2_1.exe	6.5.13.42574
Radiometer ABL80 Flex CO-OX OSM	ABL_8xx_H	DLP_ABL_H_FW_2_1.exe	6.5.13.42574
Radiometer ABL90 FLEX	ABL_90	DLP_ABL_H_FW_2_1.exe	6.5.13.42574
Radiometer ABL90 FLEX Plus	ABL_90	DLP_ABL_H_FW_2_1.exe	6.5.13.42574
Roche Diagnostics AccuChek Inform II	AccuChekII	DLP_AccuChekII.exe	6.4.104.41037
Roche Diagnostics CoaguChek XS Plus	CoaguChekXsPlus	DLP_CoaguChekXsSeries.exe	6.4.80.39284
Roche Diagnostics CoaguChek XS Pro	CoaguChekXsPlus	DLP_CoaguChekXsSeries.exe	6.4.80.39284

Roche Diagnostics CoaguChek Pro II	CoaguChekXsPlus	DLP_CoaguChekXsSeries.exe	6.4.80.39284
Roche Diagnostics Cobas h232 (old & new version)	CobasH232	DLP_CoaguChekXsSeries.exe	6.4.80.39284
Roche Diagnostics Cobas Liat	CobasLiat	DLP_CobasLiat.exe	6.5.27.43462
Roche Diagnostics Urisys 1100 (FW version 5.x+)	Urisys_100_FW5.x	DLP_Urisys_1100_FW5.x.exe	6.4.98.41037
Siemens CLINITEK Advantus	Clinitek_Advantus	DLP_Clinitek_Advantus.exe	6.4.101.41037
Siemens CLINITEK Status Plus	ClinitekPlus	DLP_ClinitekStatusPlus.exe	6.5.22.42770
Siemens DCA Vantage	DCAvantage	DLP_DCAvantage.exe	6.5.22.42770
Siemens epoc Reader and Host	Epoc	DLP_Epoc.exe	6.5.40.44136
Siemens RAPIDPoint 500 and 500e (includes v5.1)	RapidPoint500/RapidPoint500e	DLP_RapidLab.exe	6.5.57.44251
Sight Olo	SightOLO	DLP_OLO.exe	6.5.11.42744
Sysmex POCH-100i	XP100_H	DLP_XP300_H.exe	6.4.33.16746

TEM International Rotem Delta Gamma	RotemDelta	DLP_Rotem_Delta.exe	6.4.106.41037
Veda.Lab Easy Reader Plus	EasyReader	DLP_EasyReaderPlusAstm.exe	6.4.56.41037
Werfen (IL) Avoximeter 1000E	Avox1000e	DLP_Avox1000e.exe	6.4.40.41454
Werfen (IL) Avoximeter 4000	Avox4000	DLP_Avox4000.exe	6.4.86.39799
Werfen (IL) GEM 4000 HL7 Direct to Device	Gem4000	DLP_Gem4000_HL7.exe	6.4.109.41037
Werfen (IL) GEM 5000	Gem5000	DLP_Gem4000_HL7.exe	6.4.109.41037
Werfen (IL) Hemochron Signature Elite	HemochronElite	DLP_SignatureElite.exe	6.5.29.43150

DLP Version: DLP version installed by this release of the dNote Driver Pack.

Documents / Resources

	SIEMENS dNote Driver Pack [pdf] User Guide dNote Driver Pack, dNote, Driver Pack, Pack
---	---