



SKILLQUBE qube3 Patient Monitoring Simulation System User Guide

[Home](#) » [SKILLQUBE](#) » SKILLQUBE qube3 Patient Monitoring Simulation System User Guide 



Contents

- [1 Start-Up Guide for your qubeSERIE software](#)
- [2 Documents / Resources](#)
- [3 Related Posts](#)

Start-Up Guide for your qubeSERIE software

Before you can start your training with your new SKILLQUBE devices you need to take these following steps to finish setting up your new hardware

Login data for the qubeCLOUD:

As a qubeSERIES customer you'll receive an e-mail to start you off with your qubeCLOUD login data. Please choose a password for your qubeCLOUD account before you start to set up your devices.




<https://skillqube.com/wp-content/uploads/2020/11/Apple-ID-EN-V2.0-.mp4>

Software set up on your hardware


1. open settings and log in with your Apple-ID or create a new one to continue setting up your devices.

Scan the QR-code for detailed instructions

2. After logging in to all your devices go to the App-Store and search for SKILLQUBE.
3. 3a. Download the monitor surface App to your monitor iPad. (Monitor-App for qube 37/qube 15/qube x/qube)
3b. Download the ventilation-App qubeMS2 to your Monitor iPad. (Monitor-App for qube MS2)

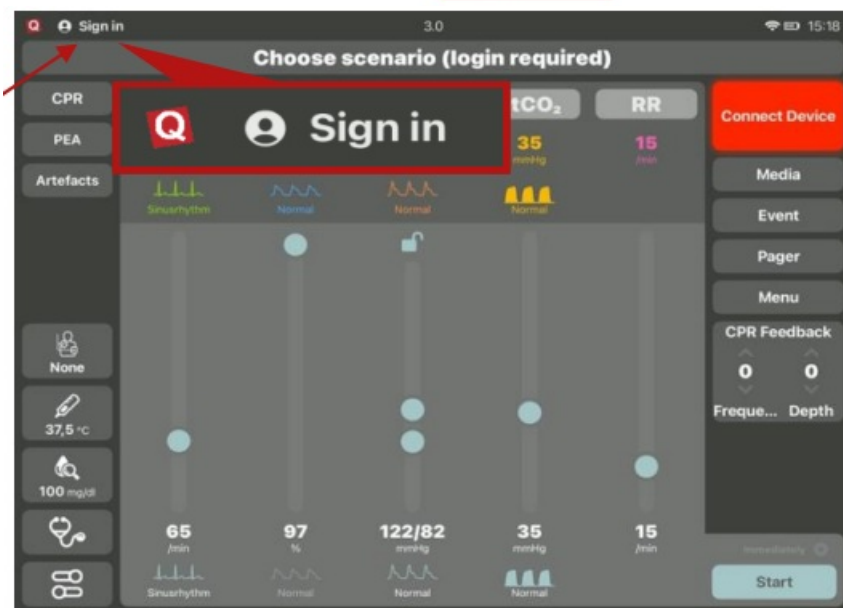

1. Download the controller-app qubeControl to your controller iPad.
2. Download the assessment-app qubeAssess to your assessment iPod.
3. Use the Log-In button in the qubeControl app to log into the qubeCLOUD with your username and password.
4. After finishing these steps, you can start training with your SKILLQUBE simulation software.

	<p>SKILLQUBE qube3 Patient Monitoring Simulation System [pdf] User Guide qube3 Patient Monitoring Simulation System, qube3, Patient Monitoring Simulation System, Monitoring Simulation System, Simulation System</p>
---	--