



stryker LIFELINKcentral AED Program Manager User Manual

[Home](#) » [stryker](#) » stryker LIFELINKcentral AED Program Manager User Manual 



LIFEPAK ® 1000 AED
Maintenance inspection update

LIFELINKcentral™ AED Program Manager

Contents

- [1 To log an inspection for one AED at a time:](#)
- [2 To log an inspection for all AEDs at once:](#)
- [3 Documents / Resources](#)
 - [3.1 References](#)
- [4 Related Posts](#)

To log an inspection for one AED at a time:

1. Visually confirm your AED(s) are "OK" and the battery shows two or more bars
2. Log in
Log into LIFELINKcentral AED Program Manager.
3. Inspections
From "Equipment," click "Manage Equipment" and then "Inspections" to see the inspection schedules for those AEDs for which you are responsible.

The screenshot displays the Stryker Lifeline Central AED Program Manager interface. The top navigation bar includes the Stryker logo, the product name 'LIFELINKcentral™ AED PROGRAM MANAGER', the user account 'Demo Customer Account / All Sites', and a welcome message for 'Scott, Michael'. The left sidebar contains navigation links: Alerts, My Account, Sites (3), Equipment (10), Manage Equipment, AEDs Ready (8 of 10), and People (6). The main content area is titled 'Equipment' and features a search bar and a list of AEDs. Two callouts are present: one pointing to the 'Manage Equipment' link in the sidebar, and another pointing to the 'Inspections' section of the 'Covington - Remote Office (AA022008)' AED details.

Manage Equipment

Inspections

4. Inspection Schedules

Next, click on the inspection schedule to which you would like to navigate. In this example, we will click on “Redmond-North Building.”

←

Inspection Schedules

Enter text to search the list below

Redmond - North Building (AA016293)

AED Inspection Schedule

Person Responsible	Howard, Ryan
Frequency	Monthly
Last Inspection Date	12/28/2020

Redmond - South Building (AA016294)

AED Inspection Schedule

Person Responsible	Bernard, Andy
Frequency	Monthly
Last Inspection Date	12/28/2020

5. Select AED

Select the AED for which you are recording the inspection:

LIFEPAK 1000 (IV620357)	
Performed By	System
Readiness Time	Not Specified
Location	Medical Clinic
NO DATA AVAILABLE	

6. Verify information (Important for first inspection, optional for future inspections)

Expand each section and verify the information.

Perform Inspection×

LIFEPAK 1000 (IV620357)

EQUIPMENT INFORMATION^

Location

Medical Clinic

Readiness Status

No Data Available

ACCESSORIES∨

INSPECTION CHECKLIST∨

NOTES∨

Submit

Cancel

Expand sections

7. Confirm AED readout

Proceed to Inspection Checklist and select the matching display readout:

EQUIPMENT INFORMATION

ACCESSORIES

INSPECTION CHECKLIST

NOTES

Inspection Checklist

Device Present ☒

Visible Damage ☐

Please select the set of indicators exactly as shown on the AED

<input checked="" type="radio"/>	<div style="background-color: black; width: 20px; height: 20px; display: inline-block;"></div>	OK	<div style="width: 20px; height: 20px; background: repeating-linear-gradient(45deg, transparent, transparent 2px, black 2px, black 4px); display: inline-block;"></div>
<input type="radio"/>	<div style="background-color: black; width: 20px; height: 20px; display: inline-block;"></div>	OK	<div style="width: 20px; height: 20px; background: repeating-linear-gradient(45deg, transparent, transparent 2px, black 2px, black 4px); display: inline-block;"></div>
<input type="radio"/>	<div style="background-color: black; width: 20px; height: 20px; display: inline-block;"></div>	OK	<div style="width: 20px; height: 20px; background: repeating-linear-gradient(45deg, transparent, transparent 2px, black 2px, black 4px); display: inline-block;"></div>
<input type="radio"/>	<div style="background-color: black; width: 20px; height: 20px; display: inline-block;"></div>	OK	<div style="width: 20px; height: 20px; background: repeating-linear-gradient(45deg, transparent, transparent 2px, black 2px, black 4px); display: inline-block;"></div>
<input type="radio"/>	<div style="background-color: black; width: 20px; height: 20px; display: inline-block;"></div>		<div style="width: 20px; height: 20px; background: repeating-linear-gradient(45deg, transparent, transparent 2px, black 2px, black 4px); display: inline-block;"></div>
<input type="radio"/>	<div style="background-color: black; width: 20px; height: 20px; display: inline-block;"></div>		<div style="width: 20px; height: 20px; background: repeating-linear-gradient(45deg, transparent, transparent 2px, black 2px, black 4px); display: inline-block;"></div>
<input type="radio"/>	<div style="background-color: black; width: 20px; height: 20px; display: inline-block;"></div>		<div style="width: 20px; height: 20px; background: repeating-linear-gradient(45deg, transparent, transparent 2px, black 2px, black 4px); display: inline-block;"></div>
<input type="radio"/>	<div style="background-color: black; width: 20px; height: 20px; display: inline-block;"></div>		<div style="width: 20px; height: 20px; background: repeating-linear-gradient(45deg, transparent, transparent 2px, black 2px, black 4px); display: inline-block;"></div>
<input type="radio"/>	<div style="background-color: black; width: 20px; height: 20px; display: inline-block;"></div>		<div style="width: 20px; height: 20px; background: repeating-linear-gradient(45deg, transparent, transparent 2px, black 2px, black 4px); display: inline-block;"></div>
<input type="radio"/>	<div style="background-color: black; width: 20px; height: 20px; display: inline-block;"></div>		<div style="width: 20px; height: 20px; background: repeating-linear-gradient(45deg, transparent, transparent 2px, black 2px, black 4px); display: inline-block;"></div>
<input type="radio"/>	<div style="background-color: black; width: 20px; height: 20px; display: inline-block;"></div>		<div style="width: 20px; height: 20px; background: repeating-linear-gradient(45deg, transparent, transparent 2px, black 2px, black 4px); display: inline-block;"></div>

Readiness Status Ready

- a. Click the “Submit” button to submit your entries.
- b. The AED status will turn green for “Ready” confirming a successfully logged inspection:

LIFEPAK 1000 (IV620357)

Performed By	Scott, Michael
Readiness Time	12/28/2020 9:17 AM
Location	Medical Clinic

READY

You will be returned to the list view of the AED Inspection Schedule page. From there, repeat the steps above to perform a maintenance inspection on the next AED in the list.

To log an inspection for all AEDs at once:

8. Visually confirm the AEDs are “OK” and the battery on each shows two or more bars
9. Log In
Log into LIFELINKcentral AED Program Manager.
10. Inspections
From “Equipment,” click “Manage Equipment” and then “Inspections” to see the inspection schedules for those AEDs for which you are responsible.

Manage Equipment

Inspections

11. Inspection Schedules

Next, click on the Inspection Schedule to which you would like to navigate. In this example, we will click on “Redmond-North Building” to log an inspection for all AEDs affiliated with that schedule:

Inspection Schedules

Enter text to search the list below

Redmond - North Building (AA016293)

AED Inspection Schedule

Person Responsible	Howard, Ryan
Frequency	Monthly
Last Inspection Date	12/28/2020

Redmond - South Building (AA016294)

AED Inspection Schedule

Person Responsible	Bernard, Andy
Frequency	Monthly
Last Inspection Date	12/28/2020

Navigate to Schedule

12. AED Inspection Schedule

Once you are on the landing page for the “Redmond-North Building” Inspection Schedule, click the “Inspect All” button to log a check for all AEDs at once.

stryker | LIFELINKcentral™
AED PROGRAM MANAGER

Demo Customer Account / Redmond - North Building

WELCOME
Scott, Michael

Alerts

- My Account
- Sites 2
- Equipment 4
- Manage Equipment >

AEDs Ready 3 of 4

- Ready 3
- No Data Available 1

People 4

Resource Center

AED Inspection Schedule

Enter text to search the list below

Inspect All **Complete Inspection**

LIFEPAK 1000 (IV620359)

Performed By	System
Readiness Time	Not Specified
Location	Security Truck

NO DATA AVAILABLE

LIFEPAK CR Plus (PC12345678)

Performed By	Majumdar, Shalini
Readiness Time	12/28/2020 8:52 AM
Location	North Building, Lobby

READY

samaritan PAD 450P (18G00002368)

Performed By	Majumdar, Shalini
Readiness Time	12/28/2020 8:52 AM
Location	First floor, next to central stairwell

READY

LIFEPAK 1000 (IV620357)

Performed By	Scott, Michael
Readiness Time	12/28/2020 9:17 AM
Location	Medical Clinic

13. Confirm

The following prompt will appear, asking you to confirm you would like to update the readiness status of all AEDs associated with this schedule to “Ready.” Click “Yes” to complete the check:

Complete Inspection

The readiness status of all Equipment not previously recorded today will be set to 'Ready' with today's date.
Do you want to proceed?

Click Yes

Yes **No**

14. Completed

You will be routed to the list view for all AEDs on that Inspection Schedule. The maintenance inspection for all AEDs listed in this schedule is complete, and all AEDs will show green for “Ready” with today’s date; in this example, 12/28/2020 at 9:23 AM:

LIFEPAK 1000 (IV620357)

Performed By
Readiness Time
Location

Scott, Michael
12/28/2020 9:23 AM
Medical Clinic

READY

LIFEPAK 1000 (IV620359)

Performed By
Readiness Time
Location

Scott, Michael
12/28/2020 9:23 AM
Security Truck

READY

For further information, please contact Stryker at 800 442 1142 (U.S.), 800 668 8323 (Canada) or visit our website at strykeremergencycare.com

Emergency Care

Stryker or its affiliated entities own, use, or have applied for the following trademarks or service marks: HeartSine, LIFELINKcentral, Stryker. All other trademarks are trademarks of their respective owners or holders.

The absence of a product, feature, or service name, or logo from this list does not constitute a waiver of Stryker's trademark or other intellectual property rights concerning that name or logo. GDR 3336346_C

Copyright © 2021 Stryker

Distributed by:

Physio-Control, Inc.
11811 Willows Road NE
Redmond, WA, 98052 U.S.A.
Toll free 800 442 1142
strykeremergencycare.com

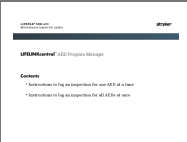
Distributed in Canada by:

Stryker Canada
2 Medicorum Place
Waterdown, Ontario
L8B 1W2
Canada
Toll free 800 668 8323

LIFEPAK 1000 AED Maintenance inspection update

stryker

Documents / Resources

	stryker LIFELINKcentral AED Program Manager [pdf] User Manual LIFELINKcentral AED Program Manager, AED Program Manager, Program Manager, Manager
---	---

References

-  [Emergency Care | Stryker](#)

Manuals+.