



**RES70051001
DISCOV-R
Battery
Ascent
Package**



SUNSET RES70051001 DISCOV-R Battery Ascent Package Owner's Manual

[Home](#) » [SunSet](#) » **SUNSET RES70051001 DISCOV-R Battery Ascent Package Owner's Manual** 

Contents

- [1 SUNSET RES70051001 DISCOV-R Battery Ascent Package](#)
- [2 Product Usage Instructions](#)
- [3 FAQ](#)
- [4 Product Information](#)
- [5 Features](#)
- [6 Ascent Package](#)
- [7 Peak Package](#)
- [8 Specs](#)
- [9 Documents / Resources](#)
 - [9.1 References](#)
- [10 Related Posts](#)



SUNSET RES70051001 DISCOV-R Battery Ascent Package



Product Usage Instructions

Charging:

Before first use, charge the device for 6 hours using the provided charger. Subsequent charges take approximately 4.5 hours for a full charge.

Replacing Oxygen Enrichment Cartridge:

To replace the oxygen enrichment cartridge, follow the instructions provided in the user manual. Ensure the device is turned off before replacing the cartridge.

Setting Flow Rates:

To adjust the flow settings, refer to the user manual for specific instructions based on your oxygen therapy needs. The device offers both pulse flow and continuous flow settings.

Connecting to Bluetooth:

Download and install the Nomad Biometrics app on your mobile device. Follow the app's instructions to pair your DISCOV-R with the app for enhanced functionality.

Traveling with the Device:

Ensure the device is FAA compliant for air travel. Follow airline regulations and guidelines when carrying the device on flights.

FAQ

- **Q: How do I clean the DISCOV-R?**

- A: Use a soft, damp cloth to wipe the exterior of the device. Do not use harsh chemicals or immerse the device in water.

- **Q: Can I use the DISCOV-R while sleeping?**

- A: It is recommended to consult with your healthcare provider before using the device while sleeping to determine the appropriate settings for safe and effective oxygen therapy.

- **Q: How often should I replace the oxygen enrichment cartridge?**

- A: Follow the manufacturer's recommendations for cartridge replacement intervals to ensure optimal performance of the device.

Item#

- RES70051001 DISCOV-R with 1 Battery (Ascent Package)

- RES70051002 DISCOV-R with 2 Batteries (Peak Package)

Product Information

Airgonomically™ designed using Belluscura's patented ModulAir™ technology, the compact DISCOV-R weighs less than 6.5 lbs, producing more oxygen per pound than any other portable oxygen concentrator.

Features

- 4 continuous flow settings
- 8 pulse flow settings
- Weighs only 6.5 lbs
- FAA compliant for air travel



Lightweight

Weighs only 6.5 pounds (2.95kg) with battery



User-Friendly

User-replaceable oxygen enrichment cartridge and \direct-charge batteries



Airgonomically™ Designed

Designed to be small and discreet for easy travel



Bluetooth Capability

Connect the DISCOV-R to your mobile device through the NomadBiometrics app



Multiple Flow Settings

8 pulse flow oxygen settings + 4 continuous flow settings delivering up to 2000mL/min



All-In-One Solution

With both pulse dose and continuous flow settings, the DISCOV-R can be used as a home unit as well as for traveling



FAA Compliant

Take your device with you when you travel by air

Ascent Package

- DISCOV-R™ Unit
- 8-Cell Battery
- Pair of User-Replaceable Cartridges
- Air Intake Filter
- AC Power Adapter
- DC Power Adapter
- Carry Bag with Handle & Shoulder Strap
- Backpack
- Cart
- Standard 3-Year Warranty

Peak Package

- DISCOV-R™ Unit
- Two (2) 8-Cell Batteries
- Pair of User-Replaceable Cartridges
- Air Intake Filter
- AC Power Adapter
- DC Power Adapter
- Carry Bag with Handle & Shoulder Strap
- Backpack
- Cart
- Standard 3-Year Warranty

Pulse and Continuous Flow Portable Oxygen Concentrator. The DISCOV-R weighs 6.5 lbs*, with a lightweight design that is perfect for patients looking to replace their heavy, metal oxygen tanks and live a more mobile, active lifestyle.

*Weight with battery.


Specs

- Dimensions L x W x H: 9.9 in x 4.1 in x 7.9 in
- Weight ≤ 6.5 lbs (2.95 kg)
- Auditory Output < 39 dB(A) at Level 2
- Charge Duration ≤ 4.5 Hrs
- Charge Time ≤ 6 Hrs

To order or for more information

- P 312.997.9980
- E sales@sunsethcs.com
- W SUNSETHCS.COM

Documents / Resources

	<p>SUNSET RES70051001 DISCOV-R Battery Ascent Package [pdf] Owner's Manual RES70051001, RES70051002, RES70051001 DISCOV-R Battery Ascent Package, RES70051001, DISCOV-R Battery Ascent Package, Battery Ascent Package, Ascent Package</p>
---	--

References

- [User Manual](#)

Manuals+, Privacy Policy

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.