



SYLONG B0BKKJKYL2 Elbow Splint User Manual

[Home](#) » [Sylong](#) » SYLONG B0BKKJKYL2 Elbow Splint User Manual 



Contents

1 How to use the elbow splint

2 Features:

3 Documents / Resources

3.1 References

How to use the elbow splint



Hold the elbow splint in your hand and confirm that the narrow side is in the direction of the palm.



Wrap 3 hook and loop straps around the elbow and thread through the buckles, then secure to desired compression.



Wrap support around the elbow and make sure the middle strap is in the elbow crease position, secure for a comfortable fit.



Continue to adjust the elbow splint in a comfortable position according to the previous 3 steps.

	
<p>Attach one Hook & Loop of shoulder strap to the back of the elbow brace.</p>	<p>Attach another Hook & Loop of shoulder strap to the front of the elbow brace.</p>
	
<p>Wrap the shoulder strap around the shoulder on the other side.</p>	<p>Adjust the tightness, then you can sleep without worrying about the elbow splint slipping off your arm.</p>

Features:

Fits: Right or left arm.

Material: Foam, Nylon, Polyester, Latex free

Care: Hand wash and air dry flat. Please remove the plastic inserts before the wash.

Color: Black, Blue

Elbow Splint

3 Plastic support plates inside



Documents / Resources



[SYLONG B0BKKJKYL2 Elbow Splint](#) [pdf] User Manual
B0BKKJKYL2 Elbow Splint, B0BKKJKYL2, Elbow Splint, Splint

References

- [User Manual](#)

[Manuals+](#), [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.