

Sylong Rest Stroke Resting Hand Splint



# Sylong Rest Stroke Resting Hand Splint User Manual

[Home](#) » [Sylong](#) » Sylong Rest Stroke Resting Hand Splint User Manual 

## Contents

- 1 [Sylong Rest Stroke Resting Hand Splint](#)
- 2 [How to use](#)
- 3 [Warranty](#)
- 4 [Documents / Resources](#)
  - 4.1 [References](#)



## Sylong Rest Stroke Resting Hand Splint



## How to use

### How to use this rest hand splint?

- **Step 1**
  - Insert 4 fingers into the corresponding finger holes.
- **Step 2**
  - Fix the thumb with the thumb strap.



- **Step 3**
  - Fix 4 fingers with a palm strap.
- **Step 4**
  - Fix the wrist with the middle wrist strap.



• **Step 5**

- Bend the lowest wrist support forcefully to make it more suitable for your arm.

• **Step 6**

- Fix the arm with the bottom wrist strap.



• **Step 7**

- Fix one end of the separate palm strap on the palm obliquely 45 degrees as shown.

• **Step 8**

- Pass between the thumb and the palm, and wrap the palm strap around the palm.



• **Step 9**

- Fix another end of the separate palm strap.

• **Step 10**

- Bend the palm rest to a comfortable angle.



• **Step 11**

- Bend the thumb splint to a comfortable angle.

- **Step 12**

- Bend the splint at the wrist to a comfortable angle. Finish.



- **Fits:** Men or Women

- **Material:** Foam, Fabric, Aluminum, Nylon

- **Care:** Hand wash and air dry flat

- **Colour:** Black

- This resting hand splint is designed by DrFinger's health technology team and orthopedics doctors through continuous research and testing to conform to the ergonomic shape of the resting splint.
- It is very suitable for patients with stroke fingers, ruptured metacarpal bones, tendinitis, joint sprain treatment, postoperative care, carpal tunnel syndrome, other wrist injuries, and various hand dysfunctions.
- If you feel that our Hand Splint is working, we would appreciate your sharing your experience on Amazon so other customers can make more informed choices and we will know our efforts pay off.
- If you think our Rest Hand Splint can be improved in some way, you also can leave a review on Amazon or email us.
- Your valuable suggestions will be highly appreciated and we will improve our products and services upon your suggestions.
- Maybe your suggestions will make a better world.
- Wish you a quick recovery in your hand
- Thank you

## **Warranty**

### **Respected user,**

- We solemnly promise the reputation of the company.
- If you contact us by email, we promise that the product guarantee period will be extended to 12 months for you. Just tell us your order number, and we will register this order number in our 12-month warranty system.
- As long as you let us register the order number for you if you find that any quality problem causes the product to not be used normally within 12 months from the date of purchase, we promise that we will send you a new product for free.
- If you think this product is not effective for your symptoms, we will refund you the full amount, even though Amazon's 30-day return period has passed.
- **Email:** [sylongus@outlook.com](mailto:sylongus@outlook.com).
- **Promiser:**



- CEO of Sylong Co., Ltd.
- High-density aluminium material inside the splint Can adjust the shape.



---

**Documents / Resources**



## [Sylong Rest Stroke Resting Hand Splint](#) [pdf] User Manual

Rest Stroke Resting Hand Splint, Rest, Stroke Resting Hand Splint, Resting Hand Splint, Hand Splint

## References

- [User Manual](#)