



Teleflex PR-34063-HPSL Pressure Injectable Three-Lumen PICC Instruction Manual


[Home](#) » [Teleflex](#) » Teleflex PR-34063-HPSL Pressure Injectable Three-Lumen PICC Instruction Manual 

Contents

- [1 Teleflex PR-34063-HPSL Pressure Injectable Three-Lumen PICC](#)
- [2 Documents / Resources](#)
 - [2.1 References](#)
- [3 Related Posts](#)

ARROW®

Teleflex PR-34063-HPSL Pressure Injectable Three-Lumen PICC

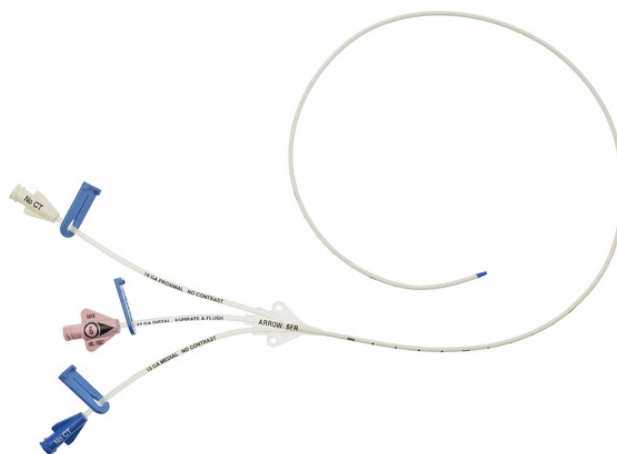


Contents:

- Three Lumen Catheter: 6 Fr. (2.14 mm OD) x 40 cm, Pressure Injectable, Blue Flex Tip®
- Peel-Away Sheath: 6.5 Fr. x 4" (10 cm) Radiopaque over 6.5 Fr. Dilator
- Spring-Wire Guide: .018" (0.46 mm) dia. x 17-3/4" (45 cm) (Straight Soft Tip on Both Ends)
- Spring-Wire Guide, Marked: .018" (0.46 mm) dia. x 51-3/16" (130 cm)(Straight Soft Tip on One End – Straight Stiff Tip on Other)
- Introducer Needle: Echogenic 21 Ga. x 2-3/4" (7 cm) TW
- Syringe: 5 mL Luer-Lock
- Skin Protectant Prep
- Dust Cap: Non-Vented
- Second Site™ Adjustable Hub: Fastener
- Second Site™ Adjustable Hub: Catheter Clamp
- Flow Rate Information Card with Injection Log
- Patient ID Card
- Chart Sticker
- Patient Information Booklet
- Checklist/CLIP Sheet
- Paper Tape Measure
- Dressing: STATLOCK®1 Catheter Stabilization Device

1A registered trademark of C. R. Bard, Inc.

Rx only. Warning: Read all package insert warnings, precautions and instructions prior to use. Failure to do so may result in severe patient injury or death. Fluid path components are non-pyrogenic. Contraindications: The Pressure Injectable PICC is contraindicated wherever there is presence of device related infections or presence of previous or current thrombosis in the intended insertion vessel or catheter pathway. Clinical assessment of patient must be completed to ensure no contraindications exist.

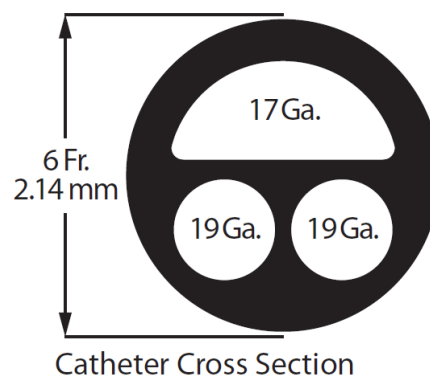


Gravity MAX Pressure Priming Flow Pump Flow Injection

- Volume* Rate† Rate‡ Flow Rate**
- Lumen (mL) (mL/hr) (mL/hr) (mL/sec)
- Distal (17 Ga.) 0.55 1650 6800 6
- Medial (19 Ga.) 0.44 550 2820 not rated
- Proximal (19 Ga.) 0.42 550 2920 not rated
- * Priming volumes are approximate and are done without accessories.
- † Flow rate values are approximate and are determined using deionized water at

- 100 cm head height.
- ‡ Pump flow rates are determined at maximum pump pressure of 10 psig and
- represent approximate tow capabilities.
- ** Pressure injection flow rates are determined at the injector pressure setting of 300
- psi maximum using media of 11.8 centipoise viscosity, with 152 cm pressure tubing
- Arrow International, Inc.
- Subsidiary of Teleflex Incorporated
- 2400 Bernville Road
- Reading, PA 19605 USA
- www.teleflex.com
- IR-34063-101D (2021-08)

Documents / Resources

	<p>Teleflex PR-34063-HPSL Pressure Injectable Three-Lumen PICC [pdf] Instruction Manual PR-34063-HPSL, Pressure Injectable Three-Lumen PICC, PR-34063-HPSL Pressure Injectable Three-Lumen PICC, Three-Lumen PICC, PICC</p>
---	---

References

- [Welcome to Teleflex | US | Teleflex](#)