



Teleflex PR-35541-HPSL Pressure Injectable One-Lumen PICC Instruction Manual

[Home](#) » [Teleflex](#) » Teleflex PR-35541-HPSL Pressure Injectable One-Lumen PICC Instruction Manual 

Contents

- 1 [Teleflex PR-35541-HPSL Pressure Injectable One-Lumen PICC](#)
- 2 [Contents](#)
- 3 [Rx only](#)
 - 3.1 [Contraindications](#)
- 4 [Specification](#)
- 5 [Documents / Resources](#)
 - 5.1 [References](#)
- 6 [Related Posts](#)

Teleflex®

Teleflex PR-35541-HPSL Pressure Injectable One-Lumen PICC



Contents

- 1: One Lumen TaperFree® Catheter: 4 Fr. (1.45 mm OD) x 55 cm, Pressure Injectable, Blue FlexTip®
- 1: GlideThru™ Peel-Away Sheath: 4 Fr. x 4" (10 cm) Radiopaque over 4 Fr. Dilator
- 1: Spring-Wire Guide: .018" (0.46 mm) dia. x 17-3/4" (45 cm) (Straight Soft Tip on Both Ends)
- 1: Spring-Wire Guide, Marked: .018" (0.46 mm) dia. x 51-3/16" (130 cm)(Straight Soft Tip on One End – Straight Stiff Tip on Other)
- 1: Introducer Needle: Echogenic 21 Ga. x 2-3/4" (7 cm) TW
- 1: Syringe: 5 mL Luer-Lock
- 1: Skin Protectant Prep
- 1: Dust Cap: Non-Vented
- 1: SecondSite™ Adjustable Hub: Catheter Clamp
- 1: SecondSite™ Adjustable Hub: Fastener
- 1: Patient ID Card
- 1: Chart Sticker
- 1: Patient Information Booklet
- 1: Checklist/CLIP Sheet
- 1: Paper Tape Measure
- 1: Dressing: STATLOCK®1 Catheter Stabilization Device
- 1A registered trademark of C. R. Bard, Inc.

Rx only

Warning:

Read all package insert warnings, precautions, and instructions prior to use. Failure to do so may result in severe patient injury or death.

California Prop. 65

WARNING: Cancer and Reproductive Harm. www.P65Warnings.ca.gov

Not made with natural rubber latex

Fluid path components are non-pyrogenic.

Contraindications

The Pressure Injectable PICC is contraindicated wherever there is presence of device related infections or presence of previous or current thrombosis in the intended insertion vessel or catheter pathway. Clinical assessment of patient must be completed to ensure no contraindications exist.

Caution



Specification



Lumen

	Priming Volume* (mL)	Gravity Flow Rate† (mL/hr)	Pump Flow Rate†† (mL/hr)	MAX Pressure Injection Flow Rate** (mL/sec)
Distal (17 Ga.)	0.58	946	5720	5

- Priming volumes are approximate and are done without accessories.
 - Flow rate values are approximate and are determined using deionized water at 100 cm head height.
 - Pump flow rates are determined at maximum pump pressure of 10 psig and represent approximate flow capabilities.
- Pressure injection flow rates are determined at the injector pressure setting of 300 psi maximum using media of 11.8 centipoise viscosity, with 152 cm pressure tubing.

Arrow International LLC

Subsidiary of Teleflex Incorporated 3015 Carrington Mill Blvd.
Morrisville, NC 27560 USA.

www.teleflex.mm.

Documents / Resources

	Teleflex PR-35541-HPSL Pressure Injectable One-Lumen PICC [pdf] Instruction Manual PR-35541-HPSL Pressure Injectable One-Lumen PICC, PR-35541-HPSL, Pressure Injectable One-Lumen PICC, Injectable One-Lumen PICC, One-Lumen PICC, PICC
---	--

References

-  p65warnings.ca - [p65warnings Resources and Information](http://p65warnings.ca).