



Timago FS 913L 2 Function Folding Walker Instruction Manual

[Home](#) » [Timago](#) » Timago FS 913L 2 Function Folding Walker Instruction Manual 

Contents

- 1 Timago FS 913L 2 Function Folding Walker
- 2 Product Usage Instructions
- 3 Frequently Asked Questions
- 4 characteristics
- 5 Elements of the walker
- 6 Application
- 7 Assembly
- 8 Technical data
- 9 Maintenance
- 10 Warranty information
- 11 Labels
- 12 Documents / Resources
 - 12.1 References





Specifications

- **Overall width:** 55 cm
- **Inner width:** 46 cm
- **Overall depth:** 46 cm
- **Height adjustment:** 72-91 cm
- **Weight:** 2.4 kg
- **Maximum capacity:** 136 kg
- **Frame color:** Silver, bronze

Product Usage Instructions

Assembly

Unfolding and folding: Make sure the pins are set at the same level in each leg.

Application

- The 2-function walker is designed to provide stable support and assistance to individuals with reduced mobility.
- It is ideal for the elderly to improve balance, and walking comfort, and prevent falls.
- The walker aids in the rehabilitation of individuals with lower limb issues and mobility challenges.

Elements of the Walker

- **a.** Handgrip
- **b.** Frame folding clip
- **c.** Function change
- **d.** Height adjustment
- **e.** Anti-sliding rubber tip

Notes

- Timago International Group does not hold responsibility for improper use, failure to comply with safety regulations, or misuse of the walker.

Storage and Transport Conditions

- Avoid exposing the device to direct sunlight, water, or moisture when storing or transporting.

Warranty Information

- All products come with a warranty described in the warranty card available on the website. Retain proof of purchase for warranty claims.

Labels

- **Reference number**
- **Manufacturer LOT number**
- **Manufacture date**
- **Serial number**
- **Medical device note**

Frequently Asked Questions


- **Q: How do I adjust the height of the walker?**
 - **A:** To adjust the height, locate the height adjustment feature on the walker and set it to your desired level within the range of 72-91 cm.
- **Q: Can the walker be used by individuals weighing more than 136 kg?**
 - **A:** It is recommended to adhere to the maximum weight capacity of 136 kg for safe usage of the walker.
- **Q: Is the walker suitable for outdoor use?**
 - **A:** While the walker can be used outdoors, it should be protected from direct sunlight, water, and moisture to maintain its quality and performance.
- **Q: How can I contact Timago International Group for support?**
 - **A:** You can reach out to Timago International Group via email at info@timago.com or by phone at +48334995000 for assistance and inquiries.

characteristics

- 2-function folding walker is made of a crosswise reinforced aluminum frame.
- **The walker has 2 functions:** the rigid and the reciprocating, which allows one to move the sides alternatively without lifting the walker.
- The walker provides a wide area of support, as well as a long range of walking. Soft and foamed handgrips provide proper stabilization and comfortable use.
- Owing to adjustable legs, the height of the walker can be adjusted according to the user's height.
- Rubber tips prevent the walker from sliding. Folded walker takes little space, and thus, can be easily stored and transported.

Elements of the walker

- **a.** Handgrip
- **b.** Frame folding clip
- **c.** Function change
- **d.** Height adjustment
- **e.** Anti-sliding rubber tip

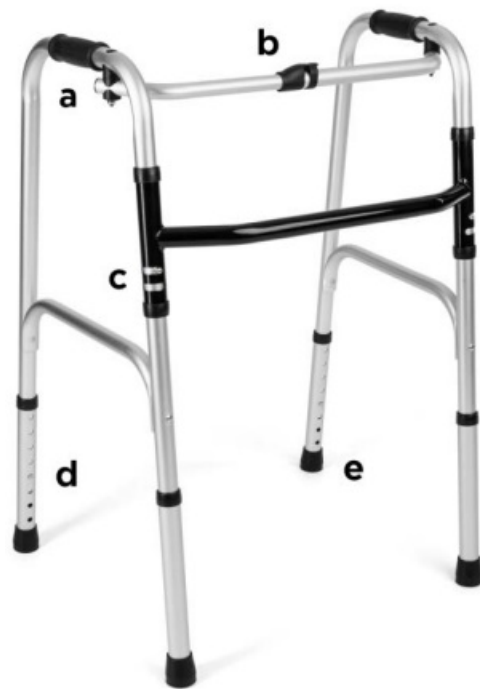

Application

- 2-function walker is a support equipment designed for people with reduced mobility who need stable support and help while walking.
- It is also recommended to the elderly as an aid in keeping balance, improving walking comfort, and ensuring safety from falling.
- The use of the walker affects positively the return to activity of people with lower limb diseases and with locomotion or verticalization problems.

Assembly

Unfolding and folding

- To unfold the walker, pull the sides apart to the maximum position until the handgrips are locked.
- To fold the walker, press the plastic element located in the middle of the crosswise frame and fold the sides.

Height adjustment

- The height should be set on such a level that the user, who is leaning on the handles, stands upright, as far as he or she can, with arms in a resting position and elbows slightly bent. It is inadvisable to hang on the walker with all the body weight.
- To adjust the height of the legs, press the metal pin, select the appropriate height of the leg, and lock the position of the legs relative to each other. The legs are adjustable in 2.5 cm increments.
- **Note!** The pins must be set on the same level in each leg.

Selection of functions

- **Rigid function** – place both of the locking pins in the upper horizontal openings, used for function changing (c)
- **Reciprocating function** – press the rubber element located in the middle of the frame, fold the legs slightly, and place the locking pins in the lower horizontal openings, used for function changing (c)

Technical data

Overall width	55 cm
Inner width	46 cm
Overall depth	46 cm
Height adjustment	72-91 cm

Weight	2.4 kg
Maximum capacity	136 kg
Frame colour	silver, bronze

- **Notes** Timago International Group does not bear responsibility of improper use of the walker, failure to comply with safety regulations and misuse.

Maintenance

- Clean the frame of the walker with a damp cloth using only water and mild detergents. Dry thoroughly.
- Do not use for cleaning: strong polishing and abrasive agents, strong detergents, bleach, petrol, or solvents (it may cause frame damage, visible defects, and in consequence – loss of warranty rights).
- Remove any dirt found on the moving parts of the walker (hinges, clamping screws, locks).










Storage and transport conditions

- The device should not be exposed to direct sunlight, water, or moisture.

Warranty information

- All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website.
- Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

- Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!
- **Timago International Group Ossowski Sp. k.**
- **ul. Karpacka 24/12**
- **43-316 Bielsko-Biała, Poland**
- **T.: +48 33 4995000**
- **F.: +48 33 4995011**
- **E.: info@timago.com**
- **01/2025(I)**


Documents / Resources

INSTRUCTIONS FOR USE
2 Function Folding Walker
 Product code: FS 93L

1. Description
 2 Function folding walker is made of extruded reinforced aluminum. It has a width of 38 cm, height 70 cm and the weight 4,5 kg. It is designed for use by people with reduced mobility, who are unable to walk independently without the help of the walker. The walker provides a wide area of support, is stable in a large range of walking, and it can be folded to provide a compact and portable device. It can be used on stairs (using its adjustable foot), the height of the walker can be adjusted according to user's height. It also comes with a seat for extra comfort. Folded walker takes little space, and thus, can be easily stored in a cupboard.

Dimensions of the walker:

- Height 70 cm
- Frame folding dia 38 cm
- Frame width 38 cm
- Weight 4,5 kg
- Folded length 60 cm



2. Application
 2 Function folding walker is a support equipment designed for people with reduced mobility, who need to support walking and balance while walking. It is also recommended for the elderly or all who are learning to walk. It improves walking posture and encourages safety when walking. The use of the walker affects posture in the relation to the ability of the person who has been fitted. It does not affect locomotion or neurological conditions.

Timago FS 93L 2 Function Folding Walker [pdf] Instruction Manual

FS 93L, FS 931L 2 Function Folding Walker, 2 Function Folding Walker, Folding Walker, Walker

References

- [!\[\]\(36f8637baaa56c4be44b454435949289_img.jpg\) Timago - Producent sprzętu medycznego](#)
- [User Manual](#)

Manuals+, Privacy Policy

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.