



HUGO Folding Walker



Timago HUGO Folding Walker Instruction Manual

[Home](#) » [Timago](#) » Timago HUGO Folding Walker Instruction Manual 



Contents

- [1 Timago HUGO Folding Walker](#)
- [2 Product Specifications](#)
- [3 Product Information](#)
- [4 Product Usage Instructions](#)
- [5 FAQ](#)
- [6 Characteristics](#)
- [7 Application](#)
- [8 Assembly](#)
- [9 Technical data](#)
- [10 Maintenance](#)
- [11 Storage and transport conditions](#)
- [12 Warranty information](#)
- [13 Labels](#)
- [14 ABOUT COMPANY](#)
- [15 Documents / Resources](#)
 - [15.1 References](#)
- [16 Related Posts](#)

Timago HUGO Folding Walker



Product Specifications

- **Overall width:** 50 cm
- **Overall length:** 56 cm
- **Height adjustment:** 78-95.5 cm
- **Weight:** 2.6 kg
- **Maximum capacity:** 136 kg
- **Frame colour:** Silver

Product Information

The Folding walker (Product code: HUGO) is made of reinforced, anodized aluminium frame with a wide area of support. It features soft and foamed handgrips for proper stabilization and comfortable use. The height of the walker is adjustable to fit the user's height, and rubber tips prevent sliding. When folded, it takes up minimal space for easy storage and transportation.

Product Usage Instructions

Application:

The walker is designed for individuals with reduced mobility who require stable support while walking. It is also suitable for the elderly to aid in balance, improve walking comfort, and prevent falls.

Assembly – Unfolding and Folding:

Ensure that the pins are set at the same level on each leg for proper assembly.

Maintenance:

Regularly visually inspect all parts of the product to ensure proper functioning.

Storage and Transport Conditions:

Avoid exposing the device to direct sunlight, water, or moisture to maintain its condition.

Warranty Information:

All products are covered by a warranty described in the warranty card available on the website. Retain proof of purchase for warranty purposes.

FAQ

- **What is the weight capacity of the walker?**

The maximum weight capacity of the walker is 136 kg.

- **How should I adjust the height of the walker?**

Use the adjustable legs to set the height of the walker according to your own height.

- **Can I store the walker in direct sunlight?**

It is not recommended to expose the walker to direct sunlight, water, or moisture for storage or transport.

Characteristics

Folding walker is made of reinforced, anodized, aluminium frame and has a wide area of support. Soft and foamed handgrips provide proper stabilization and comfortable use. Owing to adjustable legs, the height of the walker can be adjusted according to user's height. Rubber tips prevent the walker from sliding. Folded walker takes up little space, and thus, can be easily stored and transported.

Elements of the walker:**Application**

The walker is an ancillary equipment designed for people with reduced mobility who need stable support and help while walking. It is also recommended to the elderly as an aid in keeping balance, improving walking comfort and ensuring safety from falling. The use of the walker has a positive effect on the return to activity of people with lower

limbs diseases and with locomotion or tilting to erect position problems.

Assembly

Unfolding and folding

- To unfold the walker, pull the sides apart to the maximum position until the handgrips are locked.
- To fold the walker, press the plastic element located in the middle on the transverse frame and fold the sides.

Height adjustment

- The height should be set on such a level that the user, who is leaning against the handgrips, stands upright, as far as he or she is able to, with his or her shoulders in rest position and with slightly bent elbows. It is inadvisable to hang on the walker with all the body weight.
- To adjust the height of legs, press the metal pin, select appropriate height of the inner tube, and lock the position of the tubes relative to each other. The legs are adjustable in 2.5 cm increments.

Note! The pins must be set on the same level in each leg.

Technical data

Overall width	50 cm
Overall length	56 cm
Height adjustment	78-95,5 cm
Weight	2,6 kg
Maximum capacity	136 kg
Frame colour	silver

Notes

Timago International Group does not bear responsibility of improper use of the walker, failure to comply safety regulations and misuse.

Maintenance

Reviews

Systematically make a visual inspection of all the parts of the product including:

- the condition of height adjustment pins, latches, hinges etc.
- stability and visual appearance of the frame
- the condition of tubes (for cracks)

Cleaning

- Clean the frame of the walker with damp cloth using only water and mild detergents. Dry thoroughly.
- Do not use for cleaning: strong polishing and abrasive agents, strong detergents, bleach, petrol or solvents (it

may cause frame damage, visible defects and in consequence – loss of warranty rights).

- It is essential to remove impurities from moving parts of the walker (hinges, clamping screws, locks).




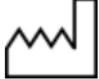





Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		


Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!

ABOUT COMPANY

- Timago International Group
- Spółka z o.o. i Spółka – Spółka komandytowa ul. Karpacka 24/12
- 43-316 Bielsko-Biała, Poland
- T.: +48 33 499 50 00
- F.: +48 33 499 50 11
- E.: info@timago.com


Documents / Resources

INSTRUCTIONS FOR USE
Folding walker
Produkt-Gebrauchsanweisung



1. Charakteristika
 Folding walker is made of reinforced, anodized aluminum frame and has a wide area of support. With its forward low height provides perfect stability and control for use. Using its adjustable size, the height of the walker can be adjusted according to user's height. Walker has perfect fit to user from sitting. Folding walker can be folded open and shut, with its own stand and transport.

Details of the walker:



2. Application:
 The walker is a walking support designed for people with reduced mobility who need stable support and help while walking. It is also recommended for elderly who are still in learning, balancing, maintaining a strong, steady and safe walking. The walker can be used for a long time. It has a positive effect on the control, stability, strength with front limbs, balance and self-confidence in sitting, as well as perfect stability.

Timago HUGO Folding Walker [pdf] Instruction Manual

HUGO Folding Walker, HUGO, Folding Walker, Walker

References

- [!\[\]\(5ba1bc70d78f05c00988641e5e513c62_img.jpg\) Timago - Producent sprzętu medycznego](#)
- [User Manual](#)

[Manuals+](#), [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.