



tobii dynamox Switcher Software User Manual

[Home](#) » [tobii dynamox](#) » tobii dynamox Switcher Software User Manual 

tobii dynamox

Switcher
User's Manual



dynamox Switcher Software

Contents

- [1 Switcher Software](#)
- [2 About Switcher](#)
- [3 Switcher](#)
- [4 Documents / Resources](#)

Switcher Software

User's manual Switcher

Version 1.0

04/2022

All rights reserved.

Copyright © Tobii Dynavox AB

No part of this document may be reproduced, stored in a retrieval system, or transmitted in any form, by any means (electronic, photocopying, recording, or otherwise) without the prior written permission of the publisher.

Copyright protection claimed includes all forms and matters of copyrightable material and information allowed by statutory or judicial law or hereafter granted, including without limitation, material generated from the software programs which are displayed on the screen such as screen displays, menus, etc.

The information contained in this document is proprietary to Tobii Dynavox. Any reproduction in part or whole without prior written authorization by Tobii Dynavox is prohibited.

Products that are referred to in this document may be either trademarks and/or registered trademarks of the respective owners. The publisher and the author make no claim to these trademarks.

While every precaution has been taken in the preparation of this document, the publisher and the author assume no responsibility for errors or omissions, or for damages resulting from the use of information contained in this document or from the use of programs and source code that may accompany it. In no event shall the publisher and the author be liable for any loss of profit or any other commercial damage caused or alleged to have been caused directly or indirectly by this document.

Content subject to change without notice.

Please check Tobii Dynavox website.

www.TobiiDynavox.com for updated versions of this document.

Contact Information:

Tobii Dynavox AB Karlsrovägen 2D 182 53 Danderyd Sweden +46 8 663 69 90	Tobii Dynavox LLC 2100 Wharton Street, Suite 400 Pittsburgh, PA 15203 USA +1-800-344-1778
Tobii Dynavox Ltd. Sheffield Technology Parks Cooper Buildings Arundel Street Sheffield S1 2NS United Kingdom +46 8 663 69 90	TingDao Electronics Science & Technology (Suzhou) Co Unit 11/12, Floor 3, Building B, No.5 Xinghan Street, SIF P.R.China 215021 +86 512 69362880



About Switcher

1.1 System Requirements

Component	Requirements
Computer and processor	i5-4200U @ 1.60 GHz (4th gen i5 with two cores / 4 threads)
Memory (RAM)	8 gigabyte (GB) RAM (recommended minimum).
Hard Disk	500 megabytes (MB) available.
Operating System	Windows 10 and Windows 11
.NET version	4.7.2
Eye tracker	Tobii Dynavox I-Series I-13 & I-16 and Tobii Dynavox PCEye 5
Additional requirements and considerations	Internet connection recommended for receiving updates.

Switcher

Switcher is a help app, allowing easy switching between installed Tobii Dynavox Apps, apps in general and open windows.


2.1 How do I Access Switcher in a TD Software?

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker.



The Switcher app will become visible in the lower middle part of the screen.

2. Select the Switcher  Switcher will open.

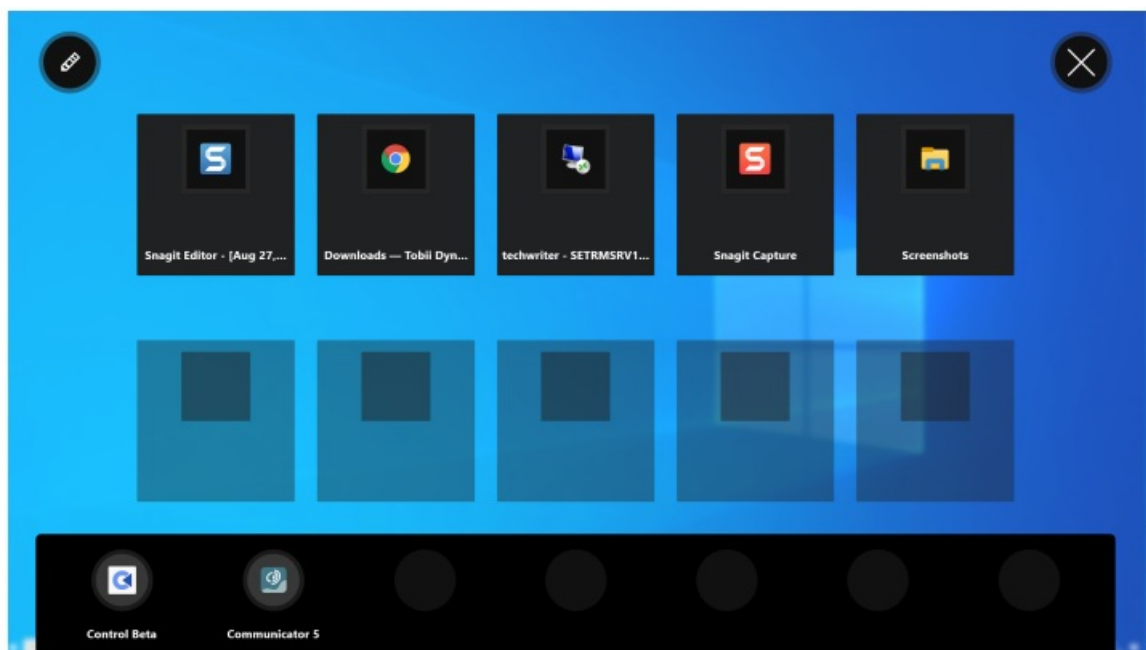

2.2 How do I Switch to an Application in Switcher



If you want to add a non Tobii Dynavox application, please make sure to start the wanted application before proceeding.

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker.




The Switcher app will become visible in the lower middle part of the screen.

2. Select the Switcher




3. Select the wanted application.

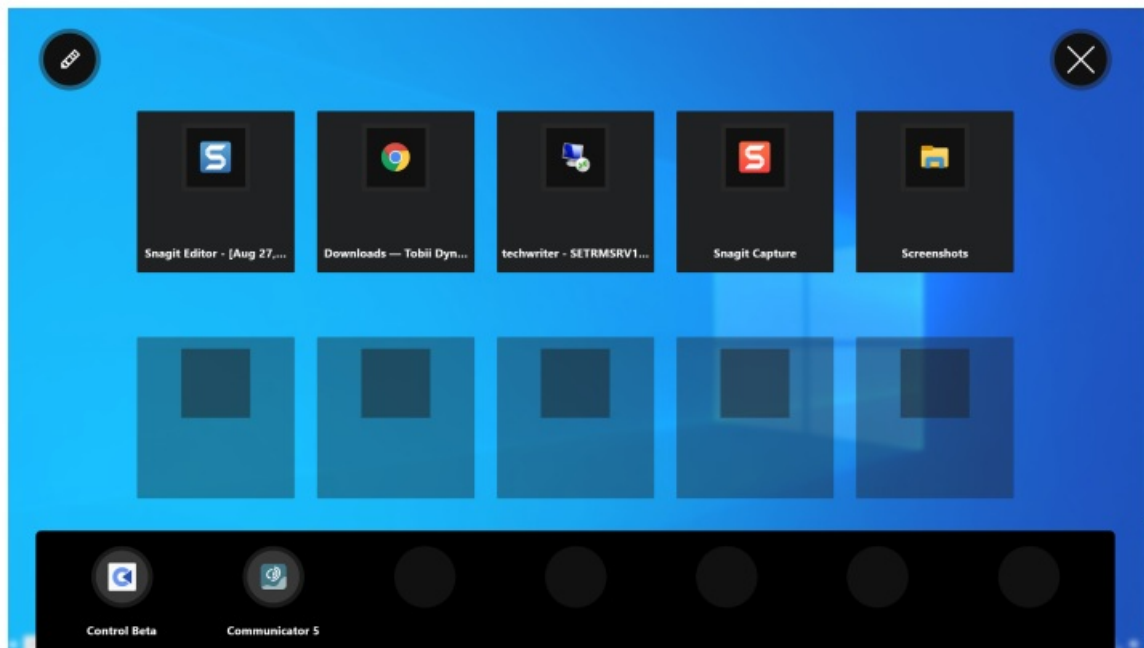

2.3 How do I Add an Application to Switcher?

If you want to add a non Tobii Dynavox application, please make sure to start the wanted application before proceeding.


1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker.

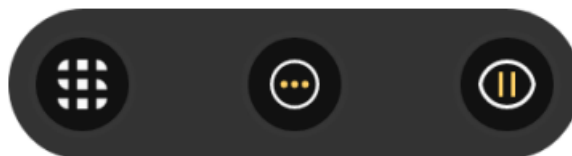


The Switcher app will become visible in the lower middle part of the screen.

2. Select the Switcher



3. Select the  (Edit) button in the top left corner.


4. Select the  (Add) button for the wanted location.

5. Select the correct tab:

- Tobii Dynavox

- Running Application

6. Select the wanted application.

7. Select the  (Done) button in the top left corner.

8. Select the  (Close) button.


2.4 How do I Remove an Application from Switcher?


1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker.




The Switcher app will become visible in the lower middle part of the screen.

2. Select the Switcher 

3. Select the  (Edit) button in the top left corner.

4. Select the  (Remove) button for the wanted location.

5. Select the Remove button to confirm.

6. Select the  (Done) button in the top left corner.

7. Select the  (Close) button.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact


Copyright ©Tobii Dynavox AB (Publ).

Not all products and services offered in each local market. Specifications are subject to change without prior notice.

All trademarks are the property of their respective owners.

#12008273 Switcher User's manual v.1.0 – en-US

Documents / Resources

 <small>Switcher User's Manual</small> <small>tobii dynavox</small>	tobii dynavox Switcher Software [pdf] User Manual Switcher Software, Switcher, Software
--	--

