

[Skip to content](#)

Manuals+

User Manuals Simplified.



tobii dynavox TD I-110 Display User Guide

[Home](#) » [tobii dynavox](#) » **tobii dynavox TD I-110 Display
User Guide**

Contents [hide](#)

[1 tobii dynavox TD I-110 Display](#)

[2 What's in the Box?](#)

[3 Getting to Know Your Device](#)

[3.1 Step 1: Initial Setup](#)

[4 Windows Setup](#)

[4.1 Step 2: Set Up TD Snap](#)

[4.2 Step 3: Mount & Position](#)

[4.3 Step 4: TD Snap Training Cards](#)

[5 Additional Resources](#)

[5.1 Scan the QR codes with your phone or use the links.](#)

[6 Documents / Resources](#)

[6.1 References](#)


[7 Related Posts](#)

tobii dynavox

tobii dynavox TD I-110 Display




What's in the Box?

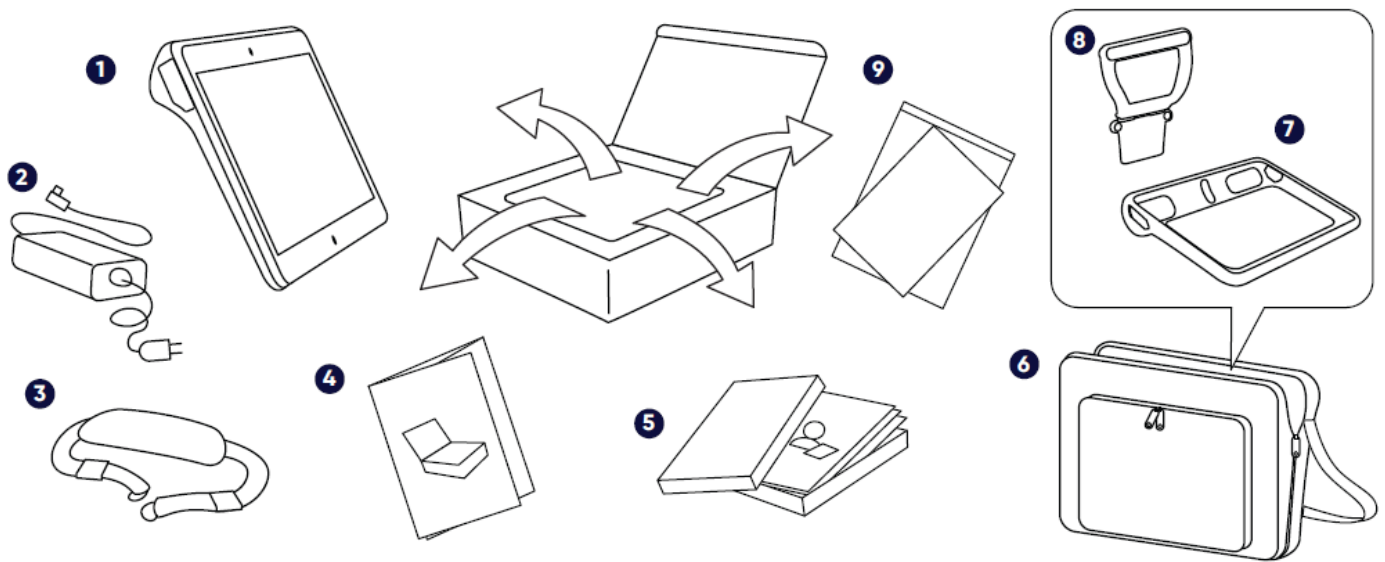

1. Device
2. Power Cable
3. Shoulder Strap
4. Getting Started
5. Guide TD Snap Training Cards
6. Carry Case
7. Durable Case
8. Handle Mount Plate
9. Safety & Compliance, License, and Configuration Documents

Getting to Know Your Device

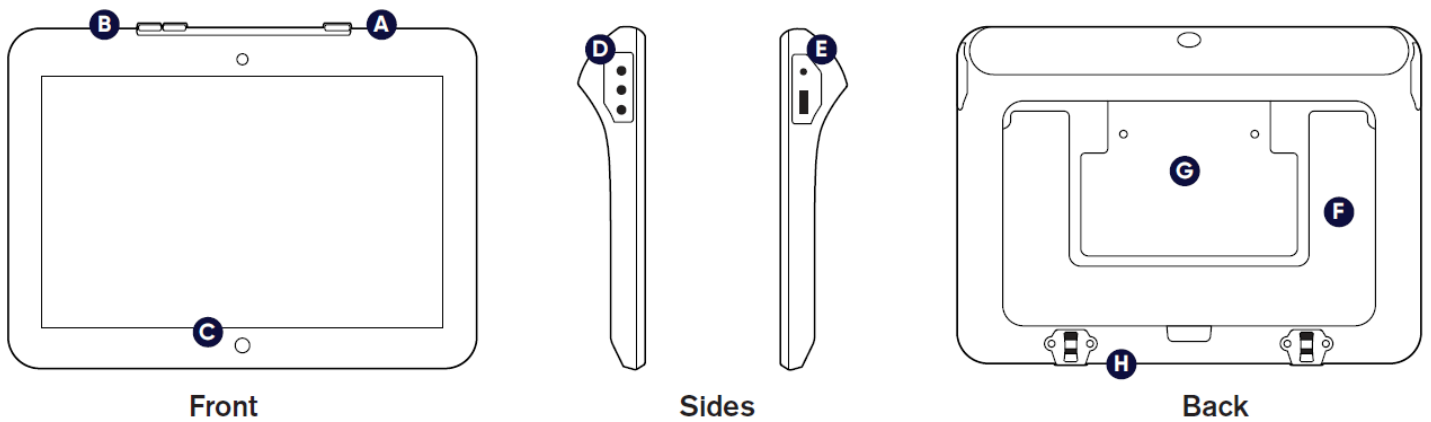


- A Power Button
- B Volume Buttons
- C Home Button
- D Audio & Switch Ports
- E Power Connector & USB Ports
- F Stand
- G Mounting Location
- H Strap Connectors

Follow the steps in this guide to set up and start using your device.


Step 1: Initial Setup

Start the Device

1. Connect the power cable to the device and plug in to an outlet

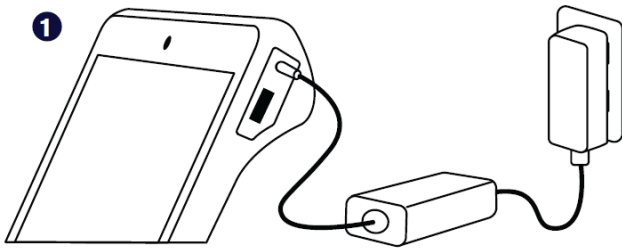


2. Press the Power button

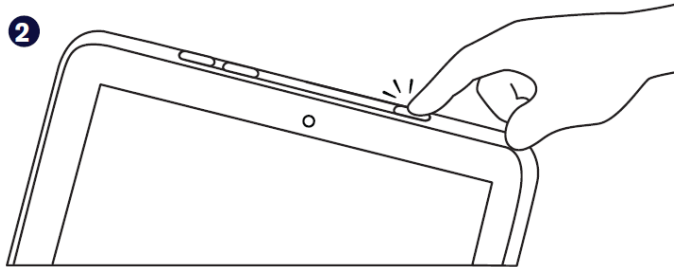

Windows Setup

If your device has not been pre-configured, you will be prompted to complete the Windows Setup process first. While this process does not require a Microsoft account, we recommend that you connect to or create the user's account and not a caregiver's account.

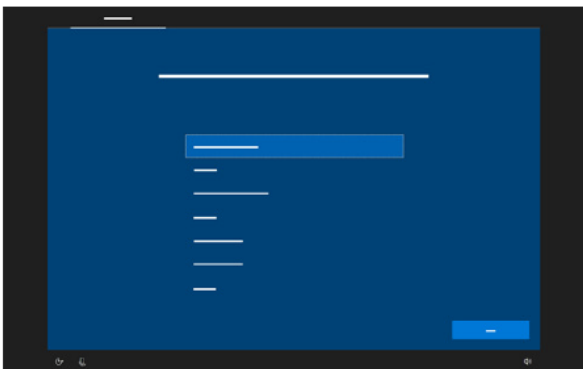

Step 2: Set Up TD Snap

Double tap the icon to launch the TD Snap app. Follow the prompts to create a new user or restore a saved user file.

Step 3: Mount & Position

The device may be positioned using a mounting system, laying it flat on a surface, or propped on the integrated kickstand. Start by situating the user comfortably, then find the device position that gives them clear screen visibility and easy access to their selection method of choice. Always position the device to suit the user, not the other way around. It is expected that the device may need to be repositioned throughout the day. Attach the shoulder strap for secure transport.

Step 4: TD Snap Training Cards

Continue your set up journey with the TD Snap Training Cards that came in the box with your device. The training cards walk you through the features of TD Snap, basic editing, backing up your data, and some activities to help you integrate TD Snap into your daily life.

Additional Resources

Scan the QR codes with your phone or use the links.

Tobii Dynavox tobiidynavox.com



TD Facebook Community qrco.de/TDFB



TD I -110 Device Documentation
qrco.de/I-110manual



Learn about access methods
qrco.de/bbA7US



myTobiiDynavox mytobiidynavox.com



Tobii Dynavox Learning Hub (English only)
learn.tobiidynavox.com



TD Snap Documentation
qrco.de/bcgYP0



North America Technical Support
1-800-344-1778 ext. 1

Documents / Resources

Getting Started Guide
TD I-110



tobiidynavox

[tobii dynavox TD I-110 Display](#) [pdf] User Guide
TD I-110 Display, TD I-110, Display

References

- [tobiidynavox](#)
- [myTobiiDynavox](#)
- [What are Access Methods?](#)
- [Documentation](#)

- [🌐 I-110 Product Support - Tobii Dynavox Global](#)
- [🟢 Tobii Dynavox Global: Assistive technology for communication](#)

[Manuals+](#),

- [home](#)
- [privacy](#)