



## TSI 4090 Certifier Pro Flow Analyzer User Manual

[Home](#) » [tsi](#) » TSI 4090 Certifier Pro Flow Analyzer User Manual 



**Certifier™ Pro Flow Analyzer**  
**Model 4090**  
**Quick Start Guide**





## Contents

- [1 4090 Certifier Pro Flow Analyzer](#)
- [2 Certifier™ Pro Instrument Components](#)
- [3 Certifier™ Pro Accessories](#)
- [4 Instrument Overviews](#)
- [5 Available Measurement Parameters](#)
- [6 Measurement Parameter Definitions](#)
- [7 Pre-Test Checks and Instrument Setup](#)
- [8 Touchscreen Operation](#)
- [9 Documents / Resources](#)
- [10 Related Posts](#)

## 4090 Certifier Pro Flow Analyzer

TSI and TSI logo are registered trademarks of TSI Incorporated in the United States and may be protected under other country's trademark registrations. Certifier is a trademark of TSI Incorporated.



## Certifier™ Pro Instrument Components

### Certifier™ Pro Test Kit (Model 4090)

Material Description	Replacement Part
Power supply 100/240 VAC to +12 VDC 2.1 mm, includes plug set with NA, UK, EU, CN, SAA	130400
Inlet filter, high flow, 22 mm M/F, HEPA	1602341
Inlet filter, low flow, 3/8 inch barb, HEPA	130380
Adapter, high pressure port	1611221
Adapter, 22 mm to 6 mm (for interfacing the Low Flow Filter to the High Flow channel)	1102091
Adapter, 15 mm ID to 22 mm OD	1102093
Adapter, 22 mm F to 22 mm F, Straight	130373
Adapter, 15 mm M to 22 mm F, Pediatric Cuff	130374
Airway pressure fitting with screen	1611330
Pressure tubing, silicone, 1/8 inch ID x 1/4 inch OD x 48 inch	3002053
Oxygen Sensor (installed)	130377
Battery, Lithium-Ion 7.2 V 6.4 Ahr (installed)	130385
Hanging Clip (to hang 4090 from a ventilator rail)	130375





## Certifier™ Pro Accessories

Description	Part Number	Image
Carrying Case, Fitted (19 x 14.75 x 6.5 inch 48 x 37.5 x 16.5 cm)	130378	
Test Lung, Adult, 1 L	130393	
Test Lung, Pediatric, 0.5 L	130396	
Computer Cable, USB- C to USB-A, with Screw Lock	130382	
Stylus, Capacitive Touch	130370	
Flow Resistor Kit, Calibrated Resistors (R p5, Rp20, Rp50)	130395	

## Instrument Overviews



## 4090 Instrument, Front



1. Low Flow Channel Inlet
2. High Flow Channel Inlet
3. High Pressure Port (Max: 150 psi / 10 bar)
4. Low Differential Pressure, Positive Port (+)
5. Low Differential Pressure, Negative Port (-)
6. Touchscreen Display

## 4090 Instrument, Back



- |                             |                                    |
|-----------------------------|------------------------------------|
| 1. Low Flow Channel Outlet  | 7. USB-C Communications Output     |
| 2. High Flow Channel Outlet | 8. Screw hole for USB-C cable lock |
| 3. Power Button (On/Off)    | 9. USB-A Connectors (2)            |
| 4. RS-232 Output            | 10. Portable / Rack Switch         |
| 5. TTL Trigger Input        | 11. Cable Retainer                 |
| 6. DC Power Input           | 12. Model / Serial Number Sticker  |



## 4090 Instrument, Bottom




- |   |   |
|---|---|
| <ul style="list-style-type: none"><li>1. Battery/Oxygen Sensor Cover</li><li>2. Cover Knob</li><li>3. Instrument Handle</li></ul> | <ul style="list-style-type: none"><li>4. Rubber Feet</li><li>5. Fan</li><li>6. Cable Retainer</li></ul> |
|---|---|



## Available Measurement Parameters




Symbol	Description	Symbol	Description
V	Flow rate	PPLAT	Plateau pressure
VPEAK	Peak flow rate	PHIGH	High pressure(4081 only)
VMIN	Minimum flow rate	PARS	Absolute pressure, equals barometric pressure if flow tube open to atmosphere.
VΔ	Flow rate delta	2	Oxygen concentration (4081 with 4073 only)
VΔ%	Flow rate delta percent	T	Temperature
VTI	Inhaled tidal volume	f	Breath frequency
VTE	Exhaled tidal volume	ti	Inspiratory time
V	Real-time volume	tIP	Inspiratory pause time
MV	Inhaled minute tidal volume	ti+P	Inspiratory time including pause time
P	Low pressure measurement (4081 only)	tR	Inspiratory rise time
PIP	Peak inspiratory pressure (4081 only)	tE	Expiratory time
PEEP	Peak end expiratory pressure (4081 only)	I:E	Inspiratory to Expiratory ratio
PMAP	Mean airway pressure (4081 only)	I : EIP	Inspiratory to Expiratory ratio including pause time
PMIN	Minimum low pressure (4081 only)	CSTAT	Static compliance
PΔ	Delta low pressure – Delta airway pressure (4081 only)		Time of day

## Measurement Parameter Definitions

Refer to the Certifier™ Pro User Manual for more detailed definitions of the measurement parameters.



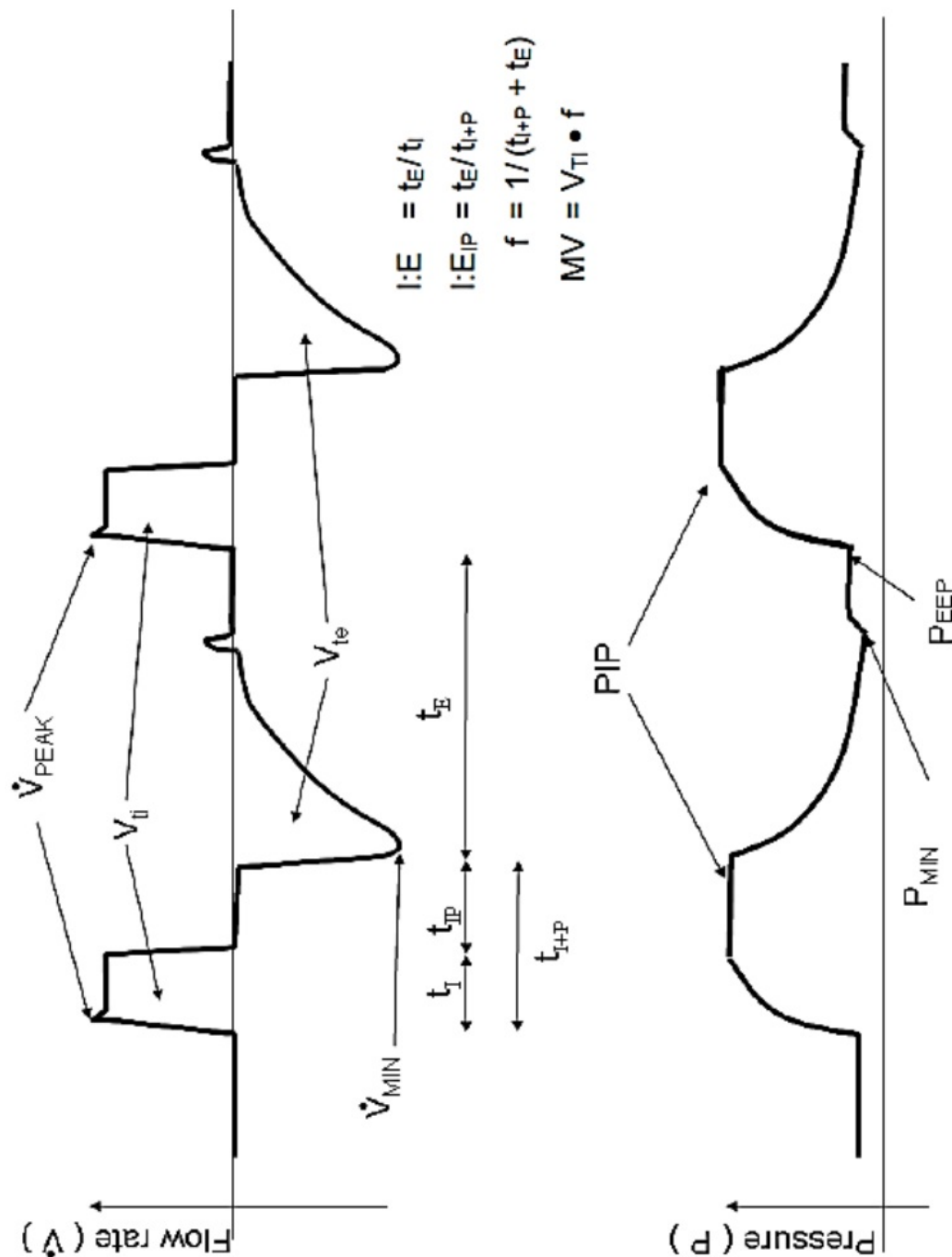

## Pre-Test Checks and Instrument Setup

### Power and Warmup

The Certifier™ Pro Flow Analyzer can be operated through AC power or on battery power. With AC power applied, the battery will automatically recharge. Upon startup, allow 1 minute for the Certifier™ Flow Analyzer's flow sensor and pressure transducers to warm up.

### Instrument Calibration

Confirm that the calibration date for the Certifier™ Pro Flow Analyzer is valid. TSI® recommends an annual calibration to ensure the most accurate flow, pressure, and temperature measurements.

### Zero Low / High Pressure

Disconnect any pressure tubing from the low and high pressure ports on the front side of the instrument to expose the sensors to ambient air. Press the zero icon on the Certifier Pro display to access the Zero Pressure Sensors screen and perform the zeroing.

### Oxygen Sensor Calibration

Calibrate the oxygen sensor daily, following an altitude change, or after replacement. Navigate to the Settings menu and select Oxygen Sensor. Follow the directions on screen to calibrate with air or oxygen.

### Connect Inlet Filter

Connect filter to the breathing circuit upstream of the Certifier Flow Analyzer. TSI® supplies inlet filters for the high




and low flow channels and recommends that filtration be used at all times during operation.

#### **Connect Low Pressure Measurement**

Attach the airway pressure fitting (1) to the breathing circuit. Cut a length of silicon tubing (2), connect one end of the tubing to the barb on the airway pressure fitting, and connect the other end of the tubing to the (+) low pressure port located at the front of the instrument.

#### **Test Circuit for Bi-directional Flow**



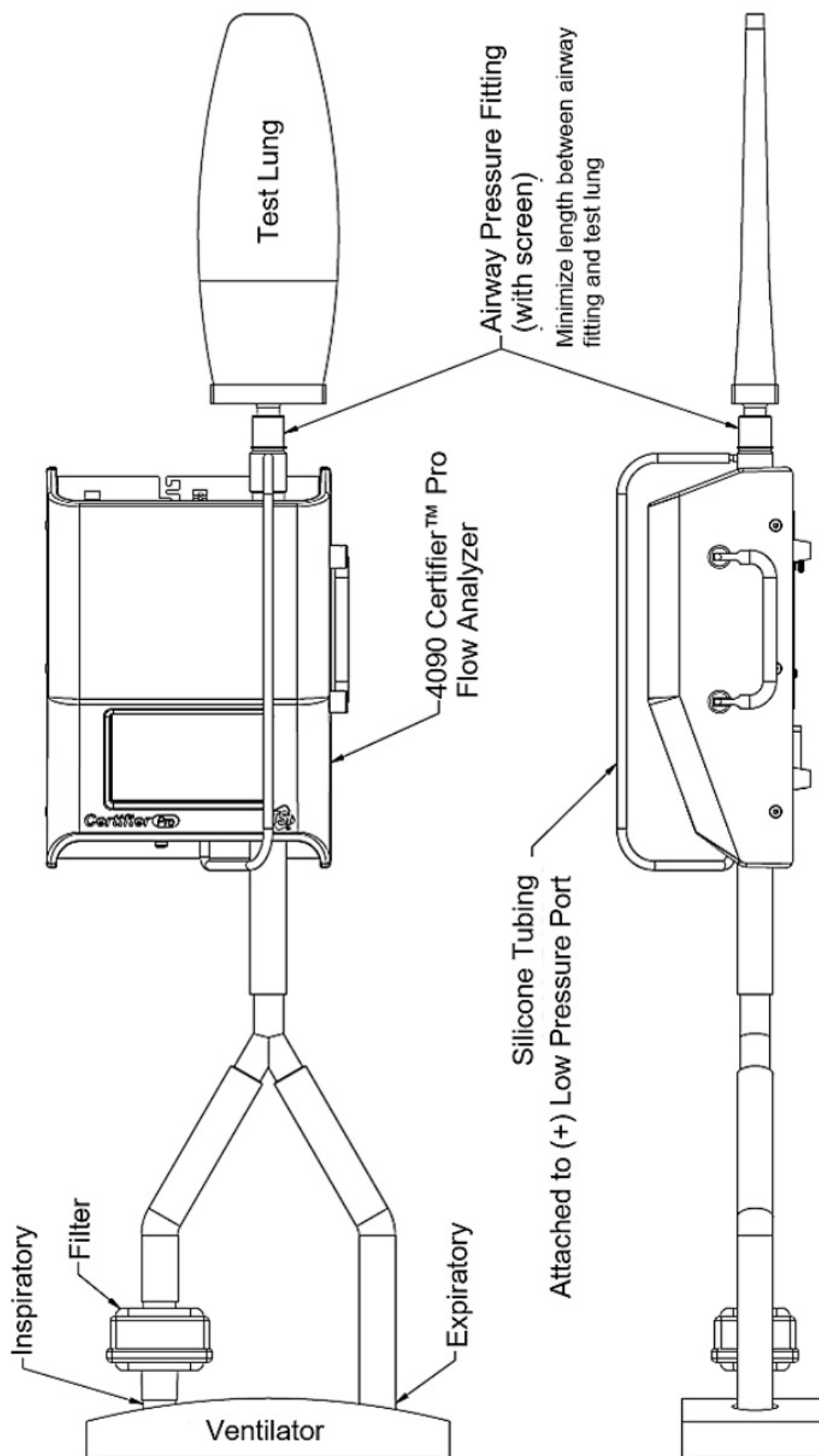


## Touchscreen Operation

The Certifier™ Pro instrument utilizes a 5-inch capacitive touchscreen display. The display interface is designed to be operated using your fingers, press lightly on the display for optimum touchscreen operation. A capacitive-touch stylus can be used as an alternative to operating with finger presses.



Operate the Certifier™ Pro Flow Analyzer by touching any on-screen element that you would like to change. Up to 18 measurement parameters can be displayed on screen at one time, and the display will automatically adjust the font size and positioning based on the number of selected measurements.


### Parameter Screen

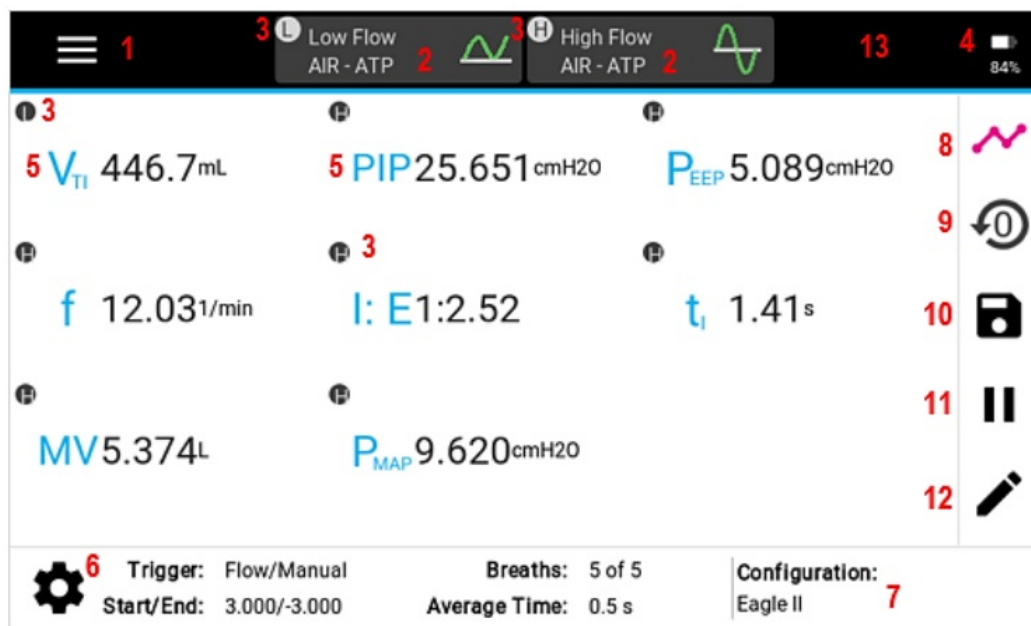

1. Menu
2. Module Cards
3. Module Indicators
4. Battery Indicator
5. Measurement and Units
6. Triggers / Averaging Settings
7. User Configurations

8. Parameter/Graph Screen Toggle
9. Zero Pressure Sensors
10. Save Data and/or Snapshots
11. Pause/Play Display
12. Edit Measurements and/or Units
13. Warnings & Errors Notifications

### Graph Screen

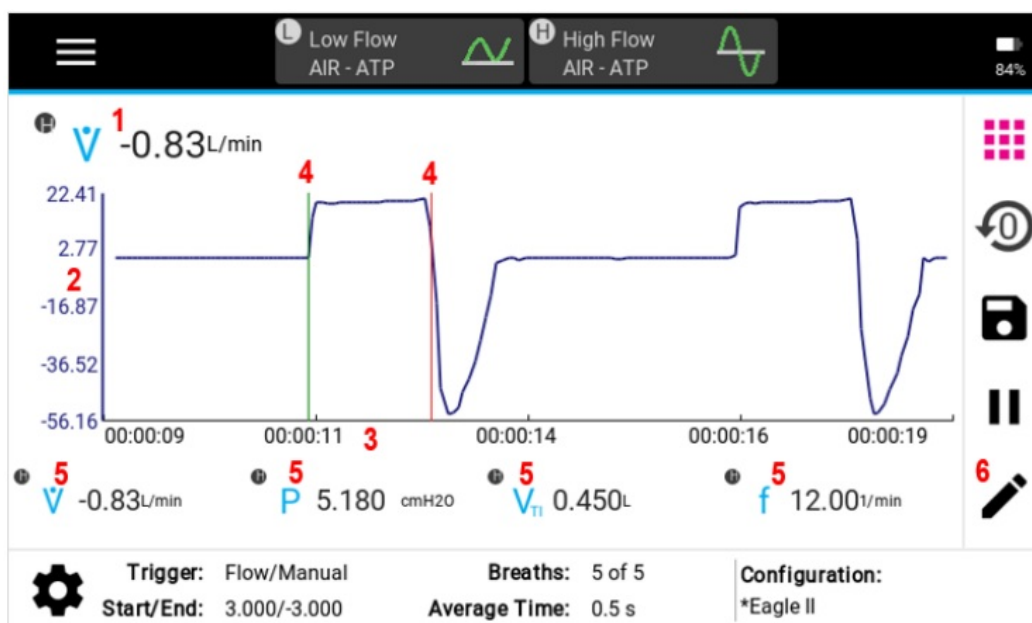



1. Plotted Measurements 2. Y-Axis (primary and secondary) 3. X-Axis	4. Breath Trigger Indicators 5. Real-Time Measurements 6. Edit Graph
---	--

## Breath Triggers

Triggers are used to detect the start of the inspiratory and expiratory breath cycles. Triggers can be based on the flow rate, pressure, or driven from a TTL voltage signal. The Certifier™ Pro Flow Analyzer can automatically detect the breath using the flow rate, or you can manually set your own start and end trigger values using the flow rate or low pressure. In most circumstances, it is recommended to use the Auto Trigger setting.

Triggers

Trigger Type

☒ Auto Trigger

☒ Flow Rate
☐ Pressure (Advanced Users)
☐ Flow Direction (TTL)

Trigger Type

☒ Flow Rate
☐ Pressure (Advanced Users)
☐ Flow Direction (TTL)

☐ Auto Trigger

Start

1

L/min ▾

End

-1

L/min ▾

## Breath Averaging

The number of breaths specifies the maximum number of breaths over which to average breath parameter measurements. The averaging time sets the rate at which real-time values are averaged and updated on the Certifier™ Pro display. Real-time values include measurements for flow rate, low pressure, high pressure, absolute pressure, oxygen concentration (if connected), and temperature.

Trigger: Flow/Auto

Breaths: 5 of 5

Average Time: 0.5 s

## Certifier™ Flow Analyzer Configurations

A Certifier™ Flow Analyzer configuration is a saved collection of settings for the measurements, units, gas type and conditions, triggering, and graphing that you have selected. Any Certifier™ setup can be saved as a configuration and later recalled. Configurations can be saved for testing various types of equipment or to optimize test setups to conform with organizational standards or personal preference.

Configurations are stored in the internal memory of the 4090 Certifier Pro instrument and can be exported to a USB mass storage drive via either USB-A port. Exported configurations can be sent as email attachments and they can be imported onto different Certifier Pro or Certifier Plus instruments.

## Data Acquisition

The Certifier Pro Flow Analyzer is capable of logging measurement data and capturing screenshots. Data can be saved from either the Parameter or Graph screen, and files are saved to the internal memory of the Certifier Pro Flow Analyzer.

Snapshots save a screenshot image as well as the data currently displayed on screen. Continuous logging captures data for all measurements selected on screen at sample rate of 1 sec to 10 mins and test length from 15 secs to up to 5 days. Waveform logging records raw sensor data at a rate of 1000 Hz for a length of 15 to 60 seconds.



## Data Export

Saved screenshots and data log files can be exported from either of the two USB-A ports located on the back of the 4090 Certifier Pro Flow Analyzer. Snapshots are exported as .png image files and data is exported as .csv files. The delimiter for data export can be set as comma, pipe, semicolon, or tab delimited. You can select the Export Delimiter in the General Settings page of the Settings menu option.

TSI Incorporated – Visit our website [www.tsi.com](http://www.tsi.com) for more information.

USA Tel: +1 800 680 1220

UK Tel: +44 149 4 459200

France Tel: +33 1 41 19 21 99

Germany Tel: +49 241 523030

India Tel: +91 80 67877200

China Tel: +86 10 8219 7688

Singapore Tel: +65 6595 638 8




©2023 TSI Incorporated

P/N 6016201 Rev. B

April 2023

## Documents / Resources

 <p>Certifier<sup>™</sup> Pro Flow Analyzer Model 4090</p> <p>Quick Start Guide</p> <p>4090, 4090 Certifier Pro Flow Analyzer, Certifier Pro Flow Analyzer, Flow Analyzer</p>	<p><a href="#">TSI 4090 Certifier Pro Flow Analyzer</a> [pdf] User Manual</p> <p>4090, 4090 Certifier Pro Flow Analyzer, Certifier Pro Flow Analyzer, Flow Analyzer</p>
--	---