



A1 Light Curing Unit User Manual

[Home](#) » [Support](#) » A1 Light Curing Unit User Manual 

A1 Light Curing Unit



Contents

- 1 Preface
- 2 Main parts and accessories
- 3 Figure instruction
- 4 Shipping and storage
- 5 Usages
- 6 Maintenance
- 7 Terms of Guarantee
- 8 Operation
- 9 Trouble shooting
- 10 Customer Support
- 11 Documents / Resources
 - 11.1 References

Preface

Thank you for purchasing A1 Light curing unit. Before using this equipment, please read this USER MANUAL carefully and follow the maintenance to enable the function to be fully implemented. Please put this USER MANUAL in a proper place for the future reading. We reserve the right to change the design, the technique, accessories, the instruction manual and the contents of the original packing list of the equipment at any time without notice.

Main parts and accessories

A set complete A 1 Light curing unit. consists the following parts:

Main unit	1 set	Power cord	1 pcs
Instruction manual	1 pcs	Guarantee	1 pcs

The main technical specifications.

POWER SOURCE: Ac 230V | Ghz

LAMP: 27W/0.610A/460lm **LAMP LIFE:** 5,000 Hours

POWER CONSUMPTION: <30W **WEIGHT:** 2.1 KGS

UNIT'S OVERALL SIZE: 280X 135 X 125 mm (D x W x H)

FUSE: 250V

Figure instruction



Notice! Refer to files



Earth Protection

Shipping and storage

A1 Light curing unit should be shipped and stored under the following environment:

Environment temperature: -40°C~70°C;
Air pressure: 500 Hpa~1060 Hpa

Usages

- Making individual tray
- Making guide plate for implant

Note

1. Please use the power source with ground wire.
2. Never install at a place near the inflammable substance.
3. In order to protect the environment, please cope with the disused machine.


Maintenance

Do not clean the machine by organic solvents.

Terms of Guarantee

Holy Medical grants the user a guarantee for its complete product range , covering any faulty functioning and defects in material or manufacture, for a period of 12 months from the date of the invoice.

Operation

1. Plug the power cord into an appropriate AC outlet with earth leakage circuit breaker. (230 Volt 50/60Hz 100 Volt 50/60Hz).
2. Depress the power switch on the backside to energize the unit.
3. The display shows a 3-minutes pre-set curing cycle. The user can adjust the curing time from 0/1 minute (6 seconds) to 99 minutes according to the required polymerization for various composites and resin- based materials.
 - Press the “▲” button to increase the curing time up to maximum 99 minutes.
 - Press the “▼” button to reduce the curing time to a minimum of 6 seconds(“0.1” shown on display). E *The pre-set curing time can be adjusted and memorized before opening the door of the hardening chamber of before turning off the unit.
4. Shape the individual tray material by using a mould and cut off the excess material. Place the whole in the hardening chamber and press the START/STOP button to start the curing cycle. The built-in safety switch ensures that the light will be turned off if the door to the hardening chamber is opened during a curing cycle, “” will light up on the display.
5. The hardened tray can be taken out of the hardening chamber when the curing time has elapsed. After examining the tray, one can decide to add some material or to proceed finishing off the edges of the tray.
6. The programmed curing time is kept in the memory of the unit. Simply press the START/STOP button again to repeat the curing cycle.

Trouble shooting

In case any malfunction or problems happens, the following solutions could be adopted. If the problems can not be solved, please contact the local dealer and send for repairing.

Problem	Causes	Solutions
There is no power of tube after presses the button.	The tube is not completely connected with holder..	Inspect and connect the tube.
	The tube is ageing.	Change new tube.


Customer Support

Alandental inc

www.Alandental.com

Fax: 023-68438829

Documents / Resources

	<p>A1 A1 Light Curing Unit [pdf] User Manual A1 Light Curing Unit, A1, Light Curing Unit, Curing Unit</p>
---	---

References

- [Dental Instruments Australia |Dental Equipment Supplies For Sale - Alandental.com](#)
- [User Manual](#)

[Manuals+](#), [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.