




unitron Remote Plus app User Guide

[Home](#) » [Unitron](#) » unitron Remote Plus app User Guide 


unitron Remote Plus app User Guide

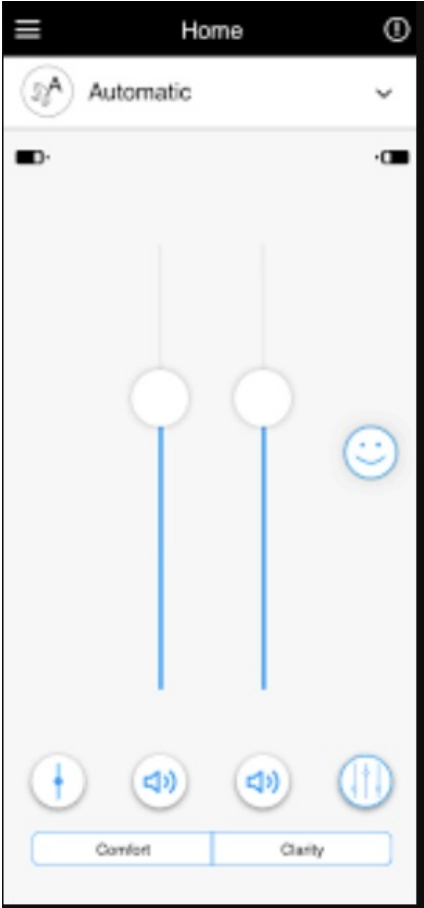

Contents

- 1 App overview
- 2 Privacy notice
- 3 Pairing with hearing aid(s)
- 4 Main screen
- 5 Enabling presets
- 6 Changing programs on the hearing aid(s)
- 7 Advanced features settings
- 8 Ratings
- 9 Settings menu
- 10 Optional Programs
- 11 Remote adjust
- 12 Compliance Information
- 13 Information and explanation of symbols
- 14 Documents / Resources
 - 14.1 References
- 15 Related Posts

App overview

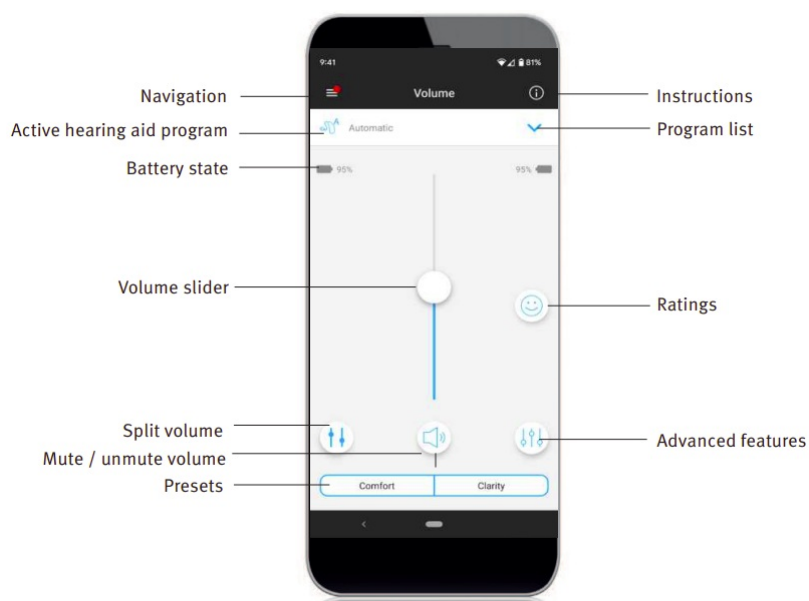

Privacy notice

Accepting the app privacy notice

To use the Unitron Remote Plus app, you need to accept the privacy notice and the anonymous data analysis of the usage from the app.

Privacy Notice

Last updated: June 15, 2020

THIS NOTICE DESCRIBES HOW PERSONAL INFORMATION ABOUT YOU MAY BE USED AND DISCLOSED, HOW YOU CAN GET ACCESS TO THIS INFORMATION AND WHICH RIGHTS YOU HAVE. PLEASE REVIEW IT CAREFULLY.

This Privacy Notice describes the following:

- The personal data that Sonova collects from and about you through the Unitron Remote Plus application ("the Mobile Application" or "Mobile App") or Unitron TrueFit fitting software together with all related technology to access or otherwise use the Mobile Application ("Mobile Service") as described below.
- How do we use and protect your personal data?
- The choices you have about how we use your personal data.
- How you can exercise your rights (e.g. right to access, right to rectification, right to complain, etc.).

This privacy notice is in line with the EU General Data Protection Regulation (GDPR), the Swiss Federal Data Protection Act (FDPA), the United States Health Insurance Portability and Accountability Act (as amended from time to time) ("HIPAA") and other applicable data protection laws that may apply to you. In the United States, any reference to HIPAA under this

[Accept](#)

Activating Insights

To opt in for Insights features including remote adjust, tap the "Activate" button. To skip this step, tap the "Later" button.

**You're unique.
So is your hearing.**

Activate Insights to help your hearing health care professional get you the best fit.

- ☒ Rate your hearing experiences
- ☒ Track your wearing time
- ☒ Track your listening environments
- ☒ Enable remote hearing aid adjustments

By activating insights you consent to store your personal information in the cloud for your hearing health care professional to access.


[Show detailed Privacy Notice](#)

[Activate](#)

[Later](#)


Pairing with hearing aid(s)

Detect your hearing aid(s)

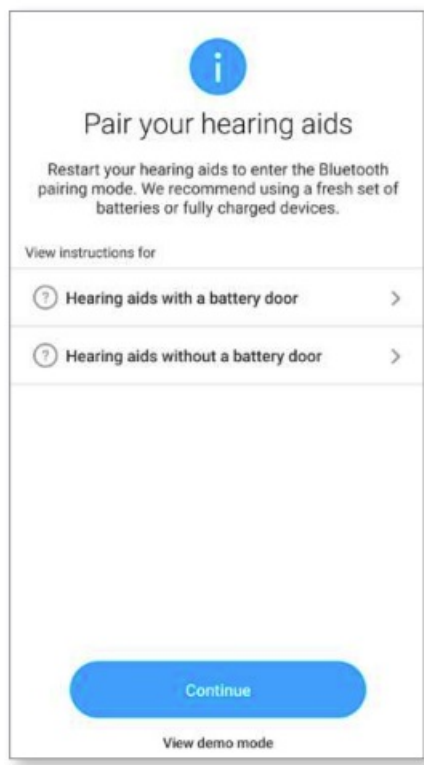

1. If your hearing aid(s) have a battery door, restart your hearing aids by opening and closing the battery door. If your hearing aid(s) does/do not have a battery door, first turn off each hearing aid by pressing the lower part of the button until the LED turns red (4sec). Then turn on each hearing aid by pressing the same button until the LED turns green (2 sec).
2. You can always select the “demo” mode to try the app without connecting a Unitron hearing aid and get a first impression of the functionalities. In this mode no remote control functionality is available for your hearing aids.

Select your hearing aid(s)

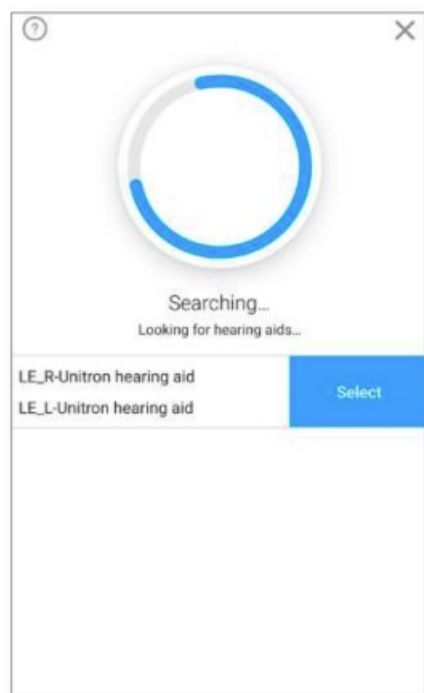

1. If more than one set of devices have been detected by the app, press the button on your hearing aid and the

corresponding device will be highlighted in the app


Main screen

Adjust hearing aid volume

Move the slider up or down to increase or decrease the hearing aid volume on both sides. Press the () “mute” button below the slider to mute or unmute the hearing aid

increase or decrease
des.
move the slider to

Volume slider
Mute / unmute volume

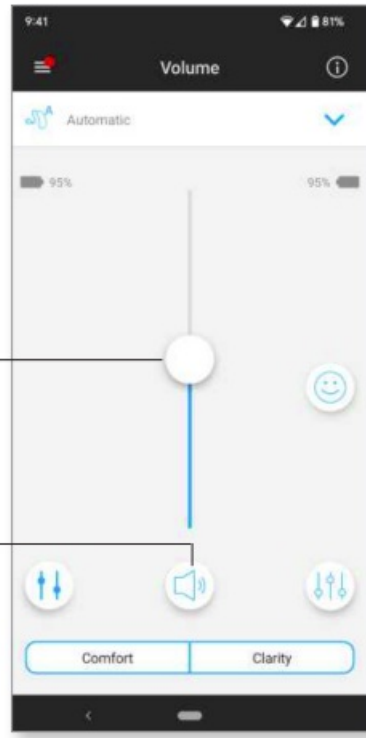

Split the volume

Press the () “split volume” button to control the volume on each hearing aid separately. Use the volume slider to change the volume. Press the () “join volume” button to merge the volume sliders

Split volume



Volume slider

Join volume




Note: For the “split volume” button to be visible “Side selection” must be enabled in Settings > App settings.

Enabling presets

Comfort and Clarity

For the Automatic Program, “Clarity” is available to enhance speech, whereas “Comfort” is used to reduce noise to improve overall listening comfort.

Clarity and Comfort are mutually exclusive, and cannot both be in the ‘On’ state at the same time.

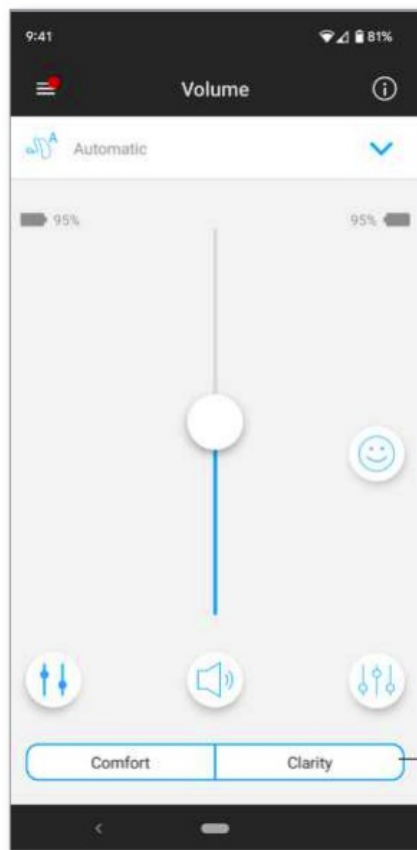


Presets Comfort/Clarity

Changing programs on the hearing aid(s)

Select another program

Tap the arrow beside the current program name to see all available programs. Select the desired program (e.g TV Connector)

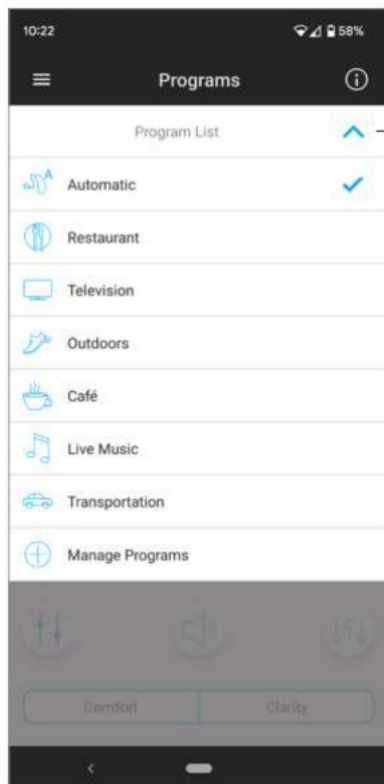


Dropdown for program list

Advanced features settings

Further adjustments are available depending on the program currently selected, your hearing aid configuration, and connected audio sources (e.g. TV Connector). Tap the () advanced features button at the bottom-right corner to access these options:



Advanced features

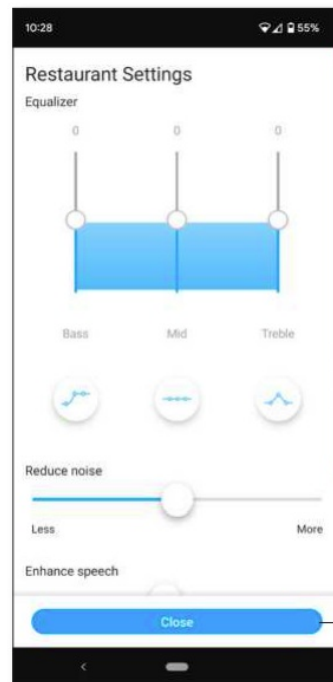

Close

Equalizer

You can adjust the advanced features Equalizer settings.

Balance

If you use an external streaming device, (e.g. TV Connector, music) you can adjust the focus to hear more of the

streamed signal or alternatively more of the surrounding environment.

Tinnitus Masker

If you have tinnitus, and have been instructed by your hearing care professional on how to use the Tinnitus Masker, you can adjust the volume of the masking noise.

Reduce Noise

The “Reduce Noise” control allows you to increase or reduce the level of noise to the desired comfort level.

Enhance Speech

The “Enhance Speech” control allows you to increase or reduce the focus on speech to the desired comfort level.

Focus Mic

You can adjust the “Focus Mic” control to focus more on sounds from the front or listen all around you.

Ratings

If you opted in for the Insights feature, you will see a happy face icon (😊) on the right side of the main screen. Tap on it to send feedback to your clinician.


Rate your experience

To access ratings, click on the Ratings “smiley” icon.




Ratings icon


1. Choose from either satisfied or unsatisfied.

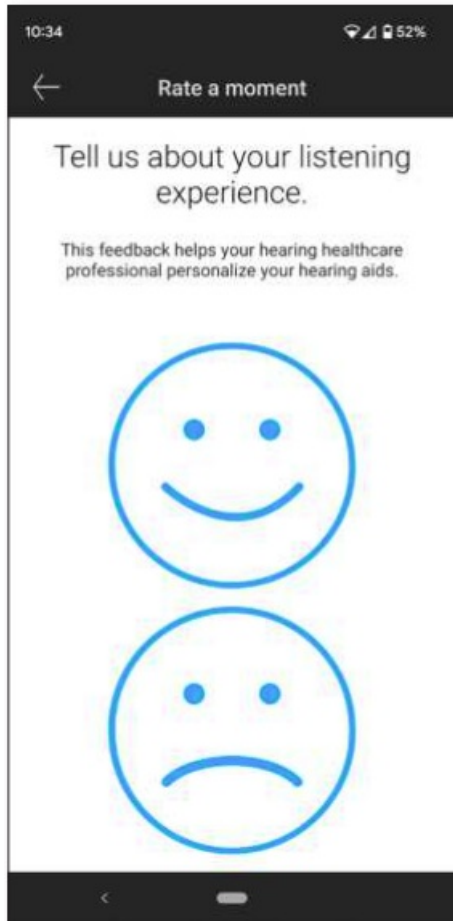

2. Select the environment you are currently in.

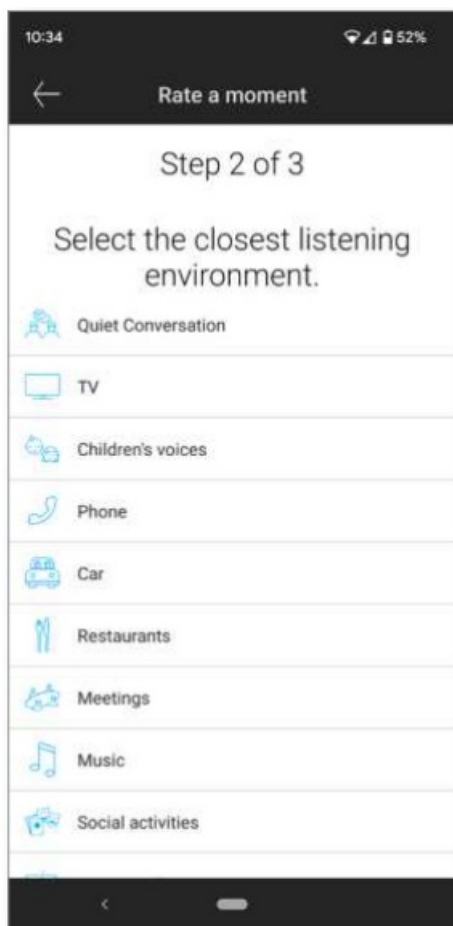

3. See a summary of your feedback and provide more comments (optional). Tap the “Submit” button to submit your feedback to your hearing care professional.

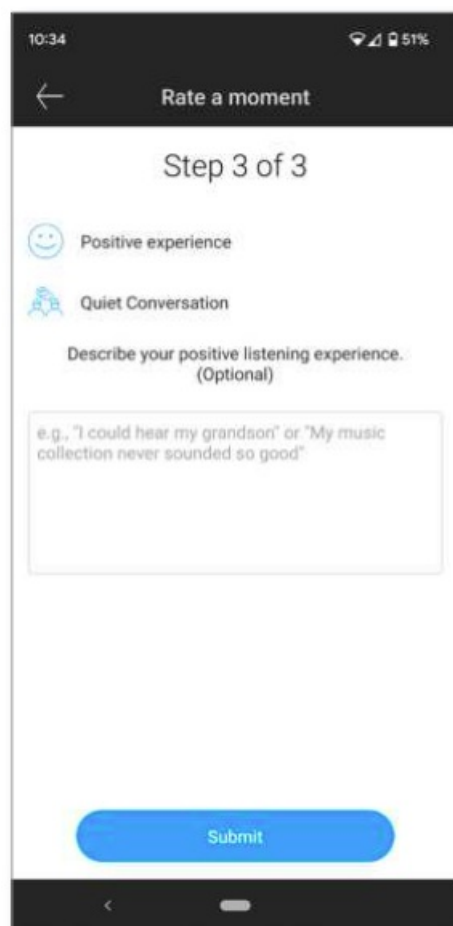

Settings menu


The app is available in different languages. It will automatically match the language of the phone's operating system. If the phone's language is not supported, the default language is English.

1. Tap the icon on the main screen to access the settings menu

Settings menu



2. Select "App settings" to access application settings.
3. Select "My Hearing Aids" to access hearing aid specific settings.
4. Select "Insights" to view the Insights privacy policy, feature information including remote adjust notifications from your hearing care professional or to opt out of this feature.
5. Select "Works with Unitron" to open the Works with Unitron feature.
6. Select "Videos" to watch how-to videos.
7. Select "FAQs" to view frequently asked questions about the app and the hearing aids in the phone's web browser.

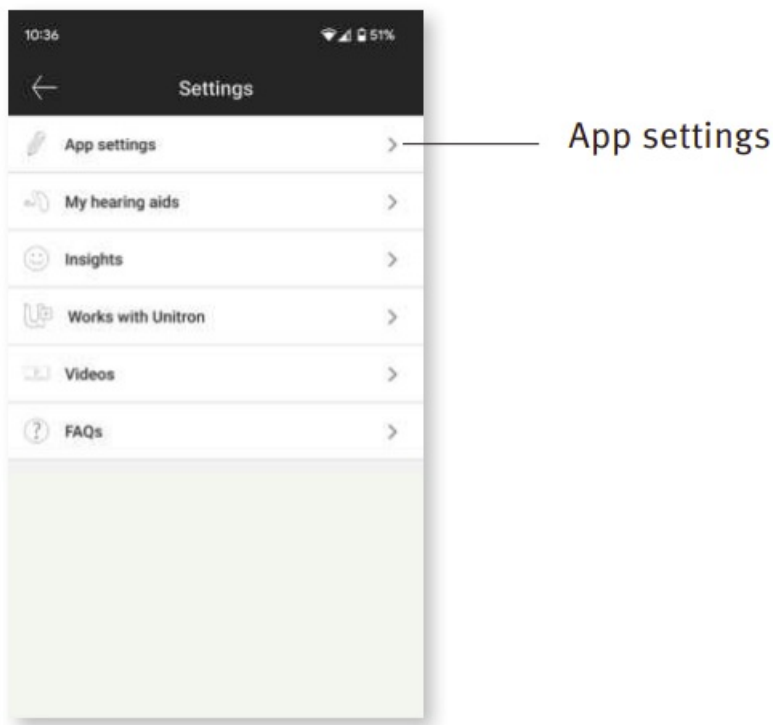

Tap control

If your hearing aids have tap control, you can customize how your hearing aids respond to your double taps. There is a built in sensor on some hearing aids, which enables the control of some hearing aid functions through tap control. The action and sensitivity of tap control can be customized as follows:

Streaming:

- Accept/end call or streaming – enable/disable the ability to accept/end calls or connect to a streaming device (e.g. TV Connector) using a double tap. You can only use tap control to connect to a streaming device if your hearing care professional has configured your hearing aids for manual connection.

Custom actions (configured separately for the left and right hearing aids):


- Pause/resume media – double tap will pause/resume media while streaming.
- Voice assistant – double tap will activate the voice assistant on your smartphone.
- Off – double tap will not perform an action

Tap sensitivity (configured separately for the left and right hearing aids):

- Gentle – most sensitive.
- Normal – default sensitivity.
- Firm – least sensitive.


To configure Tap Control:

1. . From within the app Settings menu select “My Hearing aids

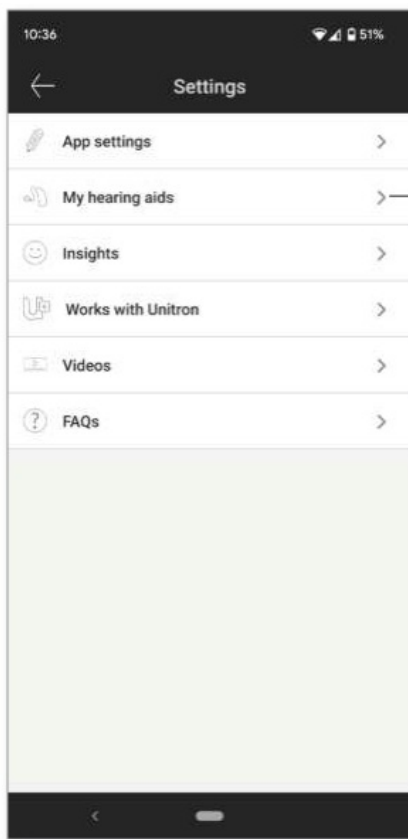


My hearing aids

2. Select "Tap control"

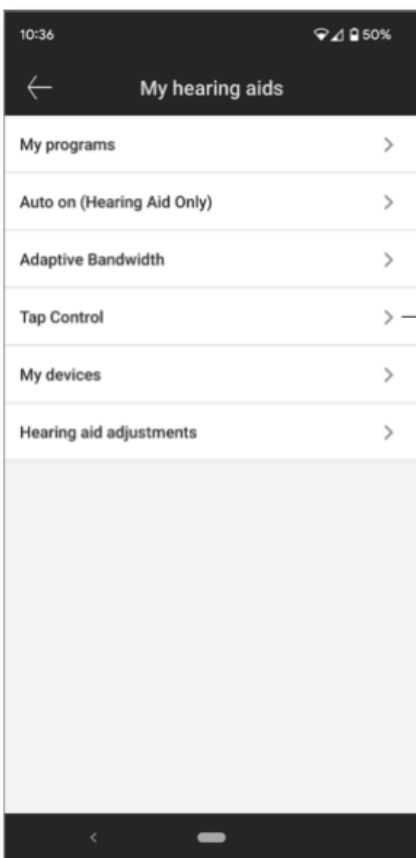


Tap control

3. Configure the action and sensitivity of tap control as desired.

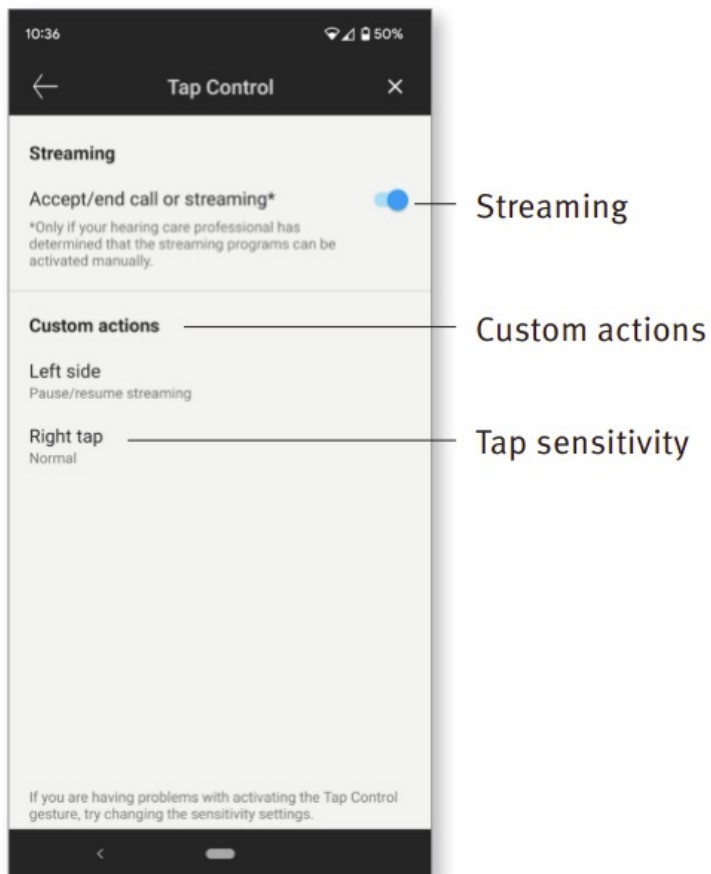

4. Once settings are configured, click on the back arrow to return to the “My hearing aids” screen or the ‘x’ to return to the main screen

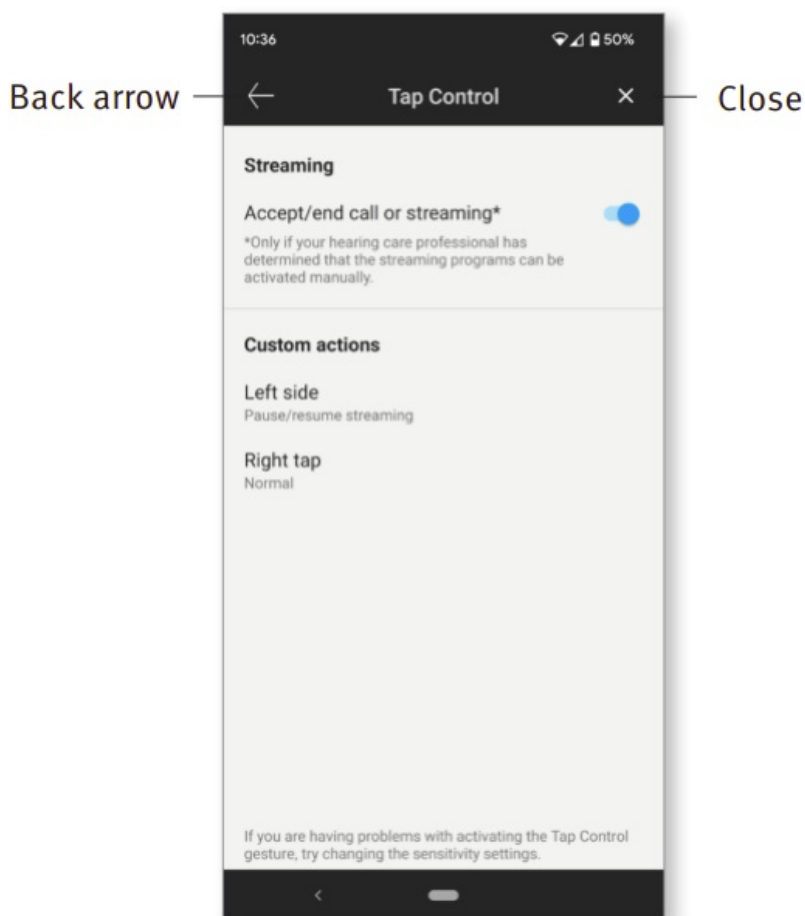


Optional Programs

Choose from a list of predefined programs so the hearing aids can be personalized for a specific situation. The core functionality is defined by the hearing aids and the app allows the user to select from 8 optional programs. The user can enable or disable the optional programs from within the app.


List of optional programs:

- Restaurant
- Television
- Transportation
- Café
- Outdoors
- Live Music
- 360 car
- Party


1. Click on the drop down to view the Program List. Choose Manage Programs to view the optional programs.

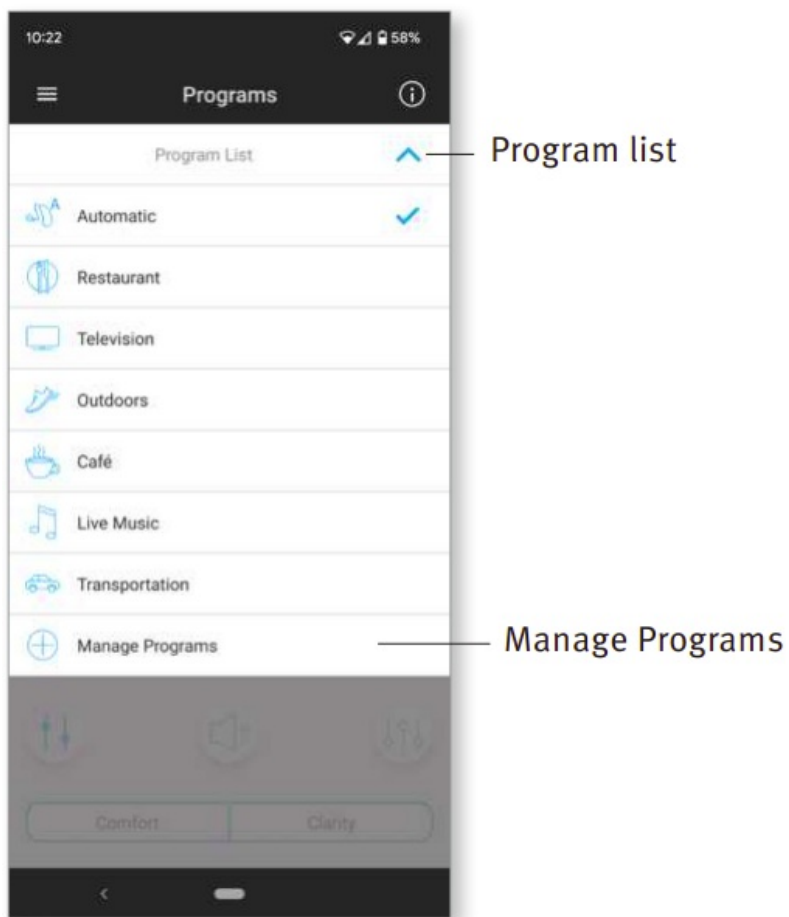


2. List of optional programs available is displayed. Click on the back arrow to go back to the program list.

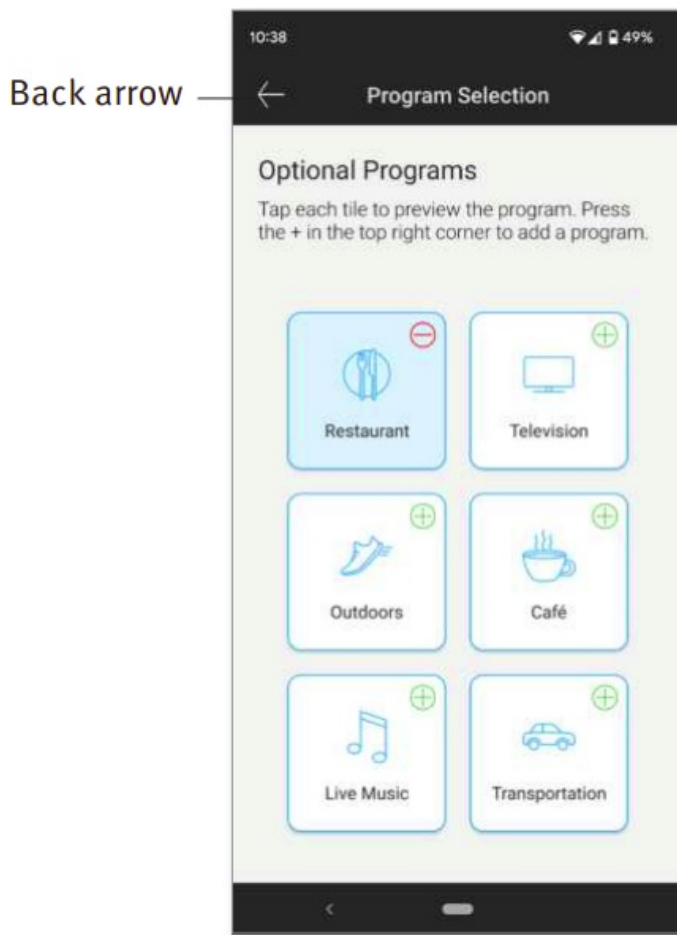

3. To quickly add an optional program click on the () green plus sign

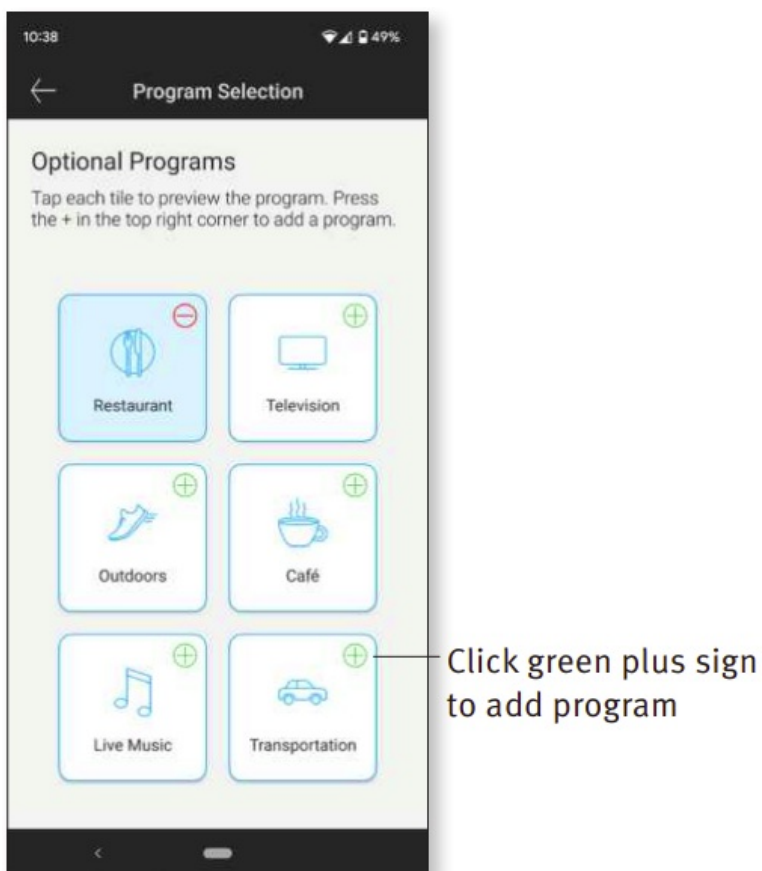


4. A message indicating the optional program has been added will be displayed. Click on the () red minus sign to remove the optional program from the program list

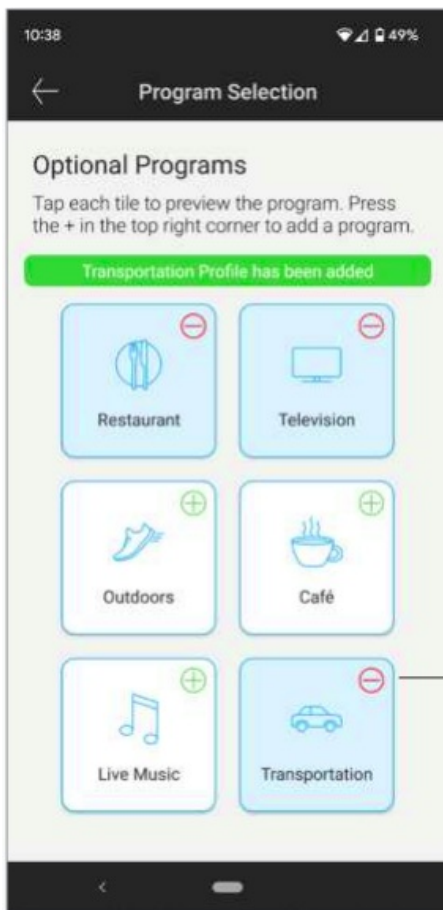


Click red minus sign to remove program

5. Click on the optional program tile to preview the program

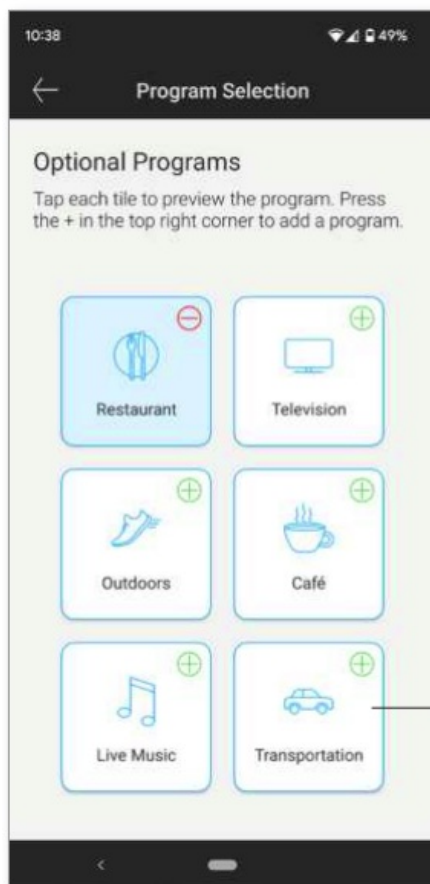

Click to preview


6. The program preview screen will be displayed. Change settings and click on 'Save' to add the optional program to the program list

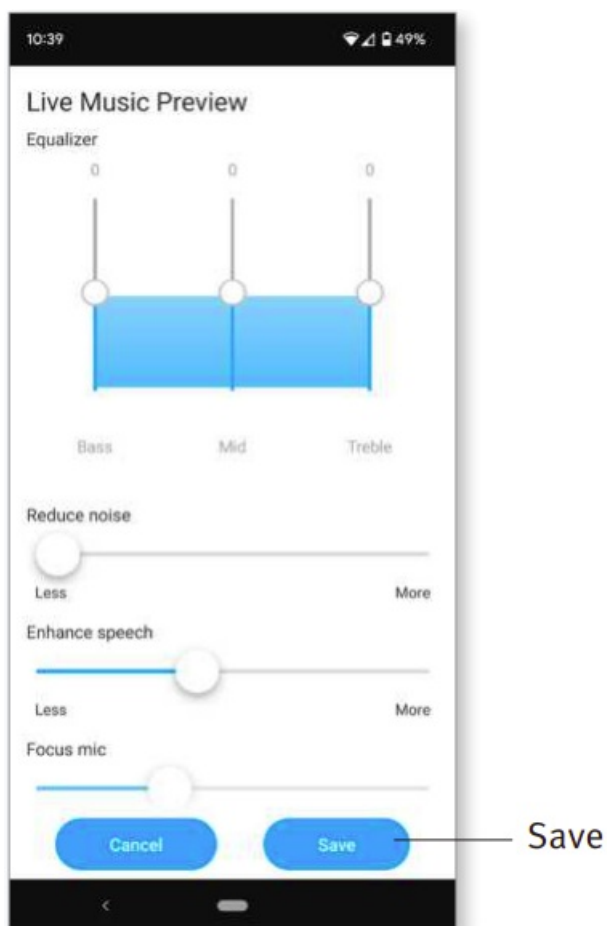


Editing name of a program

Remote Plus app allows the user to change the name of programs so you can personalize what each program means to you. You can change the program name for any program, including the optional programs. To change the program name:


1. Tap on the Settings menu, then select "My hearing aids"

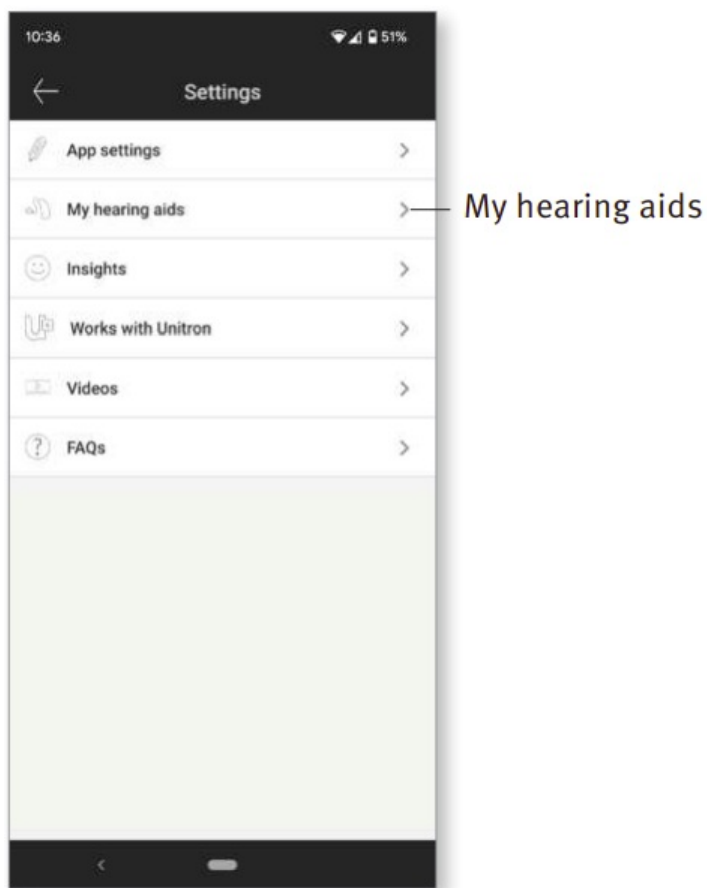


2. My hearing aids screen is displayed. Tap on “My programs”

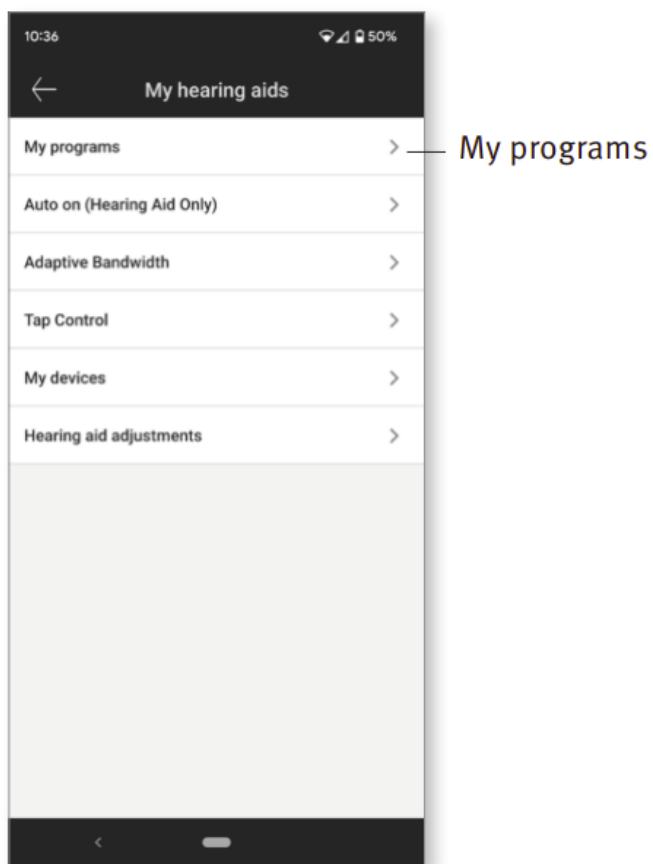

3. List of “My programs” is displayed. Tap on the desired program (e.g. Automatic

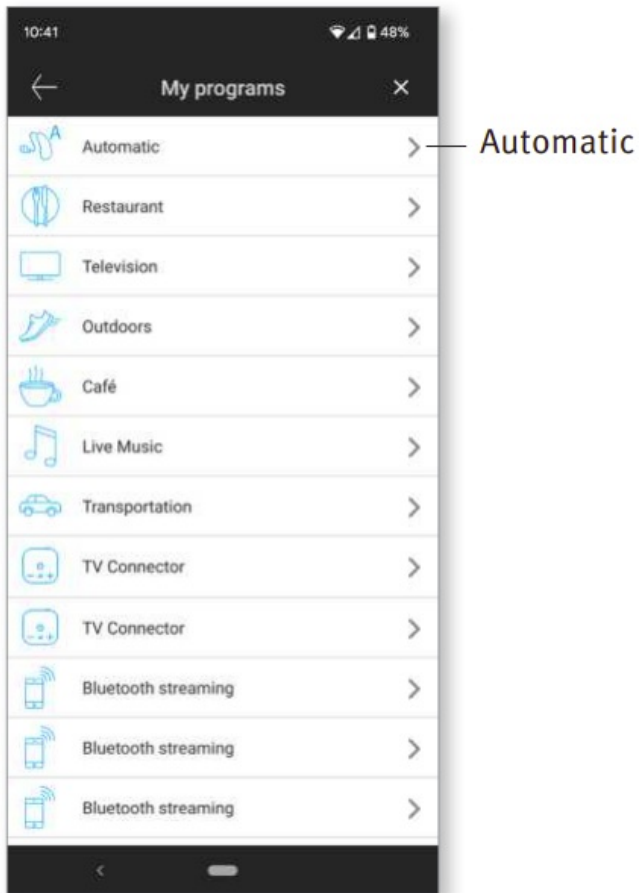

4. Tap the edit/pencil icon and change the “Display name”. This will change the name in the “Program list” drop down and the “Optional program” selection screen

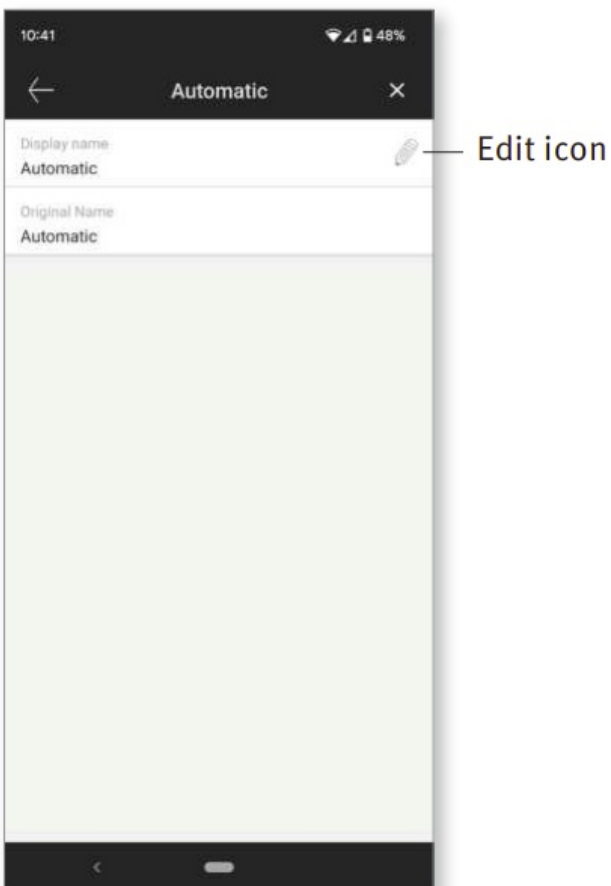


Remote adjust


If you opted in for the Insights feature, you will be able to receive push notifications that contain adjustments to your hearing aids sent by your hearing care professional.

Apply a remote adjustment

1. . Receive a personalized message from your hearing care professional.

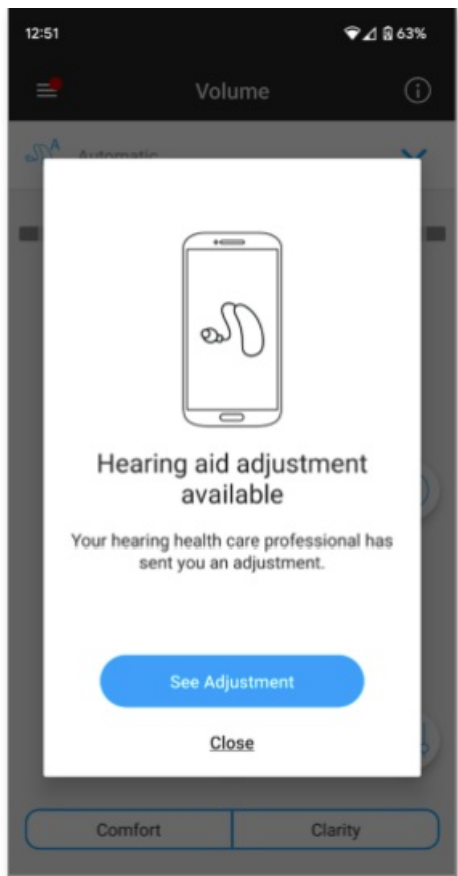


2. Click on the notification to access the adjustment. Or open the Remote Plus app and go to Settings > My hearing aids > Hearing aid adjustments

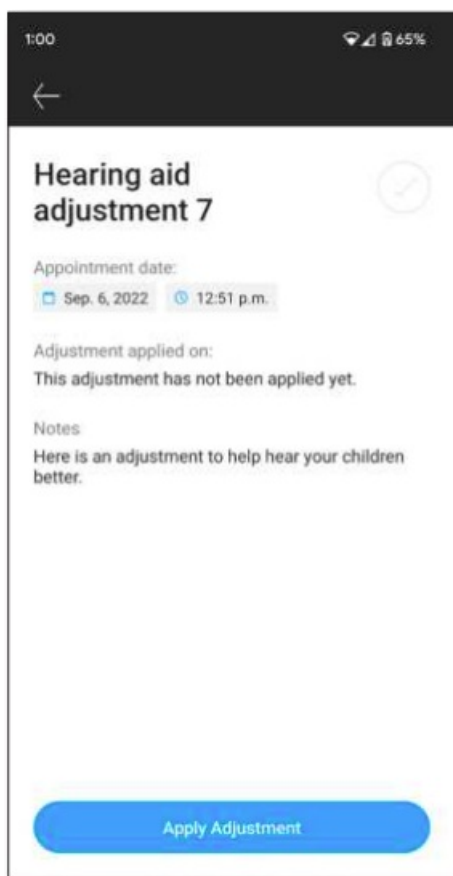

3. Select the adjustment and apply changes



4. If you preferred another setting, you can select any previous message and apply them to your hearing aids.

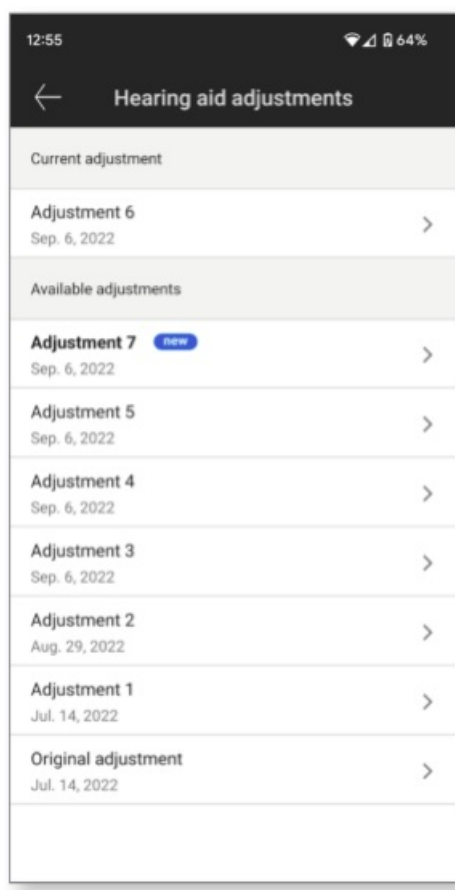

Compliance Information

Declaration of conformity

Hereby Son ova AG declares that this Unitrin product is in compliance with the essential requirements of the Medical Devices Directive 93/42/EEC. The full text of the Declaration of Conformity can be obtained from the manufacturer or the local Unitrin representative whose address can be taken from the list on [http://www.unitron.com \(worldwide locations\).](http://www.unitron.com (worldwide locations).)

If the hearing aids do not respond to the device because of an unusual field disturbance, move away from the disturbing field.

Instructions are available at: unitron.com/appguide

in Adobe® Acrobat® PDF format. To view them, you must have Adobe Acrobat Reader installed. Visit Adobe.com to download.

To obtain a free paper copy of the instructions, please contact your local Uniron representative. A copy will be sent within 7 days.

Information and explanation of symbols



With the CE symbol, Son ova AG confirms that this Unitrin product – including accessories – meets the requirements of the Medical Devices Directive 93/42/EEC.

The numbers after the CE symbol correspond to the code of certified institutions that were consulted under the above-mentioned directives.



This symbol indicates that it is important for the user to read and take into account the relevant

information in this user guide.



This symbol indicates that it is important for the user to pay attention to the relevant warning notices in this user guide



Important information for handling and effective use of the produce



Copyright symbol



This symbol shall be accompanied by the name and the address of the manufacturer (who are placing this device on the market).



Indicates the Authorized representative in the European Community. The EC REP is also the importer to the European Union.



The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Unitrin is under license. Other trademarks and trade names are those of their respective owners



Son ova AG
Laubisrütistrasse 28
CH-8712 Staffa, Switzerland




And importer for European Union:
Sonoma Deutschland GmbH
Max-Edyth-Str. 20
70736 Fellbach-Oeffingen, Germany




Australian Sponsor:
Son ova Australia Pty Ltd
12 Inglewood Place, Norwest
NSW 2153
Australia

unitron™

Documents / Resources

<div><div>Unitron Remote Plus app</div><div>User Guide</div><div><div>unitron .us.</div><div></div></div></div>	<div><div>unitron Remote Plus app [pdf] User Guide</div><div>Remote Plus app, Plus app, app</div></div>
--	---

References

-  [Adobe: Creative, marketing and document management solutions](#)
-  [Home](#)
-  [Home](#)
- [User Manual](#)