



Unitron TrueFit Fitting Software Installation Guide

[Home](#) » [Unitron](#) » Unitron TrueFit Fitting Software Installation Guide 

unitron™ Love the experience TrueFit Fitting Software Installation Guide

Contents



- [1 Installing Unitron TrueFit Software](#)
- [2 System Requirements](#)
- [3 Documents / Resources](#)
 - [3.1 References](#)

Installing Unitron TrueFit Software

1. Insert the Unitron TrueFit disk/USB or download and extract the installation file.
 - If the installation program does not start automatically, double click on setup.exe from the fitting software disk/USB or in the downloaded folder.
 - Follow the displayed step by step instructions. InstallShield installs the fitting software on your PC.
2. Select the preferred installation language: English, French, German or Dutch (for the installation procedure only) and click OK.
3. If additional Microsoft software components are required, they will be listed. Click Install to install them. If you are prompted to reboot, please follow the step-by-step instructions.
4. When the InstallShield Wizard appears, click Next >.
5. Read the license agreement and accept it to continue the installation process. Click Next >.
6. When the User Settings dialog appears, follow the instructions on the screen to select your country, your language and the installation type. Click Next >.
 - If NOAH 4 is installed, you can choose to install Unitron TrueFit under NOAH or Standalone, otherwise Standalone will be pre-selected.
7. Choose Next > on the Custom Setup screen to install all features in the default location, or follow the instruction

to make changes.

8. Choose Install to begin the installation.
9. When prompted, select Finish to complete the installation.

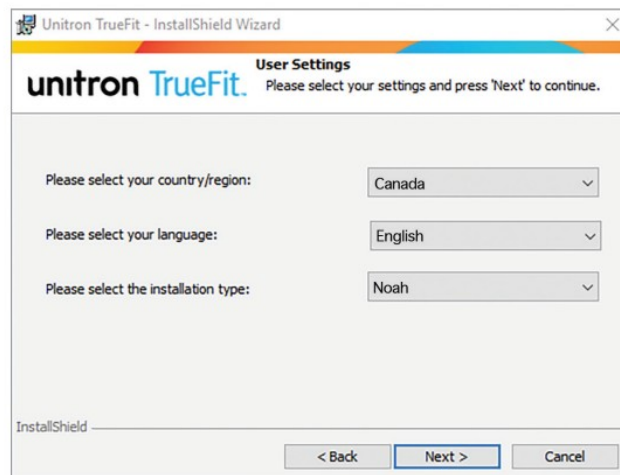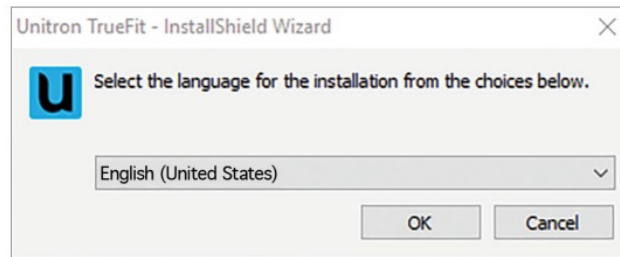

System Requirements

Operating systems supported	Windows n, Home / Pro / Enterprise / Education) Windows io, Home / Pro / Enterprise / Education
Processor	Intel Core or higher performance
RAM	8 GB or more
Hard disk space	4 GB or more
Display resolution Patient secondary display resolution	1280 x 1024 pixels 1366 x 768 pixels
Graphic card	16 Million (24 bit) screen colors or more
Drive for manual software installation	DVD USB
Serial COM port	One for HI-PRO if used via serial COM port
USB ports	One for each purpose: Bluetooth adaptor Accessory programming HI-PRO USB / HI-PRO 2 / iCube II / Noahlink Wireless
Programming interfaces	iCube II NOAHlink / Noahlink Wireless HI-PRO / HI-PRO USB / HI-PRO 2
NOAHlink driver	Latest version available
NOAHlink Wireless driver	Latest version available
Internet connection	Highly recommended
Sound card	Stereo
Playback system	20 Hz — 14 kHz
Noah version	Latest version (Noah 4.4 or higher) Please check the Noah limitations for Windows operating systems on http://www.himsa.com
Automatic REM	Noah Version 4.4.2280 or higher Natus Otosuite 4.81.00 or higher Audit data Primus version 4.1-5.2 Interacoustics Affinity Suite version 2.19-2.24 _ Signia Unity version 5.9-6.2

Need assistance?

Australia – 1800 212 313	Netherlands – +31 88 600 88 10
Austria – +49 711 658538 0	New Zealand – 0800 864 8766
Belgium – +31 88 600 88 10	Russia – +7 495 788 02 04
Brazil – +55 11 3748 0960	South Korea – +82 2 859 4902
Canada – 1 800 265 8255	Spain – +34 902 494 960
China – +86 21 61205533	Sweden – +46 (0) 8 546 20 960
Denmark – +45 64 41 78 87	Switzerland – 0800 928 800
France – 04 26 23 22 00	Taiwan – +886 2 22216722
Germany – +49 711 658538 0	Turkey – +90 212 232 67 22
India – +91-22-26871151	U.K. and Ireland – +44 (0) 1925 247810
Mexico – +52 55 5207 1703	U.S. – 1 800 888 8882

For countries not listed – +1 519 895 0100

Instructions are available at www.unitron.com/instr

CE

0459



Sonova AG

Laubisrütistrasse 28

CH-8712 Stäfa, Switzerland

EC REP

Importer for European Union:

Sonova Deutschland GmbH

Max-Eyth-Str. 20

70736 Fellbach-Oeffingen, Germany

REF

058-5178-056

029-6849-G4/V1.00/2024-04/dr/ ©2023-2024 Sonova AG or its affiliates. All rights reserved.



sonova
HEAR THE WORLD

Documents / Resources

	Unitron TrueFit Fitting Software [pdf] Installation Guide TrueFit Fitting Software, Fitting Software, Software
--	--

References

- HIMSA

- [U Instruction archives](#)
- [User Manual](#)

[Manuals+](#), [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.