


Univox NL-100 Neck Loop



Univox NL-100 Neck Loop User Guide

[Home](#) » [univox](#) » Univox NL-100 Neck Loop User Guide 

Contents

- [1 Univox NL-100 Neck Loop](#)
- [2 Product Information](#)
- [3 Product Usage Instructions](#)
- [4 FAQ](#)
- [5 USER GUIDE](#)
- [6 Maintenance, care and service](#)
- [7 Technical specifications](#)
- [8 Documents / Resources](#)
 - [8.1 References](#)



Univox NL-100 Neck Loop



Product Information

Technical Specifications

- **Resistance:** 2 x 12.5 Ohm
- **Sensitivity:** 0dB @ 55mVrms
- **Frequency Response:** 50Hz-15kHz, -3dB, rel. 1kHz
- **Connector:** 3.5 mm plug
- **Dimensions:** Circumference: 74 cm (29 in), Cable: 20 cm (7.8 in), Extension Cord: 70 cm (27.5 in)
- **Weight (net):** 20 g (0.7 oz) including extension cord
- **Part No:** 283103

Product Usage Instructions

1. Connection and General Information

Before using the NL-100, ensure your device volume is set to the minimum to prevent sudden loud sounds that could harm your hearing aid.

2. How to Use the NL-100:

The NL-100 is an inductive neck loop designed for hard of hearing individuals with T-coil equipped hearing aids. Simply connect the NL-100 to your audio device and wear it around your neck. The device will transfer audio personalized to your hearing impairment with minimal latency.

3. Maintenance, Care, and Service

The NL-100 generally does not require special maintenance. To clean, wipe the unit with a clean damp cloth. Avoid using solvents or detergents. Periodically inspect cables and connectors for damage. In case of issues, contact your dealer or local distributor for further guidance.

4. Environment

Dispose of the product responsibly following local disposal regulations to prevent harm to the environment and human health.

FAQ

Q: Can I use the NL-100 with any type of hearing aid?

A: The NL-100 is compatible with hearing aids equipped with a T-coil. Ensure your hearing aid supports this feature before using the NL-100.

Q: How long is the cable of the NL-100?

A: The NL-100 comes with a cable length of 74 cm (29 in) and an extension cord of 70 cm (27.5 in).

USER GUIDE

An inductive neck loop is a personal assistive listening accessory for hard-of-hearing people wearing hearing aids equipped with a T-coil. Each hearing aid is configured with frequency response and dynamic range tuned to the user's hearing impairment. The NL-100 transfers any audio from its connected device crystal clear, personalized and with minimal latency.

Univox® NL-100

Univox® NL-100 is capable of connecting to, and receiving signals from different sound sources. NL-100 sums both right and left channels of the audio source, ensuring a strong and steady signal for the user. Included in the package is a 70 cm (27.5 in) extension lead.

Connection and general information

How to use the NL-100:

- Wear the loop over your head around your neck
- Connect the NL-100 to the headphone jack of your device
- Turn your hearing aid to the telecoil position (T-mode)
- Adjust the volume on your device

NOTE! We recommend setting your device to the minimum volume before connecting the NL-100 to avoid any sudden loud sounds that may potentially harm your hearing aid.

Maintenance, care and service

Under normal circumstances, the product does not need any special maintenance. Wipe the unit with a clean damp cloth. Do not use any solvents or detergents. Periodically check cables and connectors for any damage. Should the unit not work as expected, contact your dealer or local distributor for further instructions.


Environment

To prevent possible harm to the environment and human health, please dispose of the product responsibly by following local statutory disposal regulations

Technical specifications

- Resistance 2 x 12.5 Ohm
 - Sensitivity 0dBL @ 55mVrms
 - Frequency response 50Hz-15kHz, -3dB, rel. 1kHz
 - Connector 3.5 mm plug
 - Dimensions Circumference 74 cm (29 in)
 - Cable 20 cm (7.8 in)
 - Extension cord 70 cm (27.5 in)
 - Weight (net) 20 g (0.7 oz) incl extension cord
 - Part No 283103
-

Documents / Resources

 The image shows the cover of the Univox NL-100 Neck Loop User Guide. It features a black and white photograph of the neck loop device, which is a flexible, looped cable with a 3.5mm audio plug at one end. The text on the cover includes 'Univox NL-100 Neck Loop' and 'User Guide'.	<p>Univox NL-100 Neck Loop [pdf] User Guide NL-100 Neck Loop, NL-100, Neck Loop, Loop</p>
--	---

References

- [User Manual](#)

Manuals+, Privacy Policy

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.