



Vibrathotics V2 User Manual

[Home](#) » [Vibrathotics](#) » Vibrathotics V2 User Manual 

Contents

- 1 [Vibrathotics V2 User Manual](#)
- 2 [RELIEVE. REVIVE. RECOVER.](#)
- 3 [INSTRUCTIONS:](#)
- 4 [CONSTANT](#)
- 5 [PULSING](#)
 - 5.1 [VIBRATHOTICS INSOLES ARE NOT WATER-RESISTANT](#)
 - 5.2 [MONEY-BACK GUARANTEE:](#)
 - 5.3 [LIMITED WARRANTY COVERAGE FOR 90 DAYS \(FOR USA ONLY\)](#)
 - 5.4 [LIMITED WARRANTY LIMIT AND EXCLUSIONS](#)
- 6 [SPECIFICATIONS:](#)
- 7 [VIBRATHOTICS](#)
- 8 [Questions about your Manual? Post in the comments!](#)
 - 8.1 [Related Posts](#)

Vibrathotics V2 User Manual



RELIEVE. REVIVE. RECOVER.

Use Vibrathotics to relieve, revive, and recover from a workout or a long day on your feet. For best results, follow the guidelines below...

Relieve/Comfort: As needed

Revive/Injury: 30-60 minutes 2 to 3 times a day or as needed

Recover/Workout: 30 minutes before and after workout/exercise or as needed

INSTRUCTIONS:

- Remove original insoles from your shoes and replace with Vibrathotics insoles.
- Shoe manufacturers vary in size. Vibrathotics insoles are created to fit all manufacturers, therefore may require minor trimming.
- To charge, remove Vibrathotics insoles from shoes and plug in using provided wall adapter and cable. To fully charge, leave connected for approximately 2 hours.
- To turn on the Vibrathotics insoles, choose the desired setting Constant (C) or Pulsing (P):

CONSTANT

- Click top left button on the remote control to turn constant setting on low.
- Continue to press the button until desired vibration level is achieved (low, medium, high).
- Press the bottom button on the remote control to turn off the insoles.

PULSING

- Press the top right button on the remote control to turn pulsing setting on low.
- Continue to press the button until desired vibration level is achieved (low, medium, high).
- Press the bottom button on the remote control to turn off the insoles.

VIBRATHOTICS INSOLES ARE NOT WATER-RESISTANT

Vibrathotics are NOT designed to be water resistant or waterproof. Do not wear Vibrathotics in shoes that will submersed in any body of water.

- Do not wash Vibrathotics insoles.
- The remote is not water resistant or waterproof and should avoid getting wet.

MONEY-BACK GUARANTEE:

Vibrathotics offers a 30-day money-back guarantee. If for any reason you are not satisfied with your pair of Vibrathotics insoles, retain product, packaging, and sales receipt and visit vibrathotics.com/guarantee for information and instructions.

LIMITED WARRANTY COVERAGE FOR 90 DAYS (FOR USA ONLY)

Vibrathotics offers a 90-day limited warranty on Vibrathotics insoles which covers defects in materials or workmanship. Vibrathotics will refund your purchase price for up to 90 days after the purchase date of your Vibrathotics insoles.

- This warranty only applies to products purchased in the United States.
- This warranty is extended only to the original purchaser of a new product which was not sold “as is”.

LIMITED WARRANTY LIMIT AND EXCLUSIONS

This warranty ONLY COVERS failures due to defects in materials or workmanship and DOES NOT COVER normal wear and tear or cosmetic damage.

THERE ARE NO EXPRESS WARRANTIES EXCEPT AS LISTED UNDER “LIMITED WARRANTY COVERAGE”. THE WARRANTOR IS NOT LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES RESULTING FROM THE USE OF THIS PRODUCT OR ARISING OUT OF ANY BREACH OF THIS WARRANTY.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference.
2. This device must accept any interference received, including interference that may cause undesirable operation.

SPECIFICATIONS:

- RF Frequency: 315MHz
- Battery Capacity: 430mAh
- Working Hours: approximately 10 hours at maximum constant setting

VIBRATHOTICS

QUESTIONS? 1-800-925-6526

VIBRATHOTICS, PO BOX 626, AMERICAN FORK, UT, 84003



FCC ID: 2ARTSREMOTE
FCC ID: 2ARTS-V3

Model #: Vibrathotics Remote V3
Model #: Vibrathotics Left/Right V3

Read More About This User Manuals...

[Vibrathotics-V2-Manual-Optimized.pdf](#)

[Vibrathotics-V2-Manual-Original.pdf](#)

Questions about your Manual? Post in the comments!

Manuals+.