



VisuDoc Ophthalmic Case Management App User Guide



[Home](#) » [VisuDoc](#) » VisuDoc Ophthalmic Case Management App User Guide 

Contents

- 1 [VisuDoc Ophthalmic Case Management App](#)
- 2 [Installation of PC database](#)
- 3 [Start VisuDoc PC](#)
- 4 [Set Server](#)
- 5 [Open VisuDoc Account Management System \(AMS\)](#)
- 6 [Enter AMS](#)
- 7 [Create User Account](#)
- 8 [Set Hospital Informaiton](#)
- 9 [Download VisuDoc App](#)
- 10 [Start VisuDoc App](#)
- 11 [Buildup New Case](#)
- 12 [Image Capture](#)
- 13 [Image Edit and Save](#)
- 14 [Add Diagnosis Opinions](#)
- 15 [Data Transfer](#)
- 16 [Contact us](#)
- 17 [Documents / Resources](#)
 - 17.1 [References](#)
- 18 [Related Posts](#)




VisuDoc Ophthalmic Case Management App

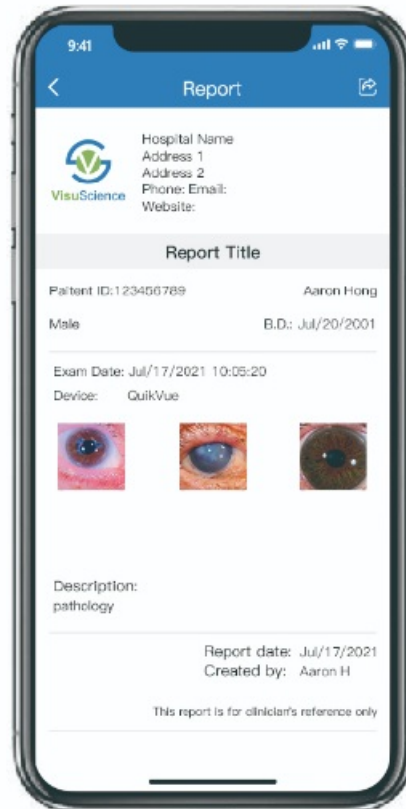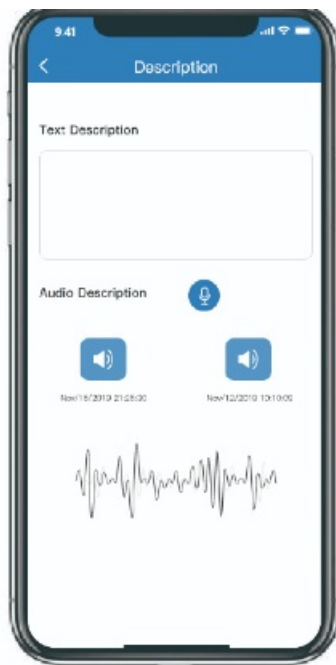

Installation of PC database

- Save the VisuDoc PC file on desktop

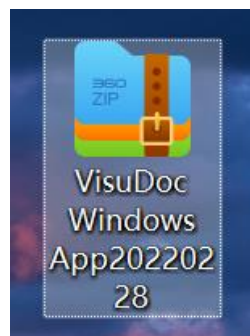

- Decompress the file on desired folder

Start VisuDoc PC

 uni_image_compress_plugin.dll	2021
 uni_process_finder_plugin.dll	2021
 url_launcher_windows_plugin.dll	2021
 VisuDoc	2021
 vs-app	2021



VisuDoc

Account

Password



VisuDoc AMS

Server Setting

Sign In


[Terms of Use & Privacy Policy](#)

Copyright © 2020-2021 VisuScience Meditech


1. Click VisuDoc.exe to open VisuDoc PC version

Set Server

Click Server setting to set server

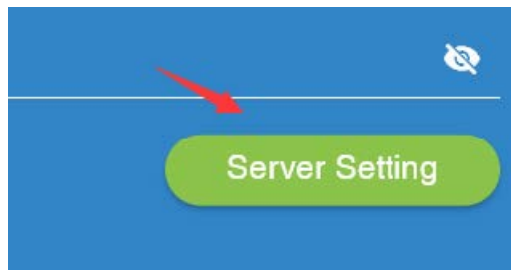


1. Select 'Local'
2. Open your wifi property and check the IP address
3. Input same IP address in 'server address' and input '4430' in server port
4. Click 'Test' to test server. When the setting is successful, click save

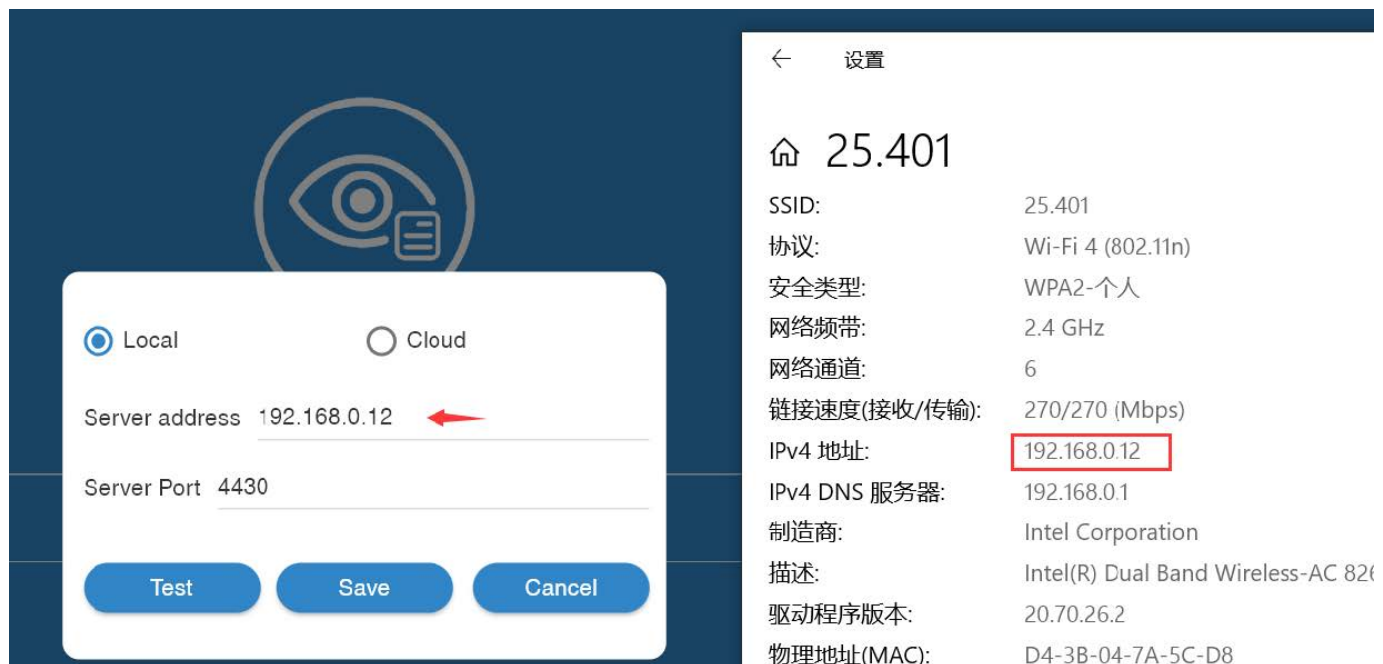


Open VisuDoc Account Management System (AMS)

1. Double click the 'VisuDoc AMS'

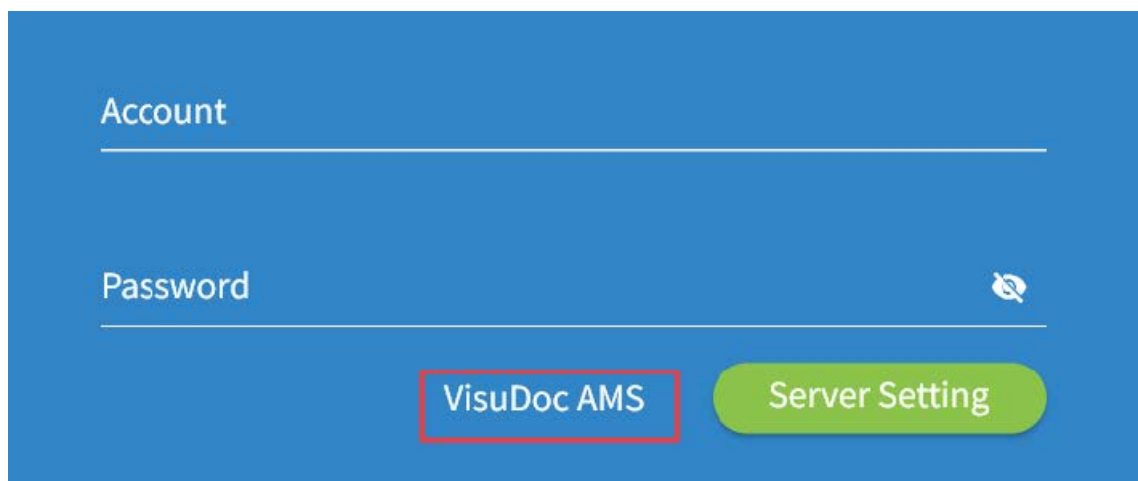

Enter AMS

1. A browser will be opened and showing VisuDoc Account Management System
2. Click 'A' icon to change Language to English
3. Enter User name 'Admin' and default password 'visu20211111'

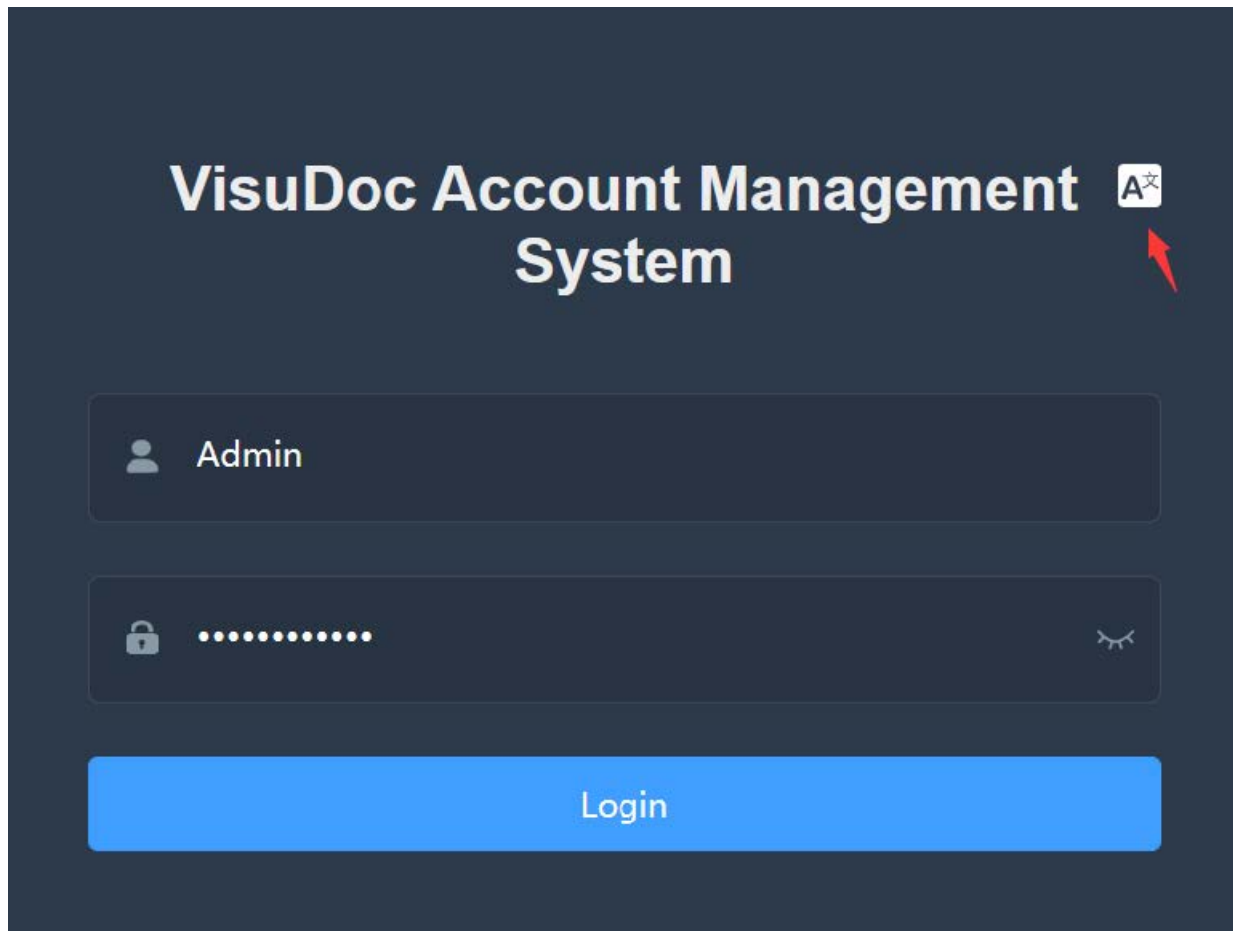

Create User Account

1. Click User on the left side

The image displays the "Users" management page in the VisuScience application. The left sidebar contains a menu with options: Dashboard, Users (highlighted), Patients, Exams, Report Setting, Operation Logs, and Change Password. The main content area shows a table of existing users. At the top right of the table, there is a "Create" button. The table has columns for User ID, Account, First Name, Last Name, Is Admin, Join Time, and Operation. Two users are listed: User ID 2 (zhouyaya, zhou, yaya, No, 2022-01-11 18:40:41 CST) and User ID 1 (Admin, Super, User, Yes, 2021-12-31 14:46:46 CST). Each user has "Update", "Password", and "Remove" buttons. Below the table, there is a pagination bar showing "Total 2" and a page selector with "1" highlighted.

User ID	Account	First Name	Last Name	Is Admin	Join Time	Operation
2	zhouyaya	zhou	yaya	No	2022-01-11 18:40:41 CST	Update Password Remove
1	Admin	Super	User	Yes	2021-12-31 14:46:46 CST	Update Password Remove

2. Click 'Create' to create new user account



Create

Time	Operation		
3:40:41 CST	Update	Password	Remove
4:46:46 CST	Update	Password	Remove

3. Input Account name, first name, last name and password. Then Click submit

Account

User1

First Name

Sara

Last Name

Zhou

Password

.....

Is Admin



Submit

Set Hospital Informaiton

1. Click 'Report Setting' to setup hospital information.




Dashboard / Report Setting




 Dashboard
  Users
  Patients
  Exams
  Report Setting
  Operation Logs
  Change Password

Title

Title EYE TEST REPORT

Hospital / Clinic Info

Name ABC HOSPITAL

Address1 NO.344 SAN LIN ROAD

Address2


Phone 123456789

Email info@abc.com

Website www.abc.com

Download VisuDoc App

1. Search 'VisuDoc' on App store and Google Play and download

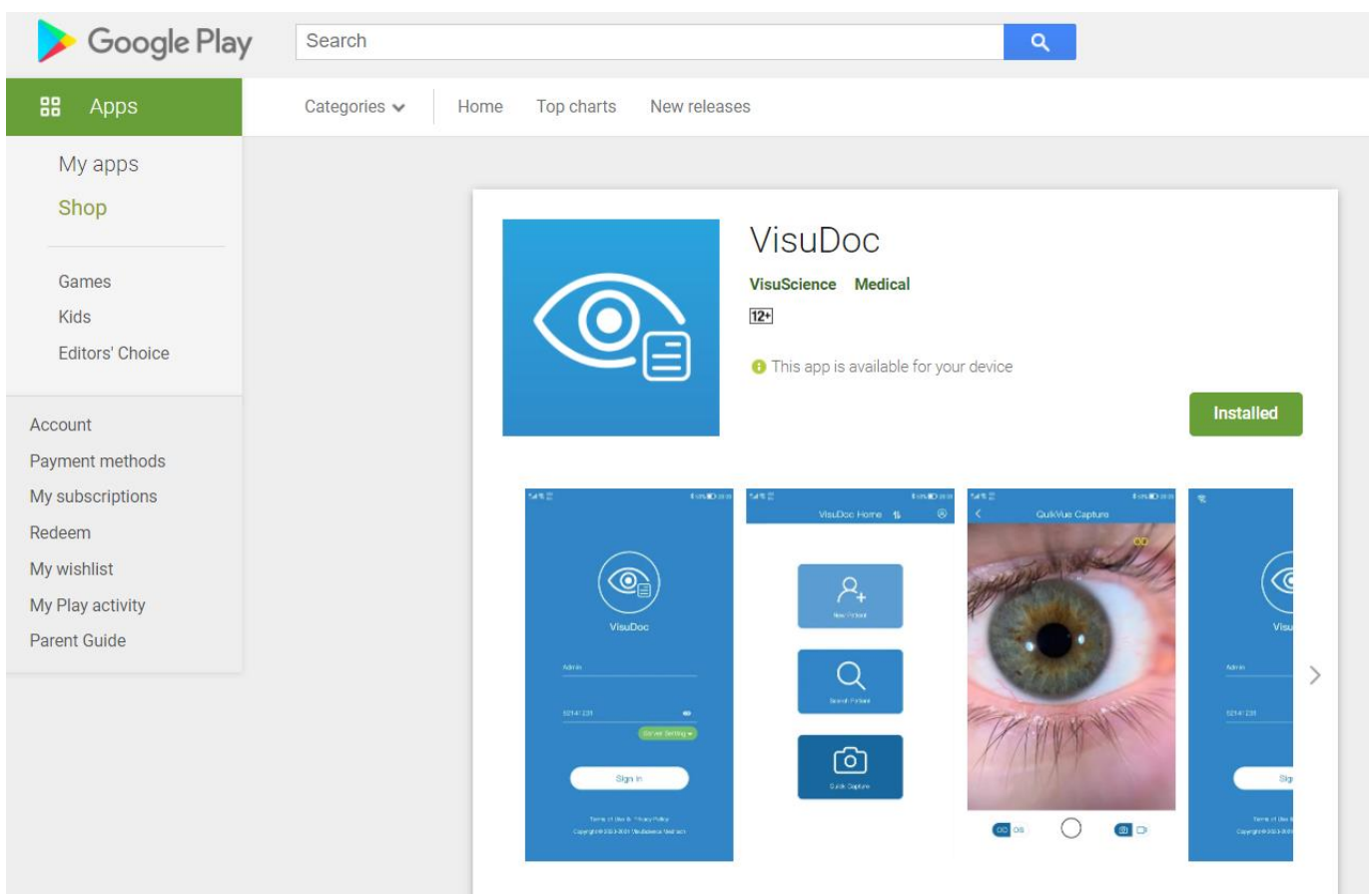

Start VisuDoc App

1. After downloading VisuDoc app, please open it
2. Tap 'server' in Sign page and input same IP address and port number as you did in VisuDoc PC
3. Click test and save



☒ Local

☐ Cloud

Server address 192.168.0.12

Server Port 4430

Test

Save


Cancel

Sign In

[Terms of Use & Privacy Policy](#)


Copyright © 2020-2021 VisuScience Meditech

- Input account name and password you created in AMS and tap signin

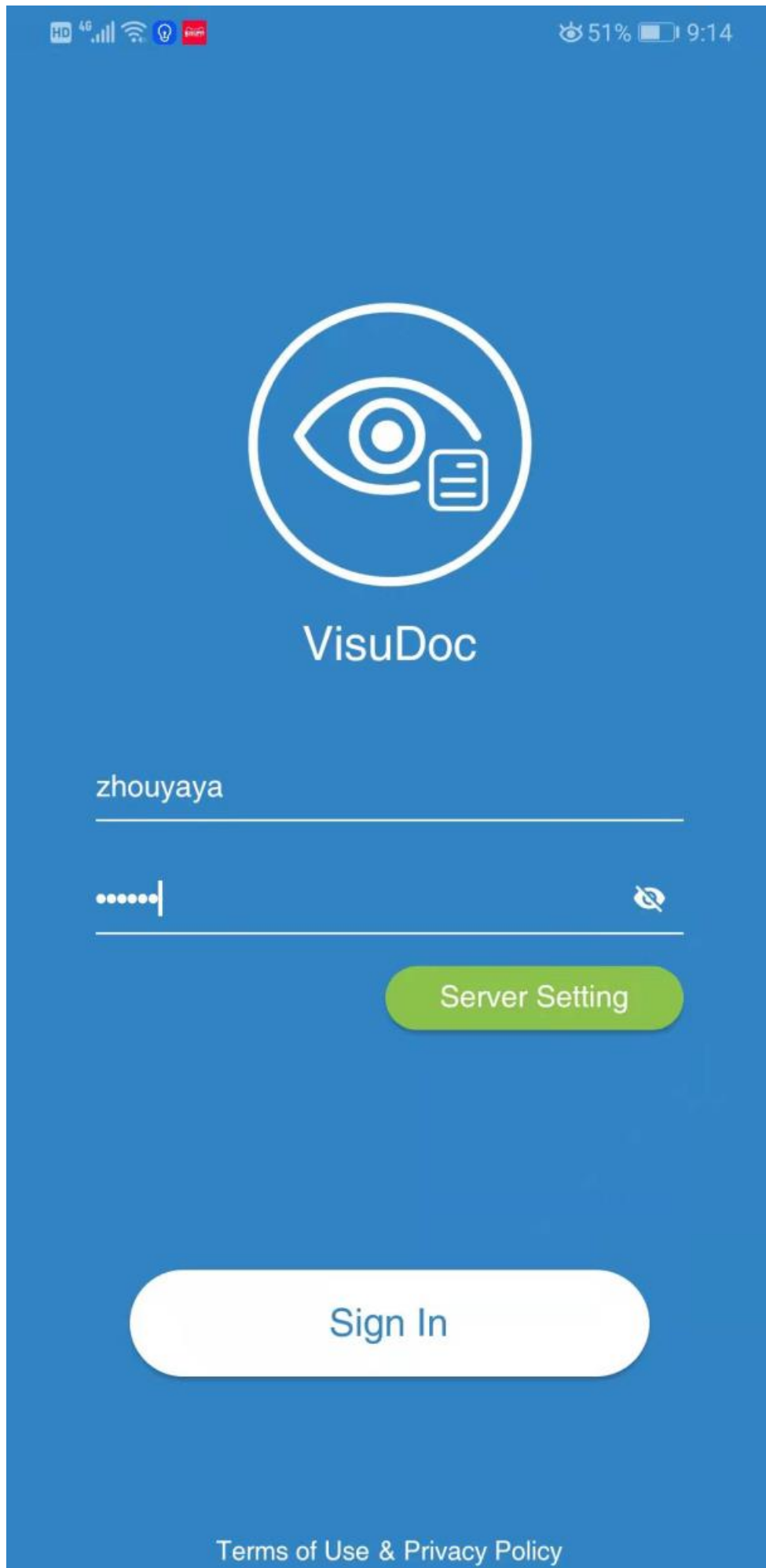The image shows a mobile application login screen for 'VisuDoc'. The background is a solid blue color. At the top, there is a status bar with various icons (HD, 4G, signal strength, Wi-Fi, battery, and time) and the text '51%' and '9:14'. In the center, there is a large white circular logo containing a stylized eye and a document icon. Below the logo, the text 'VisuDoc' is displayed in a white, sans-serif font. Underneath the logo, there are two input fields. The first field contains the text 'zhouyaya'. The second field contains a series of dots, indicating a password, and has a small icon of an eye with a slash through it to the right, likely for toggling password visibility. Below the password field, there is a green rounded rectangular button with the text 'Server Setting'. At the bottom of the screen, there is a large white rounded rectangular button with the text 'Sign In'. At the very bottom, there is a link that says 'Terms of Use & Privacy Policy'.

Buildup New Case

1. Tap 'new patient' and input patient information to build up new patient case

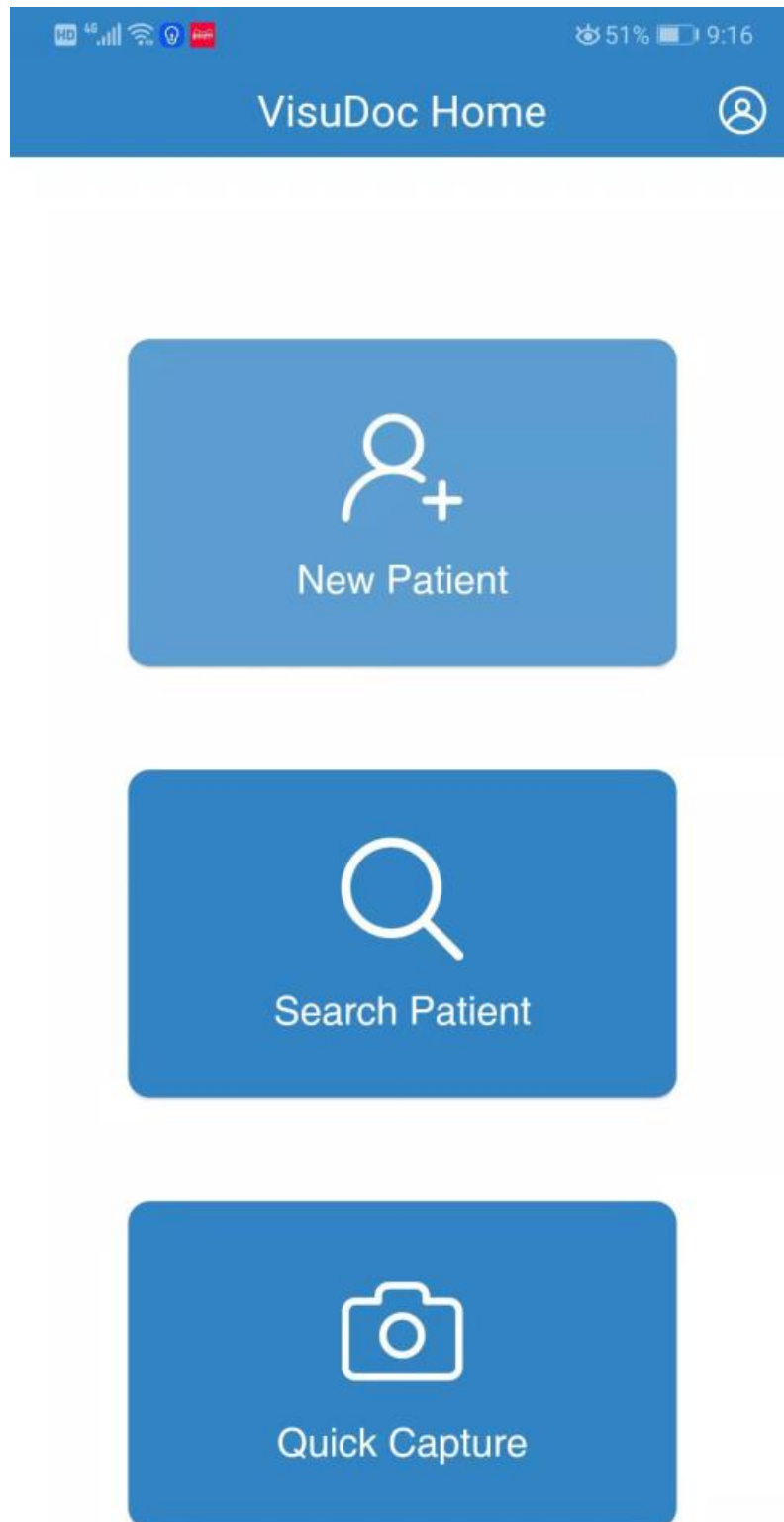

HD 4G LTE Wi-Fi V 51% 9:16

< New Patient Save

Image Capture

1. Tap the case add icon and select 'QuikVue'
- Remark: MeiboVue is still under development


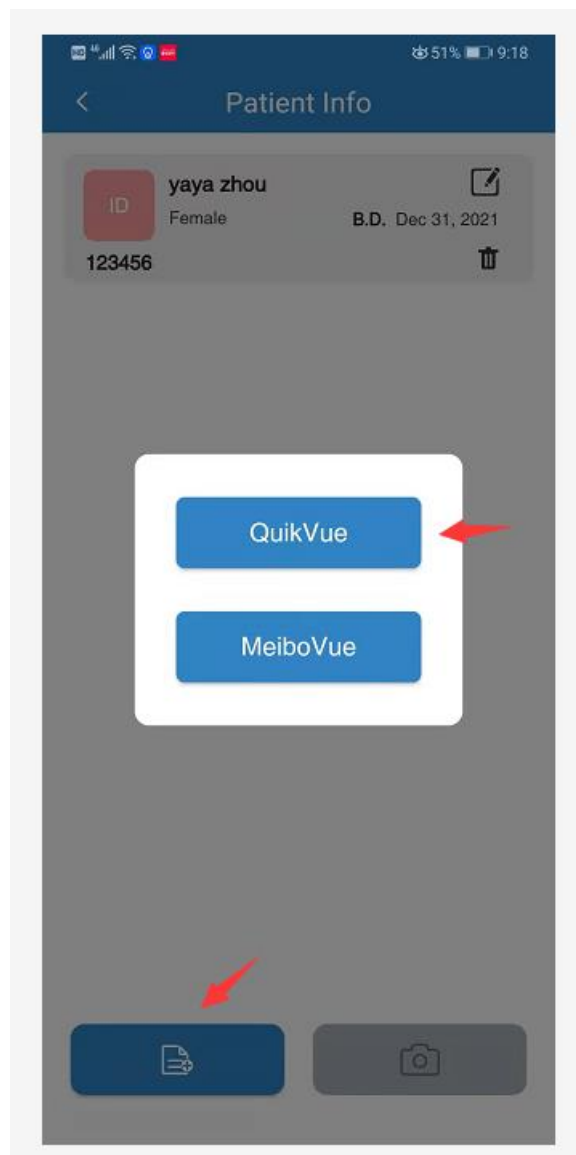


Image Edit and Save

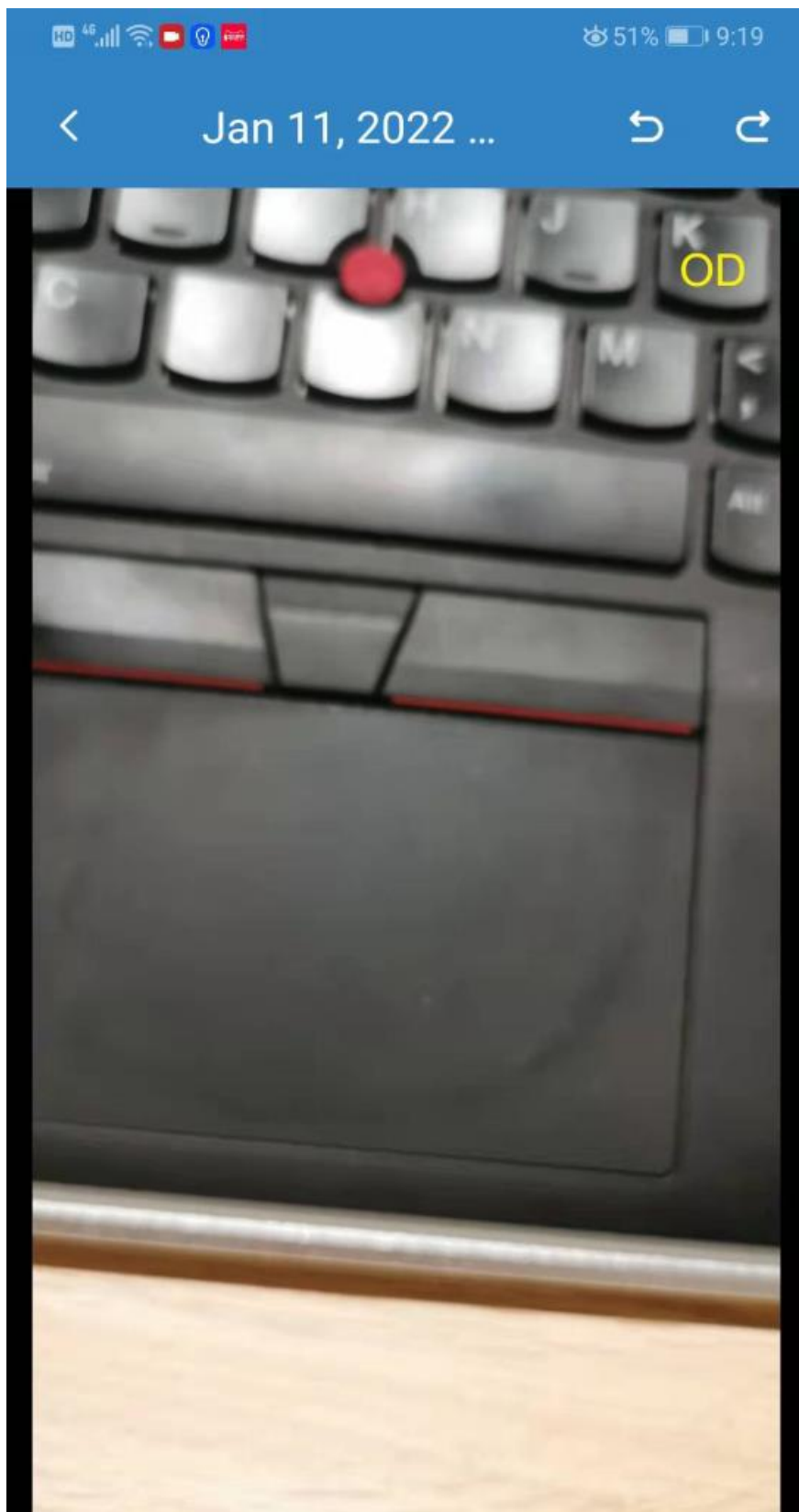

- Make mark
- Add text
- Save as favorite

- Download to phone album
- Save image to current patient case

Add Diagnosis Opinions



Patient Info

srtt sdgg

Female

B.D. Jan 11, 2022

werf



Exam Date: Jan 11, 2022 18:41:46



Device: QuikVue



Tap to Load More





Description

Text Description


Audio Description



Jan 11, 2022 18:42:01

1. Add diagnostic opinions
2. make audio diagnostic record

Data Transfer

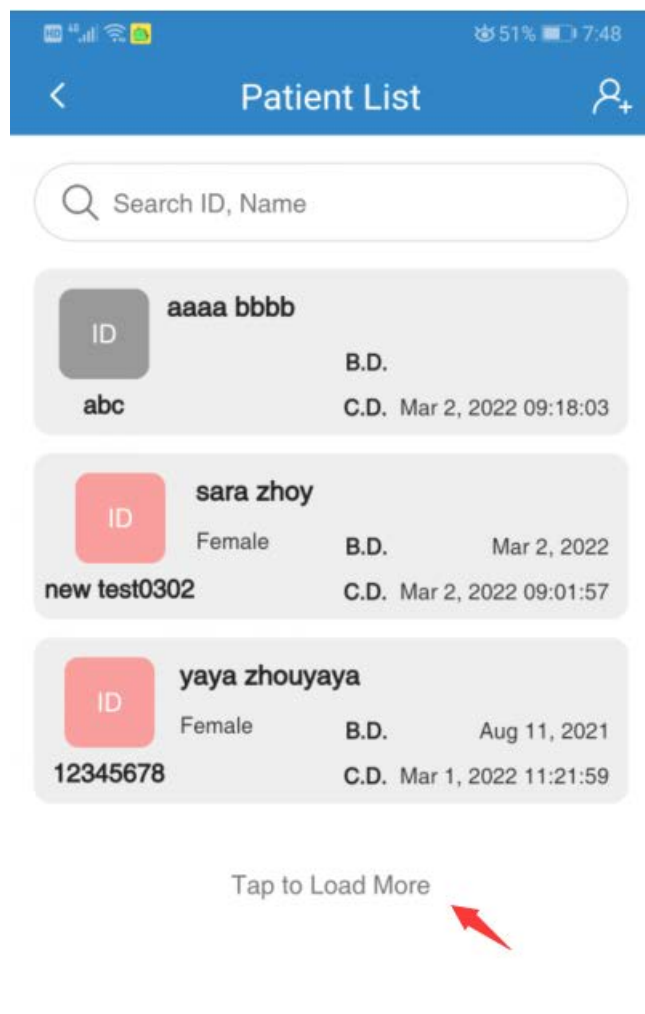

1. When VisuDoc app and PC version running together, please slide to refresh the patient list page on smartphone to sync the data from mobile to PC.

Contact us

Shanghai VisuScience Meditech Co., Ltd.
 Email: info@visuscience.com
www.visuscience.com

Documents / Resources

	VisuDoc Ophthalmic Case Management App [pdf] User Guide Ophthalmic Case Management App
---	---

References

- [Home - VisuScience](#)

