

VORTEX[®]
B0C7PG8H72
One Handed
Operation Aid



VORTEX B0C7PG8H72 One Handed Operation Aid Instruction Manual

[Home](#) » [VORTEX](#) » VORTEX B0C7PG8H72 One Handed Operation Aid Instruction Manual 

Contents

- [1 VORTEX B0C7PG8H72 One Handed Operation Aid](#)
- [2 Product Usage Instructions](#)
- [3 Frequently Asked Questions](#)
- [4 ASSEMBLING THE PARTS](#)
- [5 INSTRUCTIONS FOR CLEANING](#)
- [6 Documents / Resources](#)
 - [6.1 References](#)
- [7 Related Posts](#)



VORTEX B0C7PG8H72 One Handed Operation Aid



Product Specifications:

- **Product Name:** Vortex
- **Intended Use:** Spacer/valved holding chamber for delivery of aerosol medication with metered dose inhalers
- **Manufacturer:** PARI Respiratory Equipment, Inc.

- **Contact Information:** Tel: 1-[804-253-7274](tel:804-253-7274), Email: product.info@pari.com
- **Address:** 2412 PARI Way, Midlothian, VA 23112

Product Usage Instructions

Assembling the Parts:

1. Step 1:
2. Step 4:
3. Vigorously shake the inhaler before every use.
4. Connect the inhaler to the connecting ring on the inhalation chamber.
5. Step 5 (only if using a mask): Attach the mask or mouthpiece securely.

Instructions for Cleaning:

Washing by Hand:

1. Clean the parts before first use and then clean and disinfect once a week thereafter.
2. Remove the one-handed operation aid from the chamber.
3. **A. Hand Washing:**
 - Soak the part for 15 minutes in lukewarm water with liquid detergent. Agitate gently, then rinse in clear water.
 - Rinse the part in clean water and shake out any excess water.
 - Air dry thoroughly on a clean, lint-free cloth or drying rack before storing for future use.
4. **B. Dishwasher Cleaning:**
 - Place the part in the top shelf of the dishwasher for cleaning.
 - Remove from the dishwasher, shake out excess water, and air dry as instructed above.

Disinfection: After cleaning, disinfect all parts by placing them in boiling water for 5 minutes. Dry as instructed above.

Frequently Asked Questions

Can I use Vortex with any metered dose inhaler?

Yes, Vortex is designed to work with most metered dose inhalers for aerosol medication delivery.

How often should I clean and disinfect Vortex?

You should clean and disinfect Vortex once a week after first use to maintain proper hygiene and functionality.

Product is non-sterile, not made with BPA or latex.

For use only with PARI Vortex® Non Electrostatic Holding Chamber.


READ ALL INSTRUCTIONS BEFORE USE.

Vortex is indicated for use as a spacer/valved holding chamber for use in delivery of aerosol medication with metered dose inhalers.

ASSEMBLING THE PARTS


Step 1:

- Push the one-handed operation aid onto the inhalation chamber from the back.

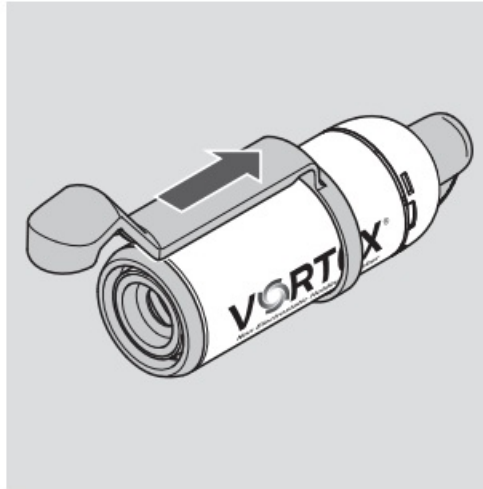


Step 2:

- Remove the cap on the metered dose inhaler.
- Shake the metered dose inhaler vigorously before every use.

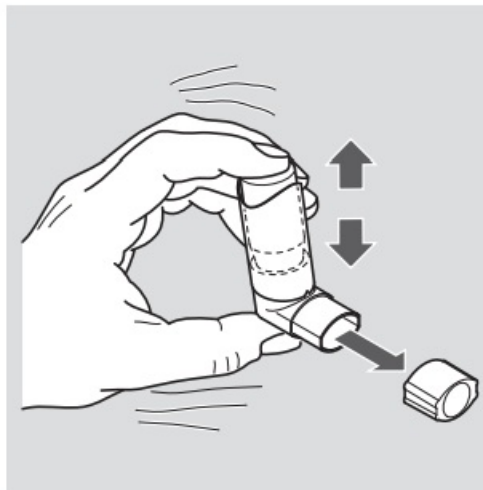


Step 3:

- Insert the metered dose inhaler in the connecting ring on the inhalation chamber.

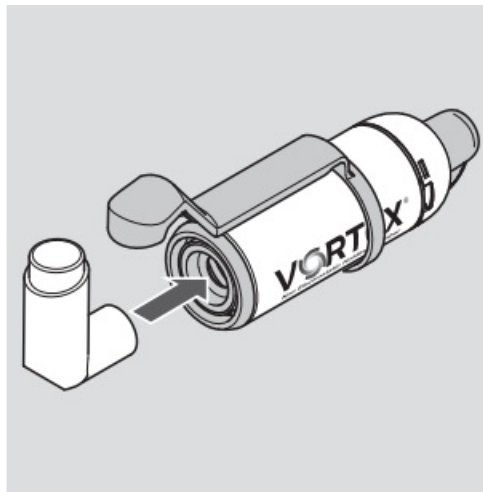

Step 4:

- Position the one-handed operation aid so that its round end encloses the metered dose inhaler. To do this, you may have to push the one-handed operation aid farther forwards or backwards.

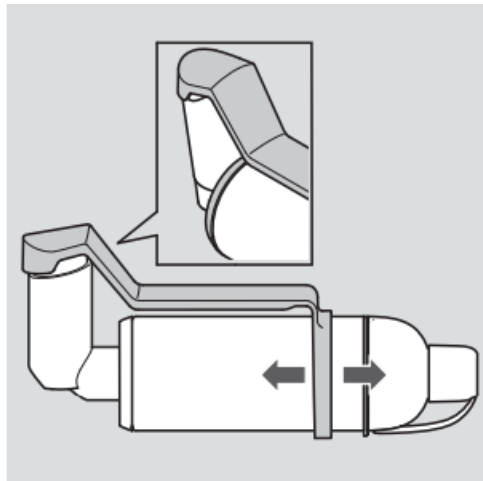

Step 5 (only if using a mask):

Attach the mask (pediatric mask)


- Take the protective cap off the mouthpiece of the VORTEX.
- Attach the mask to the VORTEX mouthpiece.

OR


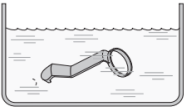
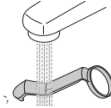
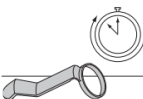


Attach the mask (adult mask)

- Take the protective cap off the mouthpiece of the VORTEX.
- Fit the mask connector into the opening of the PARI adult mask soft.
- Attach the mask to the mouthpiece of the VORTEX with the mask connector.

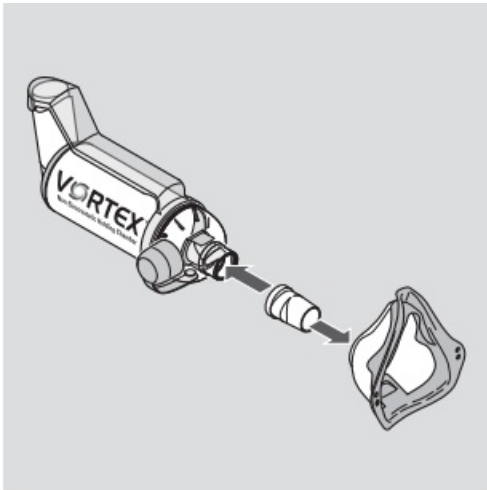


INSTRUCTIONS FOR CLEANING

Washing by Hand				
Clean as instructed before first use. After first use, clean and disinfect once a week.	Remove the one handed operation aid from the chamber.	A. Soak part for 15 minutes in luke warm water with liquid detergent. Agitate gently then rinse in clear water. OR B. Place part in top shelf of dishwasher.	A. Rinse in clean water. Shake out in sink. OR B. Remove from dishwasher. Shake out excess water.	Thoroughly air dry on a clean, lint-free cloth or on a drying rack. When dry, store for future use.

DISINFECTION: To disinfect after cleaning, place all parts in boiling water for 5 minutes. Dry as instructed above.

PARI Respiratory Equipment, Inc.
Tel: 1-[804-253-7274](tel:8042537274)
www.pari.com | product.info@pari.com
2412 PARI Way, Midlothian, VA 23112

Documents / Resources

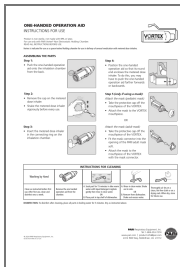

[VORTEX B0C7PG8H72 One Handed Operation Aid](#) [pdf] Instruction Manual
B0C7PG8H72 One Handed Operation Aid, B0C7PG8H72, One Handed Operation Aid, Handed
Operation Aid, Operation Aid, Aid

References

- [PARI Home - PARI](#)
- [User Manual](#)

[Manuals+](#), [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.