



# WISELIFT WLT400 Rolling Work Station Swivel Table User Manual

[Home](#) » [WiseLift](#) » WISELIFT WLT400 Rolling Work Station Swivel Table User Manual 

## Contents

- [1 WISELIFT WLT400 Rolling Work Station Swivel Table](#)
- [2 SAFETY AND PRECAUTIONS](#)
- [3 WHAT'S IN THE BOX](#)
- [4 YOUR WISELIFT TABLE](#)
- [5 ASSEMBLY](#)
- [6 OPERATION](#)
- [7 CLEAN AND CARE](#)
- [8 Warranty](#)
- [9 SPECIFICATIONS](#)
- [10 Documents / Resources](#)
  - [10.1 References](#)
- [11 Related Posts](#)



**WISELIFT WLT400 Rolling Work Station Swivel Table**



## **SAFETY AND PRECAUTIONS**

Read and follow all instructions, warnings, and notes in this manual and all other accompanying literature before attempting to operate this product for the first time. If there is any information in this manual which you do not understand, or if you require additional assistance for setup or operation, please Contact a WiseLift representative at 1-877-782-2010. Failure to follow the instructions, warnings, and notes in this manual and those located on your product can result in personal injury or product damage and will void the product warranty.

### **NO MODIFICATIONS**

Your table has been designed and engineered to provide comfort and functionality. However, under no circumstances should you modify, add, remove, or disable any feature, part, or function of your Lift Table. Use WiseLift parts only for all repairs and replacements.

### **PINCH/CRUSH POINT HAZARDS**

Do not place objects or appendages in the path of moving parts. The scissor and lift mechanisms are labeled as pinch/crush point hazards on your table. Keep the following items and similar objects clear of any Pinch/Crush points:

- Appendages
- Children
- Pets
- Wire/Tubes (ex. oxygen tubing)
- Loose items, including but not limited to clothing and blankets
- Electrical power cords

### **Storage and Operation Temperatures**

- Transportation or storage: -104°F/-75°C to 158°F/70°C
- Operation: 50°F/10°C to 104°F/40°C

### **Purchaser agreement**

By accepting delivery of this product, you promise that you will not change, alter, or modify this product or remove

or render inoperable or unsafe any guards, shields, or other safety features of this product; fail, refuse, or neglect to install any retrofit kits from time to time provided by WiseLift to enhance or preserve the safe use of this product.

**NOTE:** If you ever lose or misplace your copy of this manual, contact us and we will be glad to send you a new one immediately. You can also go to [www.wiselift.com](http://www.wiselift.com) to download our user manual.

## WHAT'S IN THE BOX

Please identify all of the items for your WiseLift Table:

Base



Table



Column



Column Port Covers




Cup Holder




USB Power Hub




### Wheel Assembly Hardware

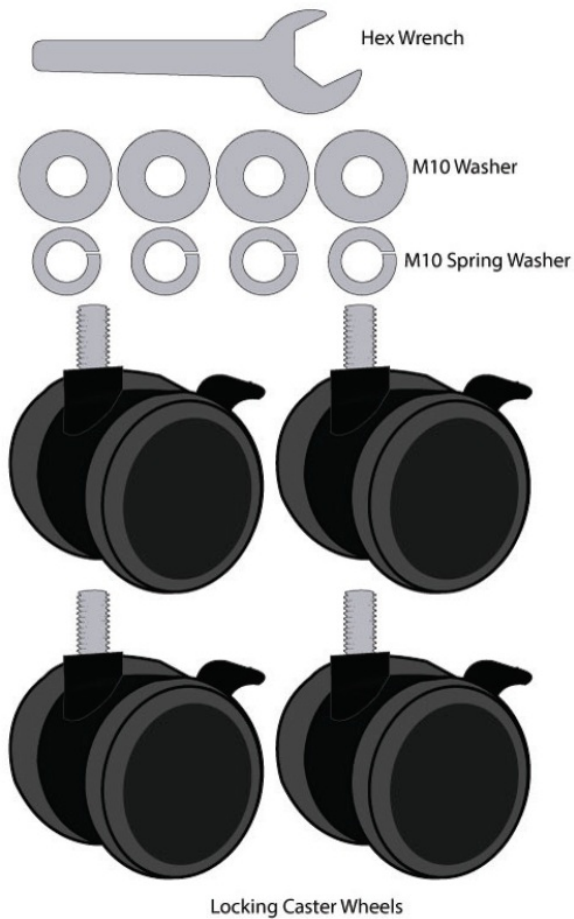


### Base Assembly Hardware

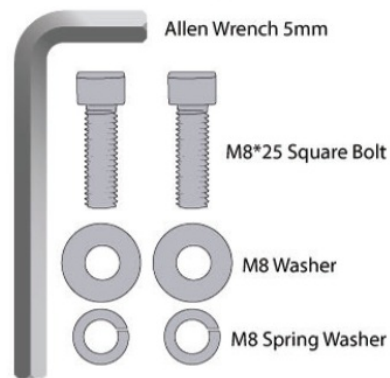

### Table-Top Assembly Hardware

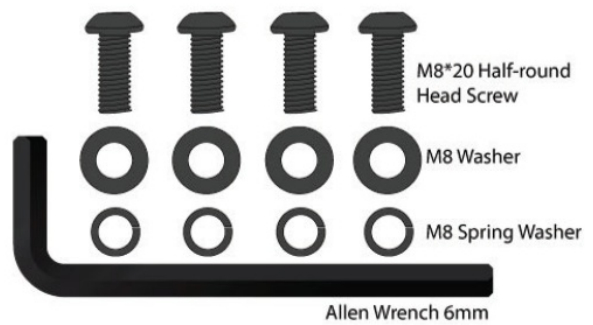


### Lift Handle Assembly Hardware

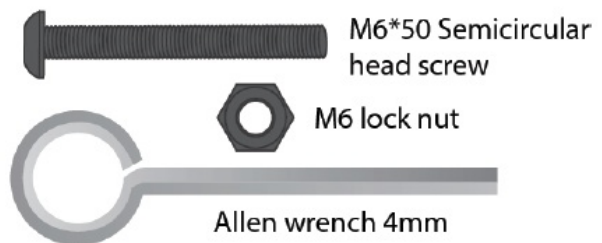

Lift Handle (top)



Lift Handle (bottom)

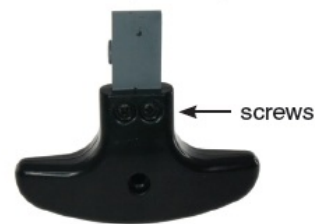

## YOUR WISELIFT TABLE

This section describes the features of your table. Carefully review the function and location of each item described.

Your WiseLift table USB port operates on standard household current and can be plugged into any standard 110-volt wall outlet. Use a surge protector with this table.


### TABLE PLACEMENT

Your lift table should be placed near a standard electrical outlet on dry, level ground where there is ample room to

allow for proper operation. WiseLift recommends that you have the assistance of an attendant when positioning the lift table to avoid the possibility of injury when lifting.

Follow these steps to position the table in a safe manner:

1. Lay the 3 main parts out on the floor similar to figure 1.1
2. Follow the assembly instructions for the table.
3. Plug the power cord into a surge protector and then into an electrical outlet. Do not use a standard extension cord!
4. Position the low voltage connection cable where it will not be pinched.

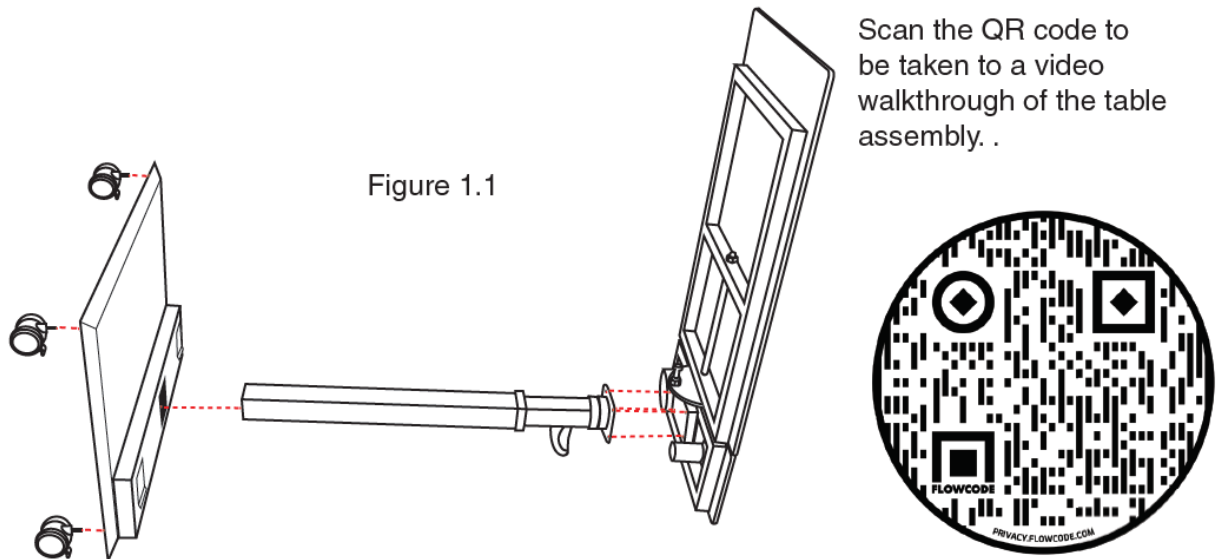

## ASSEMBLY

**NOTE:** If you discover a problem at any point during the setup and positioning of your lift table, stop and contact a WiseLift representative at 1-877-782-2010. To avoid personal injury and/or product damage, do not plug the unit into the electrical outlet until the problem is corrected and do not attempt to fix electrical problems by yourself. The WiseLift Table base has 3 position options. If you are using the wheels, for rolling the table around, only the center position is used for maximum stability. For fixed locations with a sofa or lift chair, the left or right positions accommodate either left or right side configurations. This puts most of the base under the furniture for maximum support and low profile. The WiseLift Table will be shipped to you in 3 main pieces, the table base, column and top. It is recommended that you place these peices on the floor in the orientation shown in Figure 1.1. To assemble the WiseLift Table, follow these instructions:

1. If you intend on using casters, install them in each corner of the base. Screw them in hand tight and then finish with the provided crescent wrench.
2. Install column in desired port (center port if using casters) cover the unused collumn ports with the two covers provided(fig. 1.2)

Figure 1.2


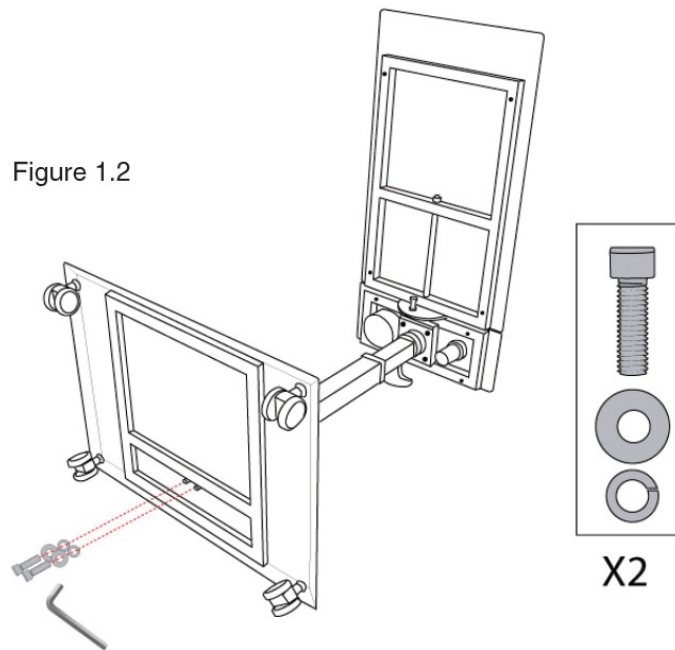



Figure 1.3

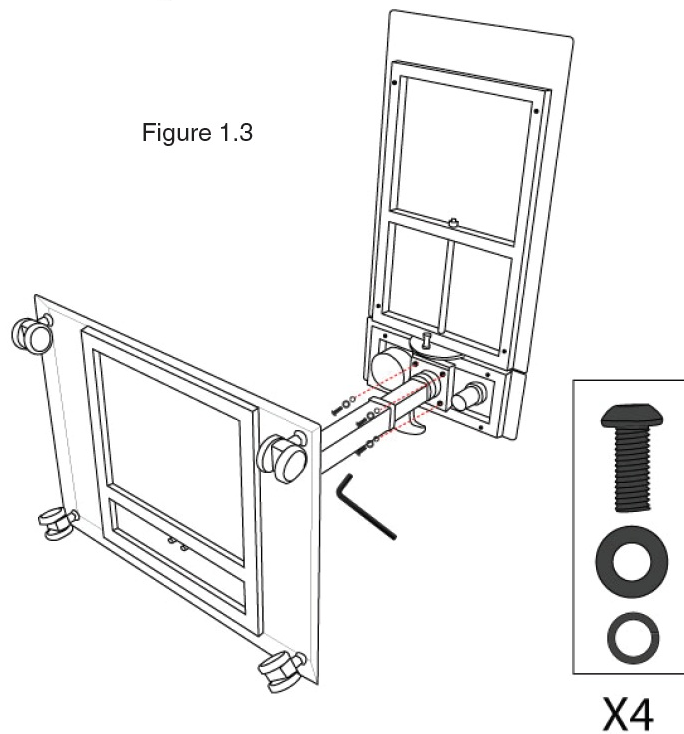

3. Line up table top and attach using the provided screws, and using the allen wrench to tighten the screws down. (fig 1.3)
4. Lift the table upright and confirm that all parts are tight and functioning properly.
5. Slide the lift handle into the opening on the side of the column, lining it up with the holes on each side. Insert the bolt through the side of the column, through the handle and out the other side, and tighten the nut on the other side. (fig 1.4)
6. Move the table to the desired location.
7. Check to see that it's stable in its final location. If your table is unstable, consider removing the wheels and placing the weight of the furniture on the table base.  
You may also need to move the base location of the column. The middle base location is for maximum stability. The other 2 base location are only to be used when the furniture weight is placed on the table base.
8. Once table has been constructed properly, add the cup holder and usb port into their correct spot.  
For easeiest construction guidance please view our video tutorial. Just go to YouTube.com and search

“WiseLift Table Assembly”.

Figure 1.4

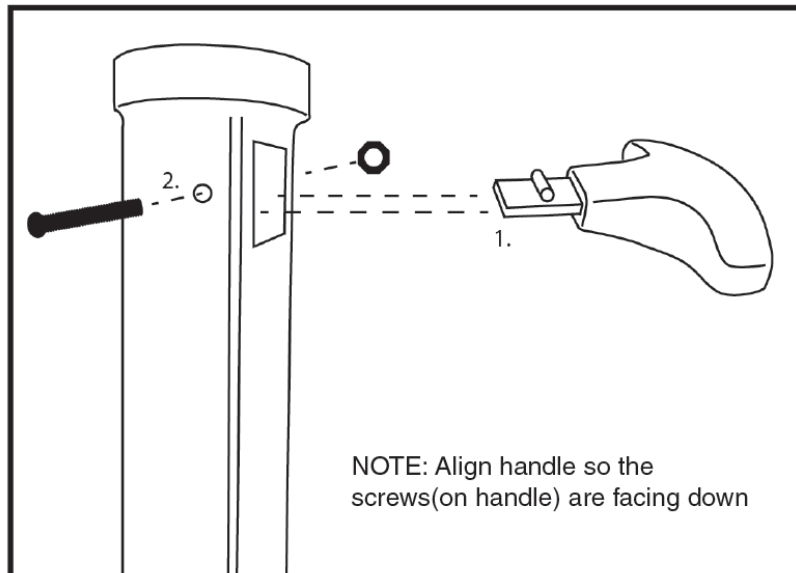

## OPERATION

There are certain precautions that should be taken during the operation of your table. Read and follow these precautions carefully in order to ensure safe table operation and to prevent injury and/or product damage.

- Plug the power cord directly into a surge protector, then into the wall outlet. Do not use a standard extension cord!
- Periodically check cord for visible damage.
- Keep children and pets away from all moving parts while operating the table.
- Do not allow children to play on or operate the table.
- Avoid pinch points, such as the lift and swivel mechanism. Keep hands and feet clear of these areas.
- Do not sit or lean on the table

## LIFT CHAIR / RECLINER

Slides under almost any lift chair or recliner and easily swivels out of the way for full motion of the lift chair. It is recommended to use the base position that is at the front of your chair to allow the base to be out of the way. Do not use the wheels.

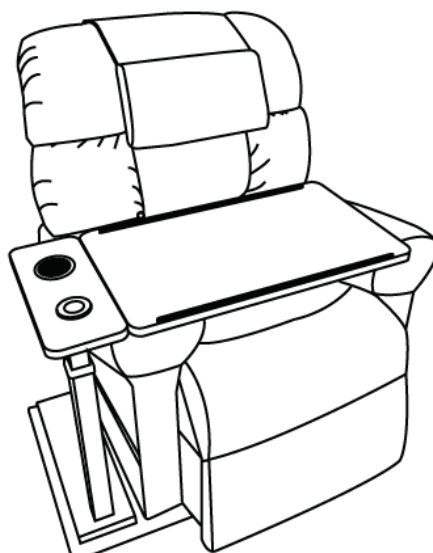


Figure 2.1

### **SOFA / LOVE SEAT**

Perfect for any couch, love seat or sofa with height adjustability. Use the base location that is most comfortable when using the table. You can place the leg of the sofa on the base for added stability.


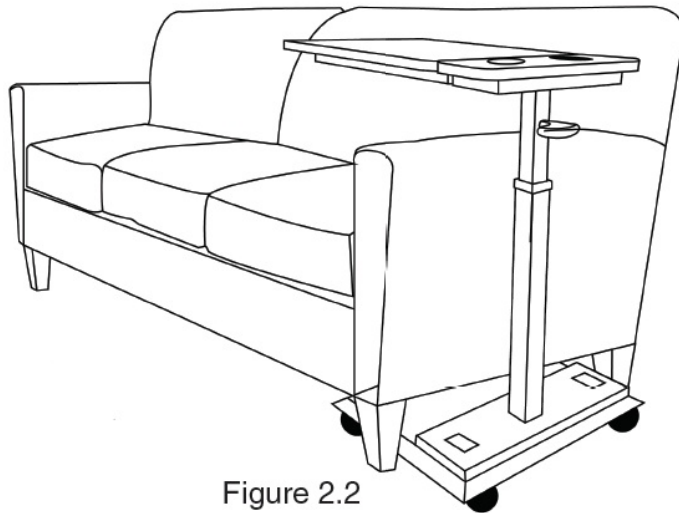


Figure 2.2

### **BED**

A must have bedside companion. Easily push the WiseLift table under the bed for a quick work desk or convenient breakfast tray. It is recommended to use the wheels in this orientation for ease of use.

Figure 2.3

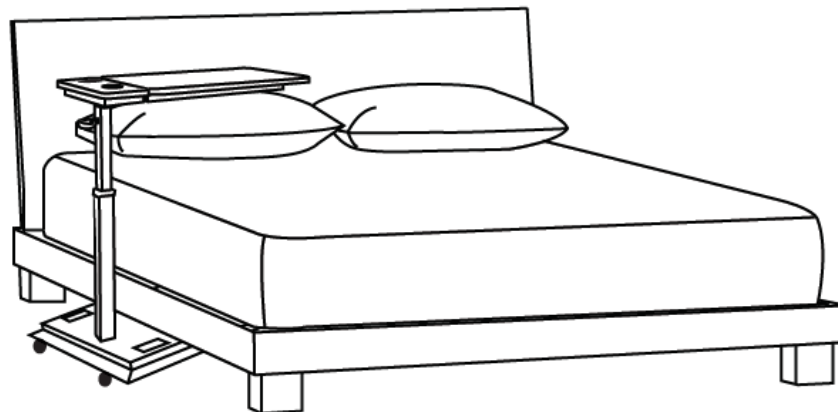

### **STANDING DESK**

Maximize your work efficiency and cut back on the sedentary lifestyle of sitting at your desk all day. Only use the center base location when using as a standing desk. You may use the wheels with the standing desk or remove them for added stability.





Figure 2.4



Figure 3.1

### ADJUST TABLE HEIGHT

The WiseLift Table accommodates sitting in the lowest chair, to laying in the highest pillow top bed, and even standing up. The height lever allows exact height selection, while the internal air piston locks in your selection. Lift and hold the black lever on the outside of the column to quickly adjust the table top up or down. When lowering the table top only press down on the stationary side table between the cup holder and USB hub. DO NOT push down on the rest of the table top or risk breaking the table.

### ADJUST TILT ANGLE

The WiseLift Table tray tilts to your custom angle for reading or using a laptop computer. Raised ledges assure your items do not slide off. The table portion with the cup holder and charging ports does not tilt. Pull the plunger as shown in figure 3.2 and adjust to the desired angle. Ensure the plunger is locked in position before using the table top.



Figure 3.2



Figure 3.3

### SWIVEL 360°

When the WiseLift Table base is under a lift chair or sofa, just swivel it and it's out of the way. When the castors are installed, lock the swivel function and DO NOT use to assure stability.



Figure 3.4



For using wheels.

Figure 3.5

### CHANGE BASE LOCATION

The WiseLift Table base has 3 position options. If you are using the wheels, for rolling the table around, the center position is used for maximum stability. For fixed locations with a sofa or lift chair, the left or right positions accommodate either left or right side configurations. This puts most of the base under the furniture for maximum support and low profile. Simply follow the assembly instruction to re-install the column in a different location.

### NEED HELP?

If at any time you have a questions about anything in this manual, contact a WiseLift representative at 1-877-782-2010. Please have the model number, and nature of the problem when calling.

Please find us on YouTube to learn more about your table and how to operate it.

Your table will require routine maintenance checks. You can perform some of these checks, but others may require assistance from an authorized provider. By following the maintenance checks in this section as scheduled, you can help ensure your table gives you years of trouble-free operation. If you have any questions regarding your lift table's care, contact your authorized provider.

## **CLEAN AND CARE**

- Unplug the table during cleaning
- Do not allow liquids to come into contact with electrical components

The Table should not be allowed to come in direct contact with any type of heat source (e.g., a space heater or cigarette). Personal injury or fire damage may occur.

## **ELECTRONICS CARE**

- Inspect all wiring to make sure they are not damaged or frayed.
- If damage is present, unplug the lift table and contact your authorized provider for service.
- Keep all electronics free from moisture and temperature extremes. WiseLift tables are intended for indoor use only!

Even though the lift table has passed the necessary testing requirements for ingress of liquids, you should keep electrical connections away from sources of dampness, including direct exposure to water. Check electrical components frequently for signs of damage or corrosion and replace as necessary.

## **Warranty**

WiseLift warrants the original purchaser that its products under normal use, in accordance with operating instructions, will perform to the specifications published by WiseLift at the time of purchase, for the following periods, from the date of purchase by the consumer:

### **1 YEAR WARRANTY**

Includes table frame, swivel and tilt mechanisms. WiseLift offers a one year warranty on the table to the original purchaser under normal use, in accordance with the operating instructions. WiseLift warrants that those products, in accordance with instructions for use, will perform to the specifications published by WiseLift at the time of purchase. In the event of a defect reported by the consumer within the applicable time period, if any, WiseLift will, at its option, replace or repair the defective products at no charge to the consumer for parts. WiseLift is not responsible or obligated to pay for labor, service calls, shipping, and/or any other charges incurred for repair of the product. All transportation costs and shipping damage incurred while submitting parts for repair or replacement are the responsibility of the purchaser.

### **WARRANTY EXCLUSIONS**

In no event shall WiseLift be liable for any costs or damages, including lost profits or revenues, or any consequential or incidental damages incurred by the consumer. Damages or malfunction resulting in unauthorized repairs will not be subject to factory warranty. Other exclusions to the warranty include damages caused by contamination, abuse, misuse, battery fluid spillage or leakage, improper operation or storage. Some state/provinces do not allow limitations on how long an implied warranty lasts, or the exclusion or limitation of special, incidental or consequential damages, so the limitations and exclusions may not apply to you. This warranty gives you specific legal rights. You may also have other rights which vary based on where you live.

## HOW TO GET SERVICE

The consumer must, within the applicable time period, contact WiseLift at 1-877-782-2010. Have your part number, invoice or receipt and description of defect ready. Product should be packaged and shipped carefully. Include company name, contact name, address, phone and email information. You can also contact WiseLift through email [Info@WiseLiftchair.com](mailto:Info@WiseLiftchair.com).


## SPECIFICATIONS

Swivel Rotation	<b>360°</b>
Tilt Angle	<b>0° / 30° / 45°</b>
Table Height Max and Min	<b>28 in / 40 in</b>
Clearance with Wheels	<b>3 in</b>
Clearance without Wheels	<b>1 in</b>
Side Table Measurements	<b>6 X 15.75 in</b>
Tray Table Measurements	<b>25.5 X 15.75 in</b>
Combined Table Top Measurements	<b>31.5 X 15.75 in</b>
Base Measurements	<b>24 X 18 in</b>
Clearance Under the Table	<b>1 in</b>
Diameter of Cup Holder	<b>3.5 in</b>
USB Output	<b>2.0 A</b>
Cord Length	<b>5.5 ft</b>
Table Capacity	<b>20 LBS</b>
Net Weight	<b>46 LBS</b>
Gross Weight	<b>51 LBS</b>
Shipping Carton Measurements	<b>36 X 24 X 6 in</b>
UPC	<b>857193006728</b>
COO	<b>China</b>
HTC	<b>94029000</b>

2200 Big Town Blvd. #180 Mesquite, TX 75149 [www.wiselift.com](http://www.wiselift.com)

© 2021 Letronics Inc.

## Documents / Resources

	<p><a href="#">WISELIFT WLT400 Rolling Work Station Swivel Table</a> [pdf] User Manual</p> <p>WLT400, Rolling Work Station Swivel Table, WLT400 Rolling Work Station Swivel Table, Swivel Table</p>
---	---

## References

- [WiseLift 450 - WiseLift](#)