



# WSAudio RFM015 Radio Module Owner's Manual

[Home](#) » [WSAudio](#) » WSAudio RFM015 Radio Module Owner's Manual 

## WSAudio

Integration manual  
for RF Module 15 (RFM015)

### Contents

- [1 RFM015 Radio Module](#)
- [2 Documents / Resources](#)
  - [2.1 References](#)

## RFM015 Radio Module

Name	Carsten Quaade Jensen
Department	R&D HW WSI DK LY Nyemoellevej 6 DK-3460 Lynge
Phone	+45 20 75 08 37
E-mail	<a href="mailto:carsten.jensen@wsa.com">carsten.jensen@wsa.com</a>
Website	<a href="http://www.wsa.com">www.wsa.com</a>
Date	2023-04-25

The “RFM015” radio module contains two radio transceivers running at 10.6 MHz and 2.45 GHz which are implemented in a single hardware platform.

The 10.6 MHz radio consists of a nearfield inductive magnetic transceiver. It uses time direct sequence /duplex and runs on a single channel with phase modulation. The signal is transmitted by a coil antenna. The intended use of this radio is to exchange data between two hearing aids or to communicate with a proprietary accessory.

The 2.45 GHz radio is mainly used for Bluetooth® Low Energy Radio communication. In addition to that, the receiver is also capable of proprietary communication modes. The transceiver is connected to a PCB-integrated

antenna. The intended use of this radio is to exchange data between hearing aids and Bluetooth accessories. The main part of the module is a set of analog and digital ASICs which contain both radios. These ASICs are mounted on a flex PCB. In addition to that the module comprises the coil antenna for the nearfield inductive magnetic system, one crystal and an EEPROM memory. The antenna for the 2.45 GHz radio is integrated in the flex PCB of the module. The matching circuitry between the radio ASIC and the antenna is also on the flex PCB. On the flex PCB additional components are mounted and connected to the radio module. These include microphones, the connector to the external speaker, push buttons, the telecoil and the battery connector. The system is powered by a lithium ion battery, all necessary voltage regulators are included in the module. The PCB with the module and all other components of the hearing aid are arranged in a plastic frame that is not user serviceable or user modifiable. The outer housing may be replaced by field service, but this is not relevant for the wireless module.

The module is thus not installed but instead assembled at the same time as the host is. The position of components and interconnection through the PCB are decided during the project phase for the different hosts and following the best practice of the engineering team in order to assure an optimal integration of the module.

Several validation steps are done through the development to assure the full compliance with all the regulatory requirements.

The user guide must contain the HVIN, the FCC ID and the IC ID:

HVIN: RFMO15

Contains FCC ID: 2AXDT-RFM015

Contains IC ID: 26428-RFM015 and following statements:

(English version)

This Class B digital apparatus complies with Canadian ICES -003.

Changes or modifications made to this equipment not expressly approved by the legal manufacturer may void the FCC authorization to operate this equipment.

This device complies with Part 15 of the FCC Rules and with ISED's licence-exempt RSSs.

Operation is subject to the following conditions:

- This device may not cause harmful interference, and
- This device must accept any interference received, including interference that may cause undesired operation.

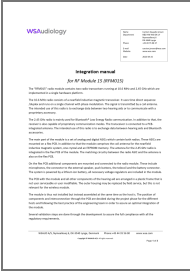
This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

The radiated output power of the device is far below the FCC and ISED radio frequency exposure limits.

For body worn operation, this device has been tested and meets the FCC RF exposure guidelines when used with the legal manufacturer's accessories supplied or designated for this product. Use of other accessories may not ensure compliance with FCC RF exposure guidelines.

**WSA**audiology

	<b>WSAudiology RFM015 Radio Module</b> [pdf] Owner's Manual RFM015, RFM015 Radio Module, Radio Module, Module
--	--

References

-  [Wonderful Sound for All - WS Audiology](#)
- [User Manual](#)

Manuals+ Privacy Policy

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.