



XDcuff 09-634504 Reusable Cuff Securement Straps User Manual

[Home](#) » [XDcuff](#) » XDcuff 09-634504 Reusable Cuff Securement Straps User Manual 



09-634504 Reusable Cuff Securement Straps
User Manual



For use with Stryker® Power-PRO XT® cots
REUSABLE CUFF SECUREMENT STRAPS

Contents

- [1 09-634504 Reusable Cuff Securement Straps](#)
- [2 INSTALLATION OVERVIEW](#)
- [3 WRIST & ANKLE POSITIONS](#)
- [4 CUFF SECUREMENT STRAP – WRIST & ANKLE POSITIONS](#)
- [5 LEFT ANKLE POSITION](#)
- [6 RIGHT ANKLE POSITION](#)
- [7 Documents / Resources](#)
 - [7.1 References](#)
- [8 Related Posts](#)


09-634504 Reusable Cuff Securement Straps

PACKAGE OF 4
UNIVERSAL STRAPS
PRODUCT REF 104

The XDcuff ® Cuff Securement Straps (CSS), was developed to function as part of a system that enables you to quickly limit a patient's movement, on the stretcher, should the need exist.


Made from black nylon with DuraGrip® hook & loop fasteners, the CSS is attached to the stretcher's frame. Once in position, they secure the XDcuff ® disposable or reusable restraints, (Product REF 102, 103,101) to the stretcher and store them for rapid access.

CUFF SECUREMENT STRAP (CSS)

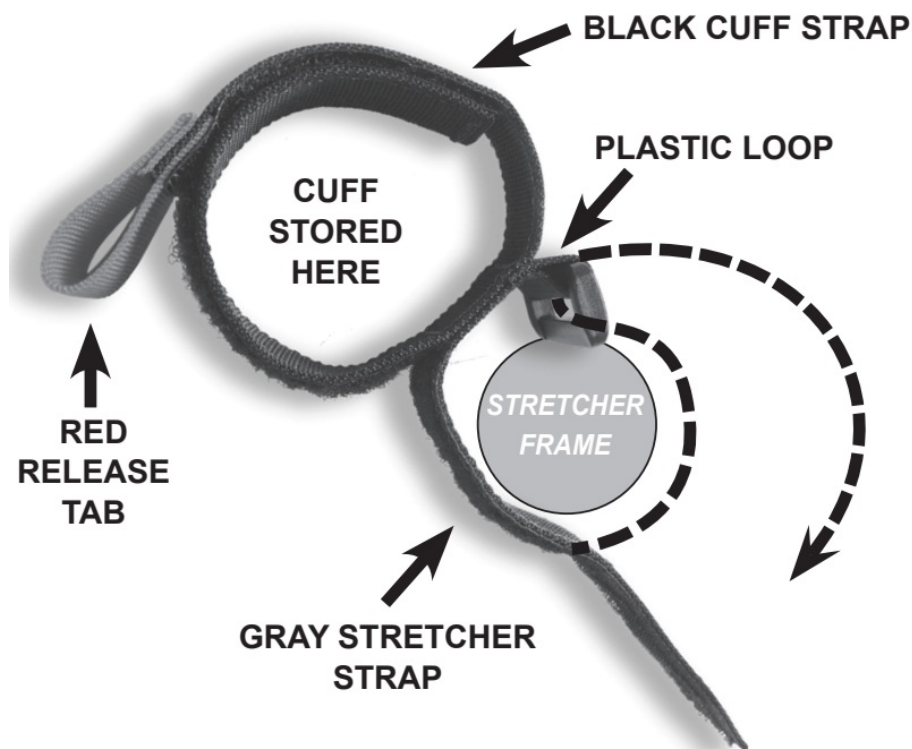


INSTALLATION OVERVIEW

Attaching the Cuff Securement Strap to StretcherAt the four recommended locations (fig-3), wrap the end of the gray stretcher strap around the stretcher the frame (as indicated in the diagram at left), feed it through the plastic loop and back onto itself. Smooth the hoop & loop fastener to secure (fig-1).

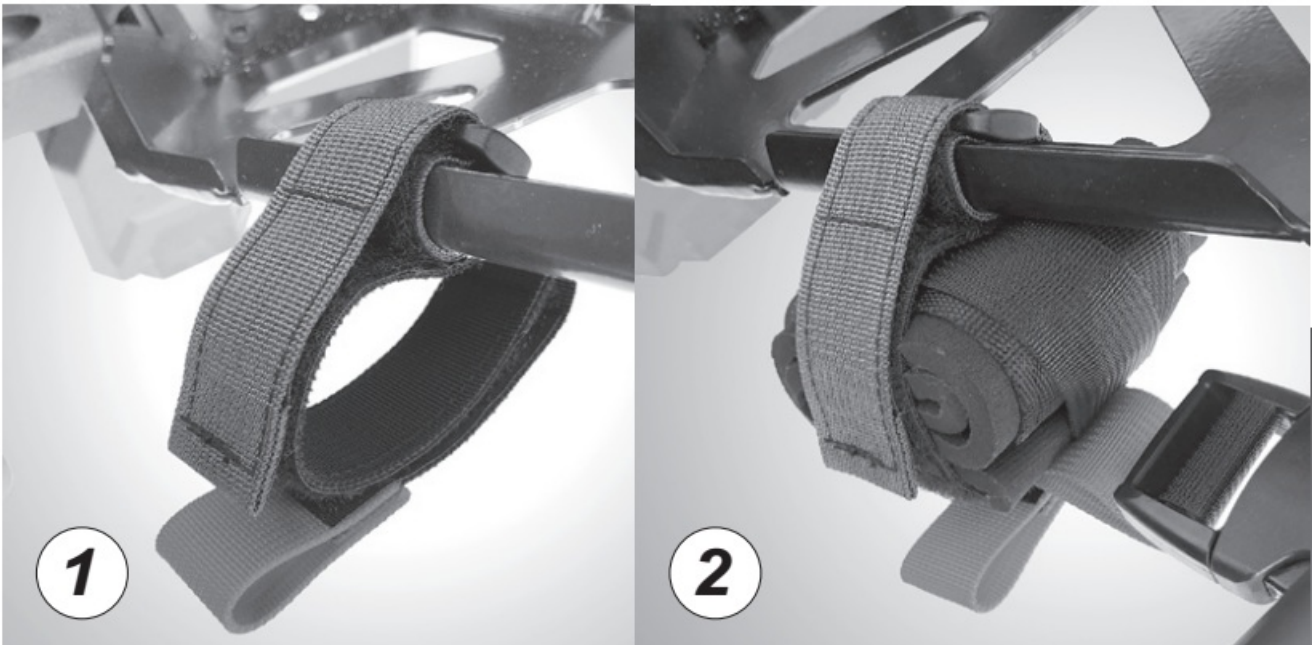


When attaching the restraint to the CSS, pull the red release tab on the black cuff strap and open the hook & loop fastener. In the opening, place a folded (or packaged) restraint and re-attach the cuff strap. Smooth the hook & loop to hold the restraint in place. Your XDcuff ® restraint is now ready for use (fig-3).

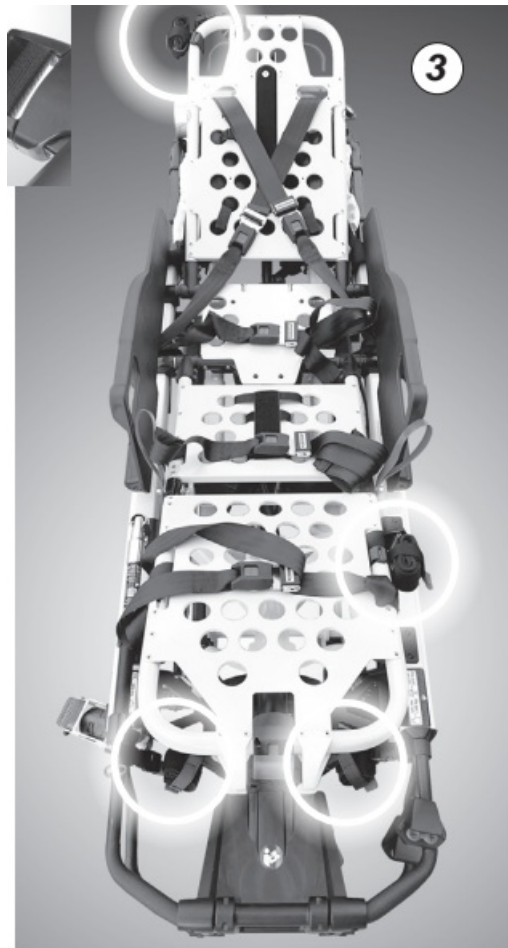


WRIST & ANKLE POSITIONS

RIGHT WRIST POSITION – This position is located at the top of the stretcher's head lift section, on the patient's right side (fig-3).

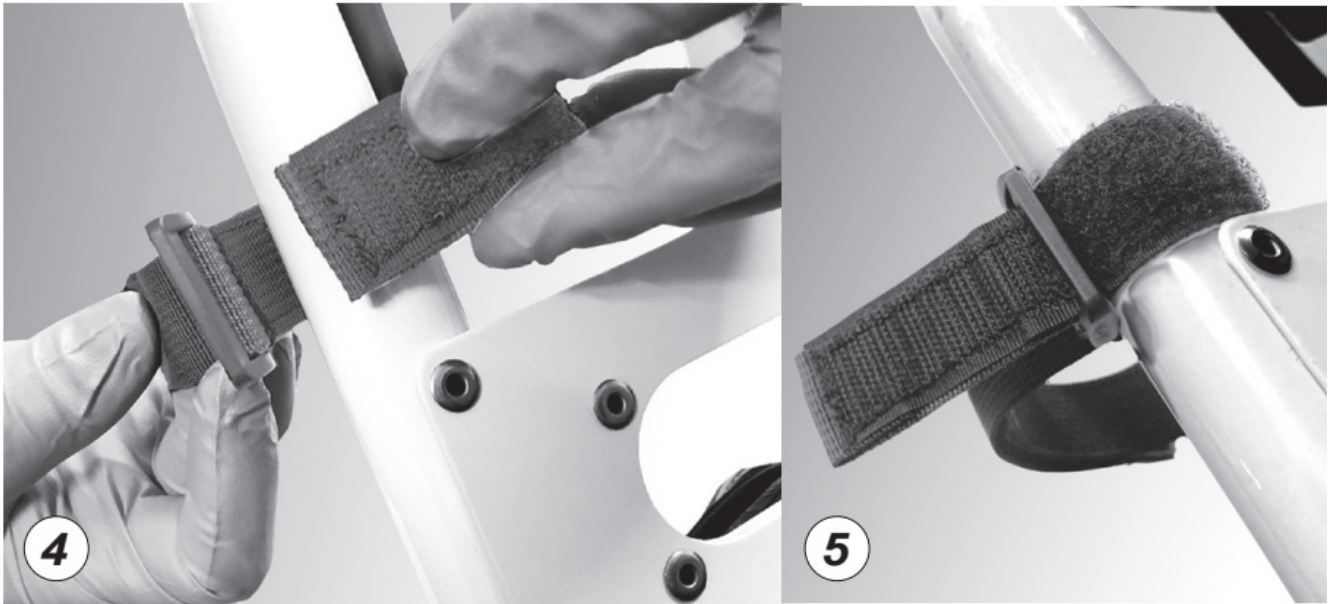



- 1) Wrap the gray stretcher strap around the tubular frame rail, and feed the end back through the plastic loop (fig-4).
- 2) Pull the gray stretcher strap tightly and fasten it back to itself (fig-5 & 6). Completed right wrist position viewed from the back. (fig-7).

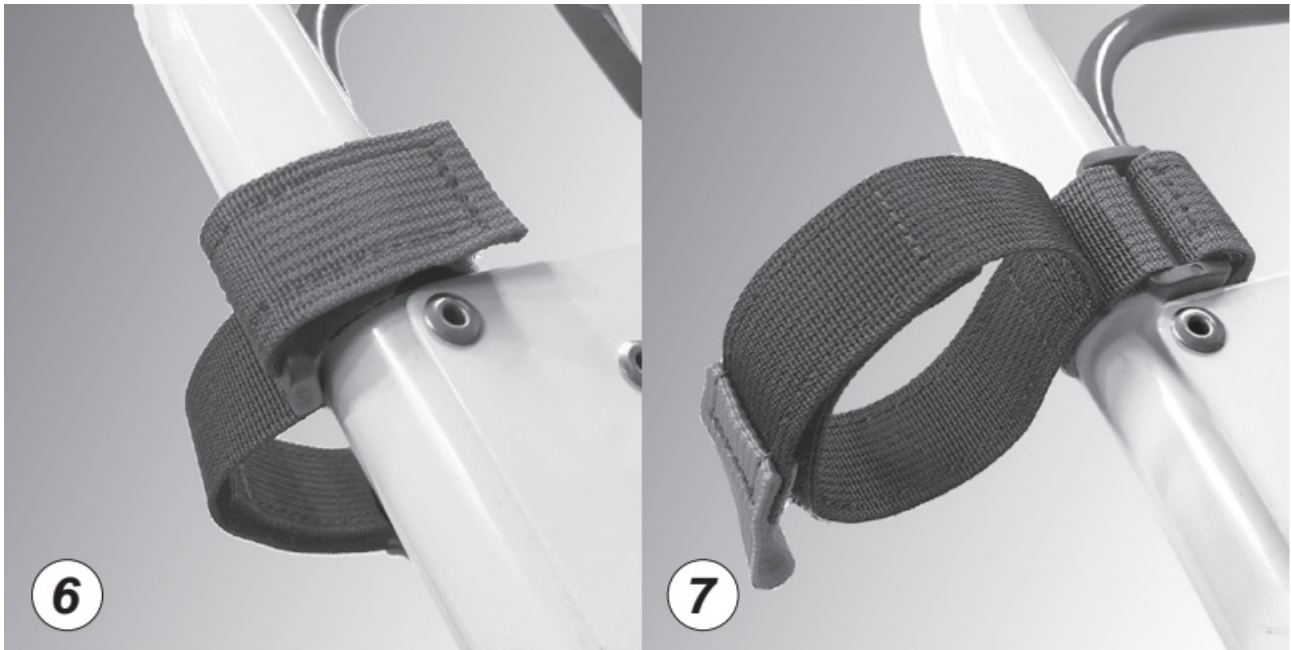

See User Guides packaged with the Restraints for detailed instructions on installing the XDcuff ® system.

CUFF SECUREMENT STRAP – WRIST & ANKLE POSITIONS

LEFT WRIST POSITION – Unlike the right wrist, this position is located on the tubular frame rail on the foot lift portion of the cot, toward the patient's left knee (fig3). 1) Wrap the gray stretcher strap around the frame rail (fig-8), and feed through the plastic loop (fig- 9). 2)

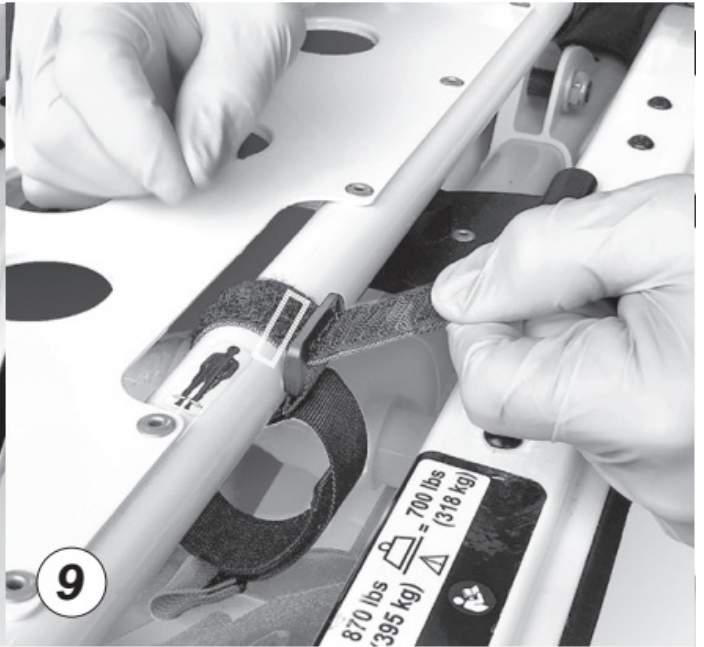
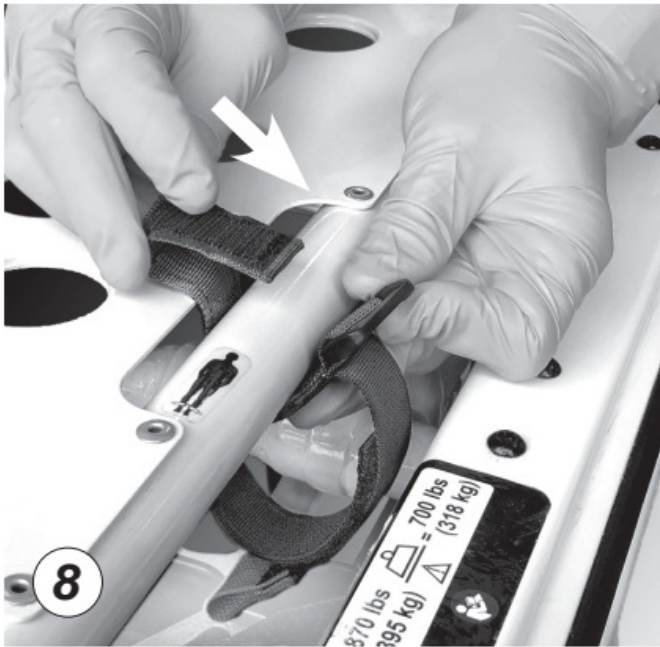


Pull the gray stretcher strap tightly and fasten it back onto itself (fig-10). Spin the stretcher strap up so that you can access the red release tab (fig-11).



LEFT ANKLE POSITION

– The left ankle position is located on the foot-end hitch, at the patient's left side toward the foot. Ensure that each red release tab is pointing towards the life bars/handles for easy access in emergencies.

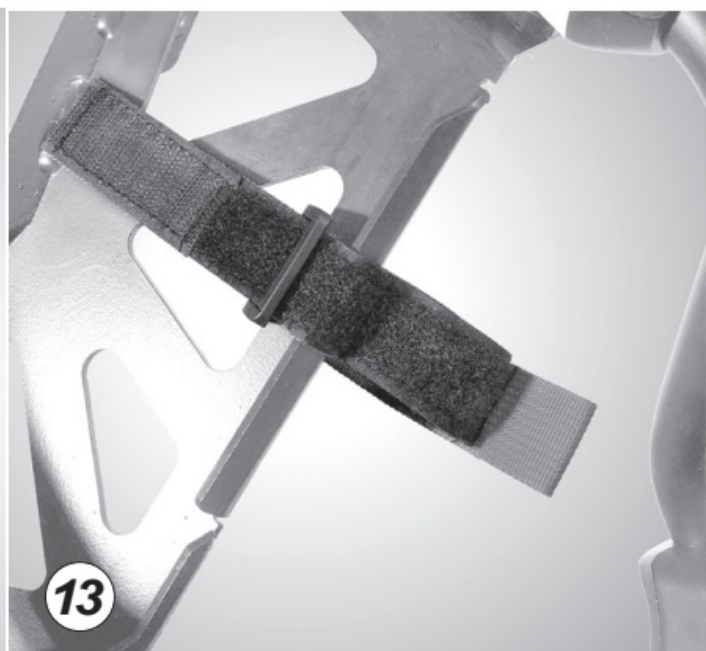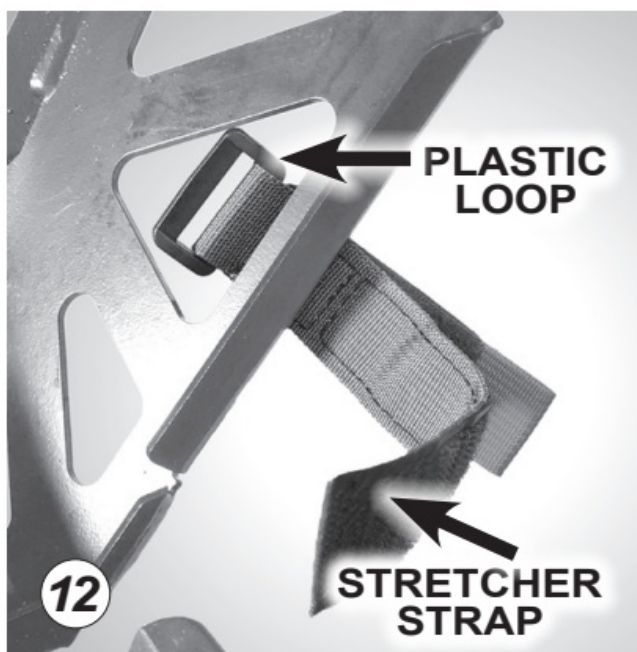

1. Wrap the gray stretcher anchor around the foot-end hitch (fig-12).
2. Feed the gray stretcher strap through the plastic loop (fig-13).




3. Pull the gray stretcher strap tightly and fasten it back to itself (fig-14). Completed left ankle position with black cuff strap secured (fig-15).


RIGHT ANKLE POSITION

– The right ankle position is located on the opposite side of the foot-end hitch.

1. Wrap the gray stretcher strap around the foot-end hitch (fig-16)

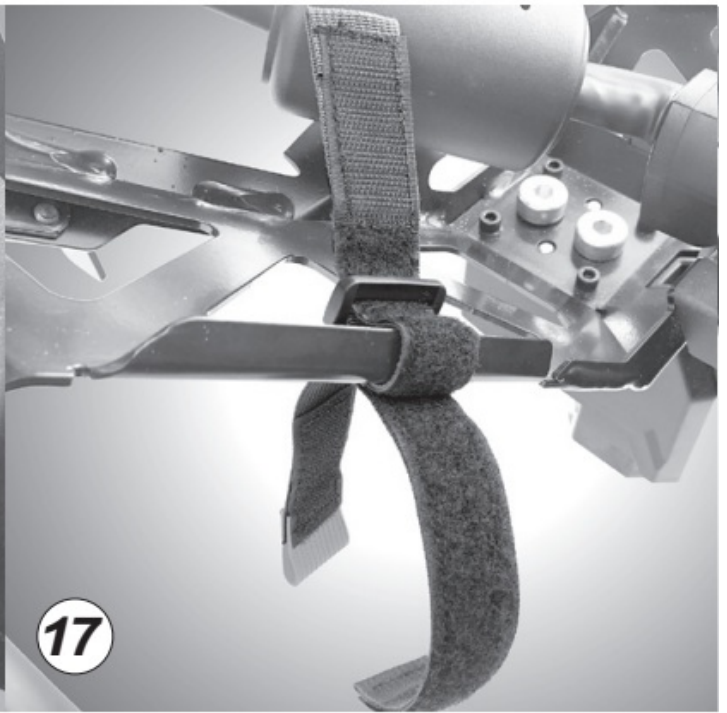

2. Feed the gray stretcher strap through the plastic loop (fig-17)
3. Pull the gray stretcher strap tightly, and fasten it back to itself (fig-18).

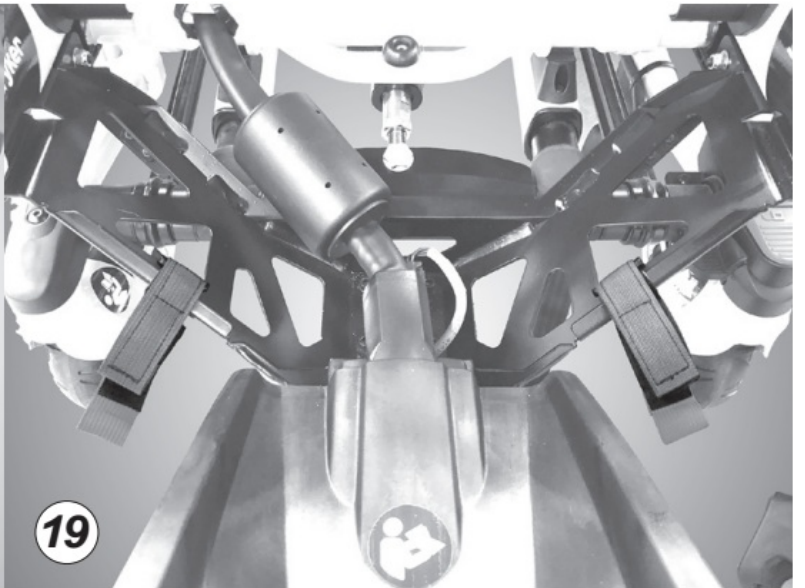

Completed left and right ankle position with black cuff straps secured (fig-19).
Visit our YouTube channel (XDCuff®) for helpful instructional videos for installing the Cuff Securement Straps, Wrist, and Ankle Restraints.

XDcuff®

www.xdcuff.com
david@xdcuff.com
386 275 8361



Documents / Resources

	<p>XDcuff 09-634504 Reusable Cuff Securement Straps [pdf] User Manual 09-634504 Reusable Cuff Securement Straps, 09-634504, Reusable Cuff Securement Straps, Cuff Securement Straps, Securement Straps, Straps</p>
---	--

References

- [Emergency Medical Supplies & Equipment | Medical Products | QuadMed](#)
- [Stretcher Integrated Limb Restraint System | Daytona Beach | XDcuff](#)

Manuals+.