



## XPLORoxygen X-PLOR Portable Concentrator Resources User Guide

[Home](#) » [XPLORoxygen](#) » XPLORoxygen X-PLOR Portable Concentrator Resources User Guide 

# *XPLO Roxygen*

X-PLOR Portable Concentrator Resources



User Guide

[XPLORoxygen.com](http://XPLORoxygen.com)



## Contents

- [1 X-PLOR Portable Concentrator Resources](#)
- [2 Documents / Resources](#)
- [3 Related Posts](#)






## X-PLOR Portable Concentrator Resources



### CHECK LIST

Locate the following items in your X-PLOR® box to get started:

	X-PLOR Unit
	Carrying Bag
	Replaceable Cartridge (found in Mylar bag)
	Nasal Cannula

	AC Power Supply
	DC Power Supply
	Handle (Attaches to Carrying Bag)
	Shoulder Strap (Attaches to Carrying Bag)
	8-Cell Battery


**DO NOT TURN ON THE X-PLOR UNTIL YOU REACH STEP 4.**

It is quick and easy to get your X-PLOR started, but please read the entire User Manual before using.



**REMOVE DUST CAP & INSERT CARTRIDGE**

- Remove cartridge from mylar bag.
- Remove the dust cap from the cartridge.
- Then insert the cartridge all the way into the unit so the handle lays flat.

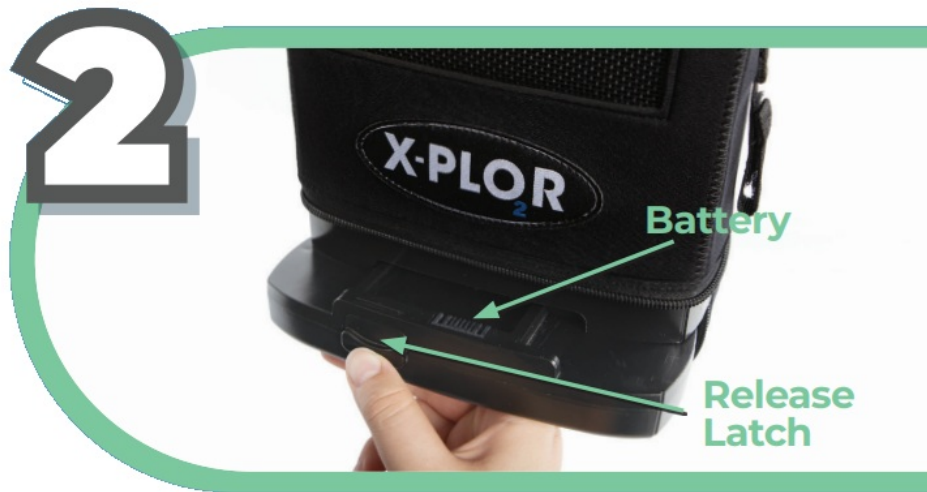

## LOCATE THE BATTERY

- Insert your X-PLOR into the carry bag. When the X-PLOR is correctly inserted into the bag, the LCD screen is seen through the plastic window and the air intake and exhaust are aligned with mesh panels.
- To attach the battery, slide it on until the latch is in the locked position and zip the bag.
- To disconnect the battery, simply push the release latch down and slide the battery towards you. 55



## CHARGE THE BATTERY

- Place in a well-ventilated area to fully charge the battery before using your X-PLOR, which can take up to 6 hours for the 8-cell battery.
- To charge, insert the AC power connector into the battery's power receptacle and the other end into an electrical outlet.






#### TURN ON & SET OXYGEN FLOW

- Press the POWER button for 2 seconds to turn on your X-PLOR.
- When powered on, position the unit upright to warm-up (Warm-up time is less than 2 minutes.)
- Use the UP/DOWN ARROWS to set the oxygen flow level, as specified in your oxygen prescription.



#### NASAL CANNULA

- A nasal cannula is included in your shipment box.  
Additional cannulas must be purchased separately.
- Connect the nasal cannula to the oxygen output nozzle located on top of the X-PLOR.
- Place the other end in your nostrils. Oxygen will be delivered through the nasal cannula with each breath.
- Remember to breathe through your nose as normal.

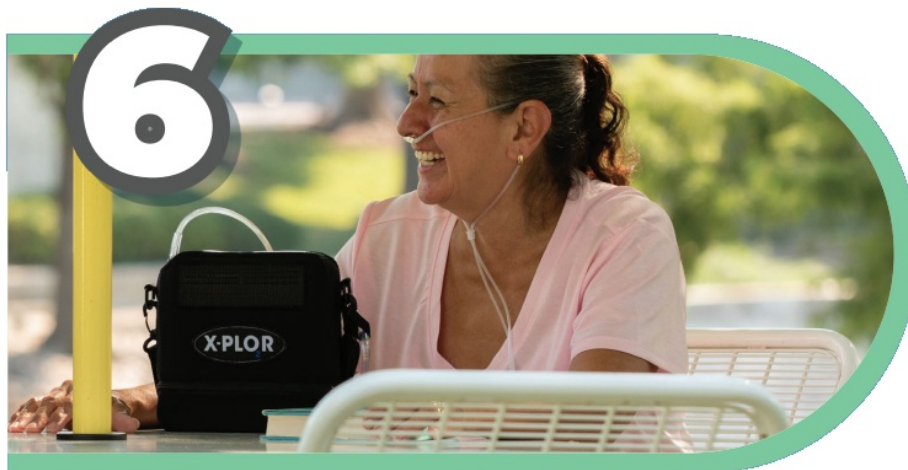

## GO X-PLOR!

- Attach the shoulder strap and handle to your X-PLOR device and enjoy your new found mobility!

**LIVE LIFE. BREATHE EASY.  
GO X-PLOR.**



### TRAVEL WITH X-PLOR

The X-PLOR lets you travel with ease.

It's perfect for carrying on buses, trains, planes, and cruise ships! Now with X-PLOR supplemental oxygen and direct-charge batteries, exploring has never been so easy.

**X-PLOR IS FAA COMPLIANT.**



**XPLORoxygen.com | (877)-GO-XPLOR**  
©2022 Belluscura LLC. All Rights Reserved.

## Documents / Resources



[XPLORoxygen X-PLOR Portable Concentrator Resources](#) [pdf] User Guide  
X-PLOR Portable Concentrator Resources, X-PLOR, Portable Concentrator Resources, Conce  
ntrator Resources, Resources